

Loop^{SORG} RS

SPARE PARTS LIST



Dear customer,
in the following pages you will find a complete spare part list with all parts used to build our wheelchairs.

To ensure a accurate delivery of spare parts it is vital to provide the following information fro the identification label in addition to the Article-Number (Art.-No.).

To find the identification label on the chair please look up the following link for the exact location:

http://www.sorgrollstuhltechnik.de/fileadmin/user_upload/pdf/Download/Finding_identification_label.pdf

1. Serial number of the chair

The serial number enables us to identify the configuration of delivery of the specific chair in.

2. Seat width, seat depth and back height

The measures on the identification label are specified for the delivery configuration. If you changed any of these measures after delivery e.g. by growing the chair, please specify the actual new measures.

3. Lower leg length

We need the LLL to decide for the right version of several attachments which are connected to the LLL.

4. Frame colour

Please indicate the frame colour of the chair when ordering a powder coated spare part. Please note also if the finish alternative is shiny or matt.

5. Any other change after delivery (e.g. attachments)

We cannot know about changes after delivery. If you are not sure if such a change has any influence on the needed spare parts, please give us some details about it.

List of abbreviations

Pcs.	piece number
item no	item number to order specific spare part
Q/1	quantity per wheelchair/assembly group
SW	seat width
SD	seat depth
BH	back height
alt.	alternative
l/r	left/right

Handling annotations

Only items with item number are to order.

right / left are in relation to driving direction.

If you need further assistance, please contact us:

SORG Rollstuhltechnik GmbH & Co.KG

Benzstraße 3-5

68794 Oberhausen-Rheinhausen

Fon +49 7254 - 9279 0

Fax +49 7254 - 9279 10

info@sorgrollstuhltechnik.de

<http://www.sorgrollstuhltechnik.de>

Opening times:

Monday – Friday 8.00 am – 4.30 pm

Technical alterations and misprints excluded.

Prices on demand.

Contents

Frame

Frame	1
Rigid frame	3
Anti tipper	4
Anti-tipper for 12 and 16 inch wheels	6
Step tube for 20, 22 and 24 inches	7
Step tube for 12 an 16 inch wheels	8

Leg support

Leg support	10
Adjustable foot plate	15
Heel edge	27
Heel support	28
Calf rest	29
Calf belt	31

Seat

Seat	33
Seat extension	34
Seat cushion	35
Gas pressure spring seat tilt	37
Abduction wedge	38
Therapy table, version 1, hinged sideways	39
Therapy table, version 2, fixed to the arm pad	40

Back

Back, adjustable in angle by raster	41
Back with gas pressure spring adjustment	43
ERGO back shell	44
Back cushion	45
Curved back plate	46
Back extension	48
Seat cover, adjustable	49
Stabilizer bar + push bar holder	50
Push bow, adjustable	51
Head rest	54
Side truss pads STANDARD	55
Side truss pads ERGO	56
Side guard FIX for ERGO back	57
Connection for back shell	58

Contents

Side guard

Side guards Standard	59
Side guards with guidance	60
Side guards ERGO	61
Knee guidance for ERGO Side guards	63
Wheel cover Large	64
Wheel cover small	65
Wheel cover 16 inch	66
Arm rest 240x50mm up to SD32	67
Arm rest 240x50mm from SD34	68
Arm rest 260x80mm	69

Caster wheel

Caster wheel adapter	70
Adapter for lower seat heights	72
Caster wheel fork	73
Caster wheels	74

Rear wheel

Adaptation plate 12 inch	75
Adaptation plate 16 inch	76
12 inch wheels	77
16 inch wheels	78
Wheel base extension	79
Screw adapter	80
Drum brake anchor	81
Rear wheels	82
Tyres	83
Handrims	84
Spoke guards	85
Handrim covers	86
Double handrim Standard	87
Double handrim Drum brake	88
Knee lever brake for 12 inch wheels	92


Brake

Knee lever brake	91
Brake lever extension	93
Braking handle drum brake	94

Options

2-point Hip belt with Velcro and synthetic lock	95
2-point Hip belt with metal lock	96
4-point Hip belt with Velcro and synthetic lock	97
4-point Hip belt with metal lock	98
Butterfly harness	99

Frame

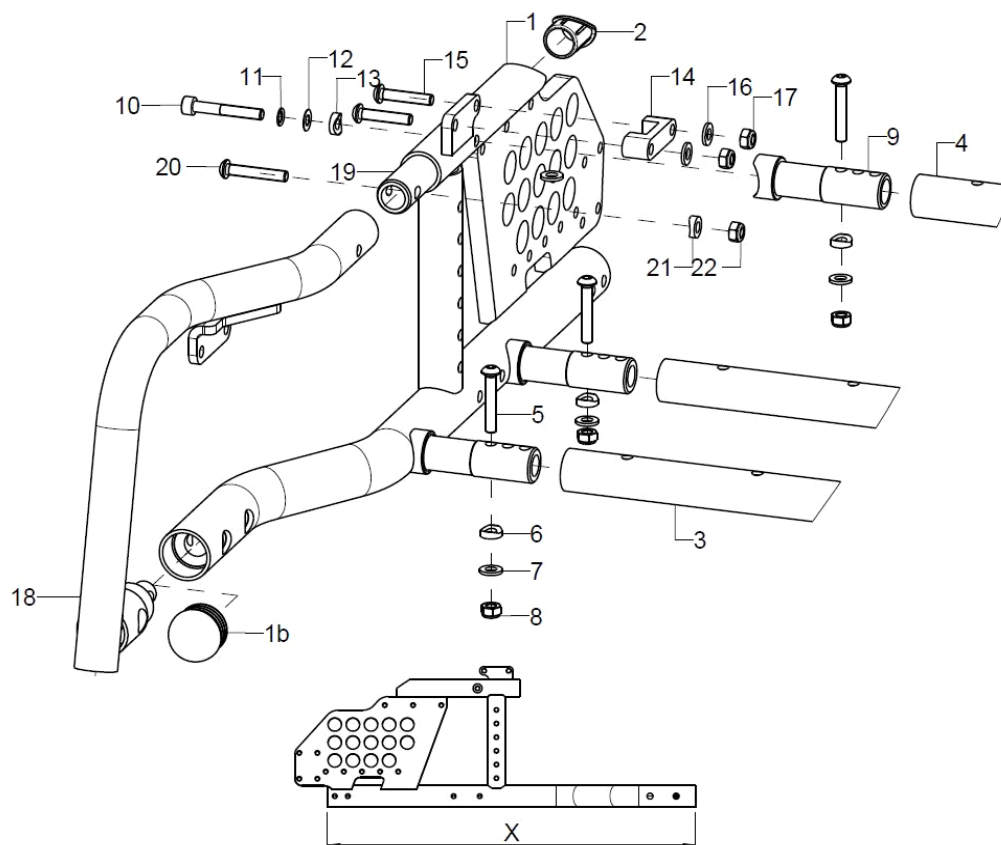


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1+2	1	59760102			59.76.01.00.02.76
o.B.	1	59760101			59.76.01.00.01.76
1+2	1	59760202			59.76.02.00.02.76
o.B.	1	59760201			59.76.02.00.01.76
1b	2	59760103	plug half sphere Ø30		04.63.10.30.01.10
2	1				59.80.10.01.00.80
3	2	58760100			58.76.01.01.00.76
4	1	5834SB	crossbar D=24x2xSW		58.34.01.00.00.69
4-8	3	583499	mounting set crossbar on frame		XXXXXXXXXXXX
5	2		lens head screw with hexagon socket ISO 7380 - M6x35		01.42.08.59.65.00
6	2		saddle washer Ø12/6x2,5		35.04.16.01.00.10
7	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
8	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
9-13	2	583501	crossbar adapter		58.35.01.01.00.71
10	1				01.22.08.60.62.40
11	1		lock washer S6		02.99.10.60.01.10
12	1				02.68.06.00.30.30
13	1		saddle washer Ø12/6x2,5		35.04.16.01.00.10
14-17	2	59760110			59.76.01.10.00.76
15	2		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
16	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
17	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
18-19	1	59760502			59.76.05.00.02.76
o.B.	1	59760501			59.76.05.00.01.76
19	2	59760504			59.76.05.04.00.76




o.B.	2		reinforcement bushing for 12"	59.76.01.12.00.76
20-22	2	583499	mounting set crossbar on frame	XXXXXXXXXXXXX
20	2		lens head screw with hexagon socket ISO 7380 - M6x35	01.42.08.59.65.00
21	2		saddle washer Ø12/6x2,5	35.04.16.01.00.10
22	2		hexagon nut self-locking low form DIN 985 - M6	02.32.00.60.01.20


Rigid frame

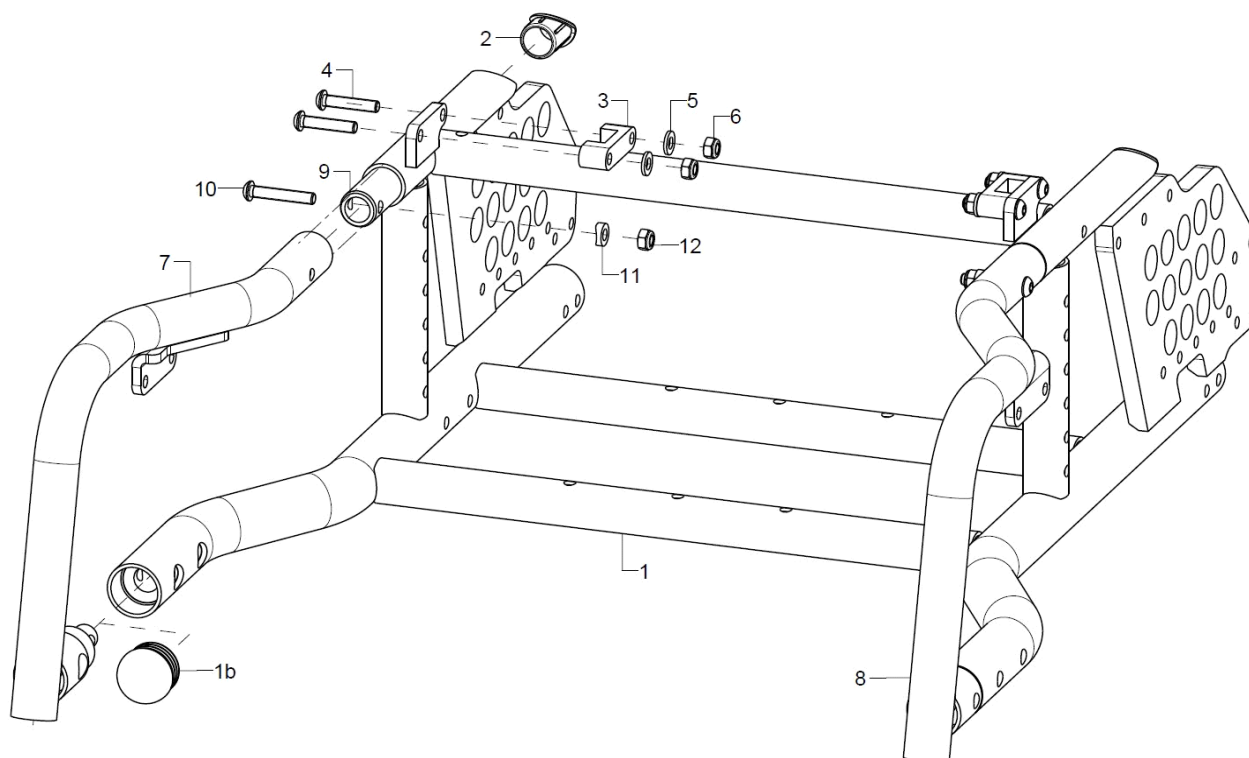

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1+2	1	59761101			59.76.10.00.00.76
1+2	1	59761201			59.76.11.00.00.76
1b	2	59760103	plug half sphere Ø30		04.63.10.30.01.10
2	1				59.80.10.01.00.80
3-6	2	59760110			59.76.01.10.00.76
4	2		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
5	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
6	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
7	1	59760502			59.76.05.00.02.76
8	1	59760501			59.76.05.00.01.76
9	2	59760504			59.76.05.04.00.76
o.B.	2		reinforcement bushing for 12"		59.76.01.12.00.76
10-12	2	583499	mounting set crossbar on frame		XXXXXXXXXXXX
10	2		lens head screw with hexagon socket ISO 7380 - M6x35		01.42.08.59.65.00
11	2		saddle washer Ø12/6x2,5		35.04.16.01.00.10
12	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20

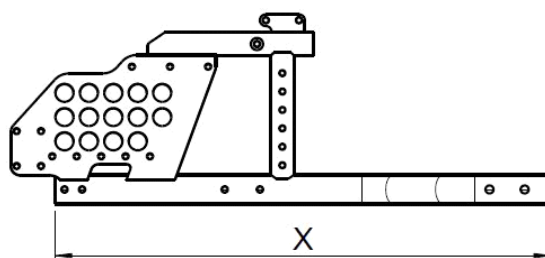


Anti tipper

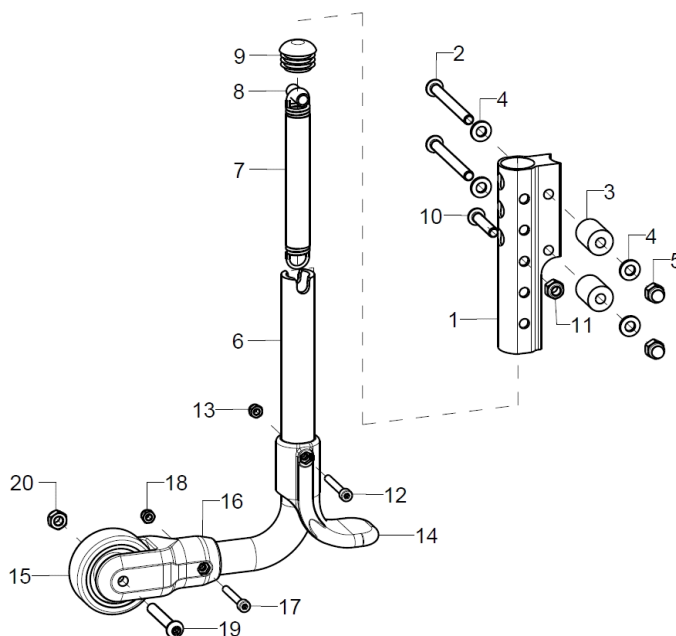

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-20	1	721501	anti-tipper compl., pos. 1 + mounting set A+ anti-tipper bail premounted for 20/22" size driving wheel		72.15.01.02.00.74
altern.	1	721502	anti-tipper compl., pos. 1 + mounting set A+ anti-tipper bail premounted for 24" size driving wheel		72.15.01.03.00.74
altern.	1	721503			
1	1	72150106	anti-tipper holder		72.15.01.06.00.74
2-5	1	7215010699	mounting set anti-tipper holder		XXXXXXXXXXXX
2	2		lens head screw with hexagon socket ISO 7380 - M6x35		01.42.08.59.65.00
altern.	2		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
altern.	2		lens head screw with hexagon socket ISO 7380 - M6x55		01.42.08.63.65.00
altern.	2				01.42.08.64.65.00
altern.	2				
3	2				72.15.76.02.00.76
4	4		washer DIN 125 - A 6,4		02.61.06.41.41.20
5	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
6-20	1	72150001	anti-tipper bail premounted for 20/22" size driving wheels		72.15.01.99.00.74
altern.	1	72150002	anti-tipper bail premounted for 24" size driving wheels		
altern.	1	72150003			
7-13	1	72150107	tension spring, loop 180°, 1/1 centred, 13,5x2x113		04.22.01.41.13.00
8	1		spacer bushing 18,4 mm		45.03.10.00.00.73
9	1		plugs for tube ø22x1-2, black		04.63.10.22.00.10
10	1		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
11	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
12	1		cylinder screw DIN 912 M4x30		01.22.07.38.62.40
13	1				02.33.00.40.01.20
14	1	72150104	tipping lever black		72.15.01.04.00.74
15	1	720004	wheel ø50, width 18, nob width 20, axle ø 6, PU grey		04.52.42.50.00.30
16-20	1	72150101	holder anti-tipper for wheel ø50 mm, incl. mounting set		72.15.01.01.00.74
17	1		cylinder screw DIN 912 M4x30		01.22.07.38.62.40
18	1				02.33.00.40.01.20
19	1		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
20	1		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20




Anti-tipper for 12 and 16 inch wheels

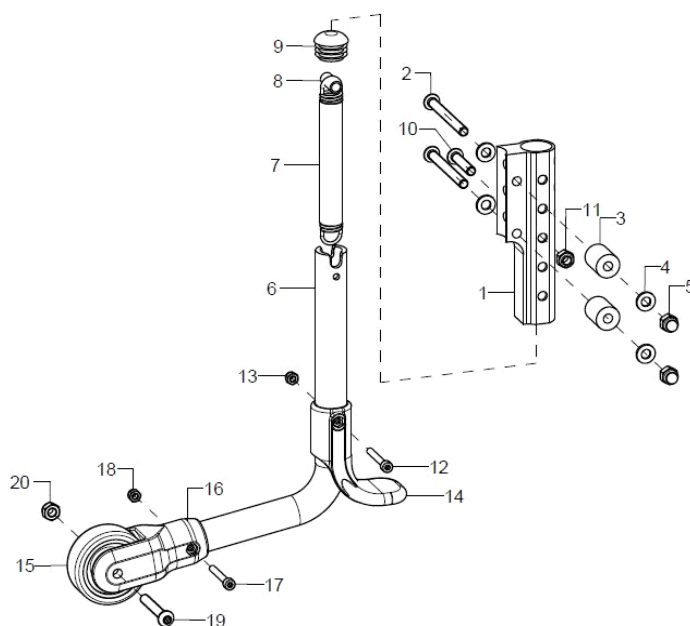


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-20	1	721512			
altern.	1				
1	1	72150106	anti-tipper holder		72.15.01.06.00.74
2-4	1	7215010699	mounting set anti-tipper holder		XXXXXXXXXXXXX
2	2				01.42.08.64.65.00
altern.	2				
3	2				72.15.76.02.00.76
4	2		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
5	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
6-20	1	72150012			72.15.76.99.00.76
altern.	1	72150001	anti-tipper bail premounted for 20/22" size driving wheels		72.15.01.99.00.74
6	1				72.15.76.01.00.76
altern.	1				72.15.01.02.00.74
7-13	1	72150107	tension spring, loop 180°, 1/1 centred, 13,5x2x113		04.22.01.41.13.00
8	1		spacer bushing 18,4 mm		45.03.10.00.00.73
9	1		plugs for tube ø22x1-2, black		04.63.10.22.00.10
10	1		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
11	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
12	1		cylinder screw DIN 912 M4x30		01.22.07.38.62.40
13	1				02.33.00.40.01.20
14	1	72150104	tipping lever black		72.15.01.04.00.74
15	1	720004	wheel ø50, width 18, nob width 20, axle ø 6, PU grey		04.52.42.50.00.30
16-20	1	72150101	holder anti-tipper for wheel ø50 mm, incl. mounting set		72.15.01.01.00.74
17	1		cylinder screw DIN 912 M4x30		01.22.07.38.62.40
18	1				02.33.00.40.01.20
19	1		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
20	1		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20

Step tube for 20, 22 and 24 inches

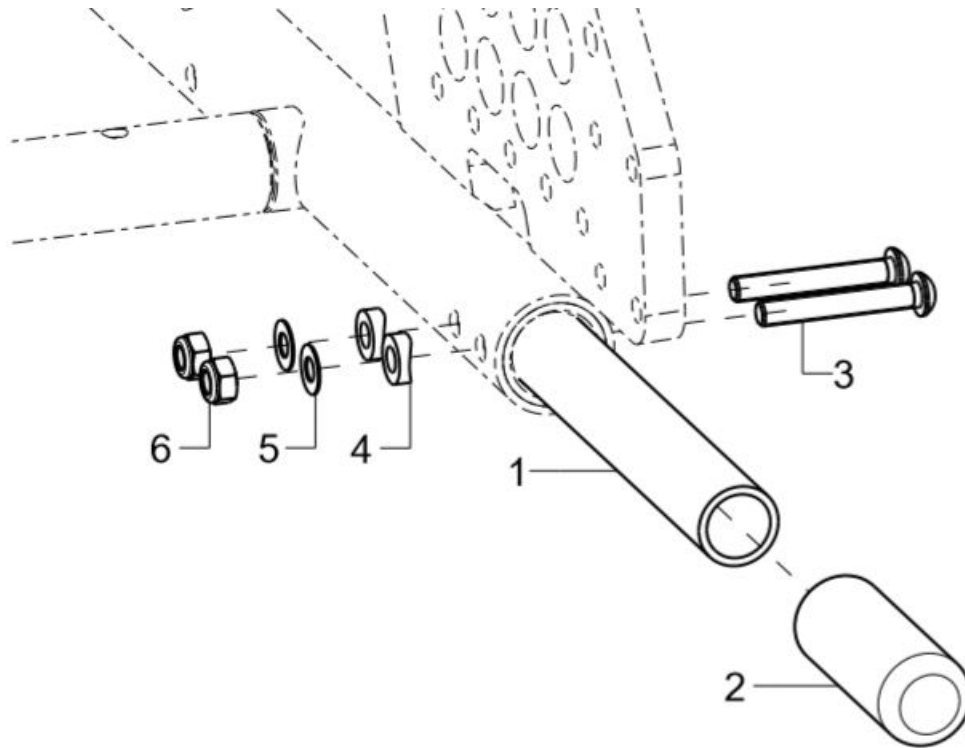


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5	1	7407			
1	1		tipping lever		74.07.01.01.00.76
2	1	740102	covered cap		74.01.02.01.00.10
3	2		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
4	2		saddle washer Ø12/6x2,5		35.04.16.01.00.10
5	2		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
6	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20

Step tube for 12 an 16 inch wheels

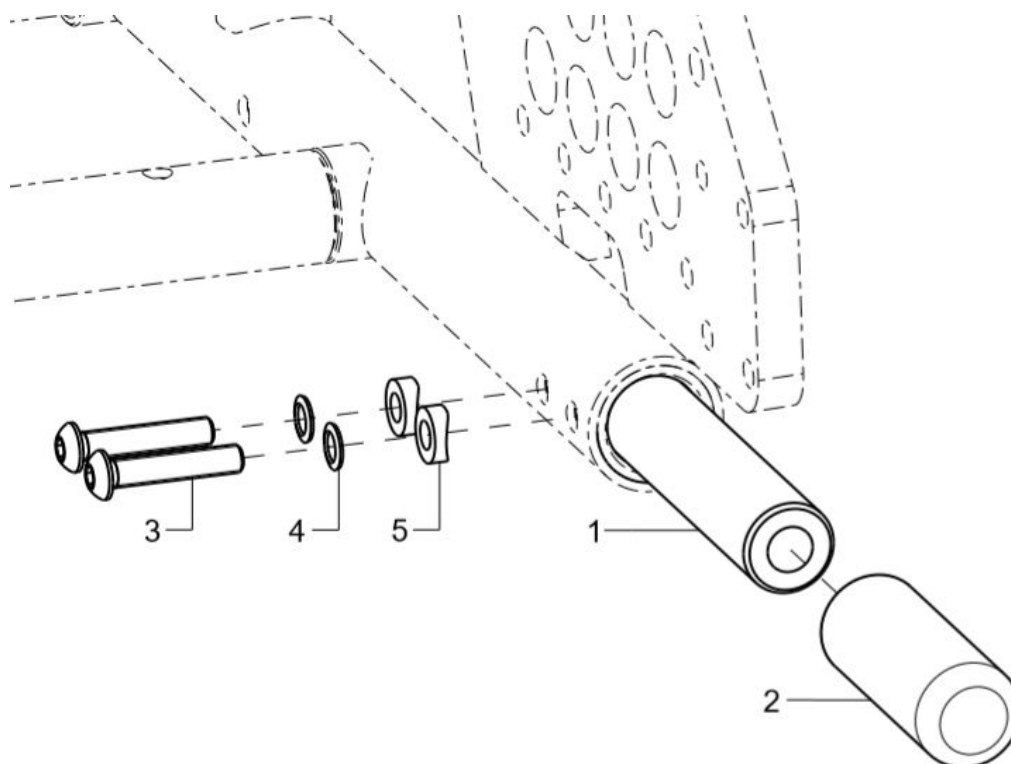

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5	1	740705			
1	1				74.07.01.05.00.76
2	1	740102	covered cap		74.01.02.01.00.10
3	2		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
4	2		lock washer S6		02.99.10.60.01.10
5	2		saddle washer Ø12/6x2,5		35.04.16.01.00.10


Leg support

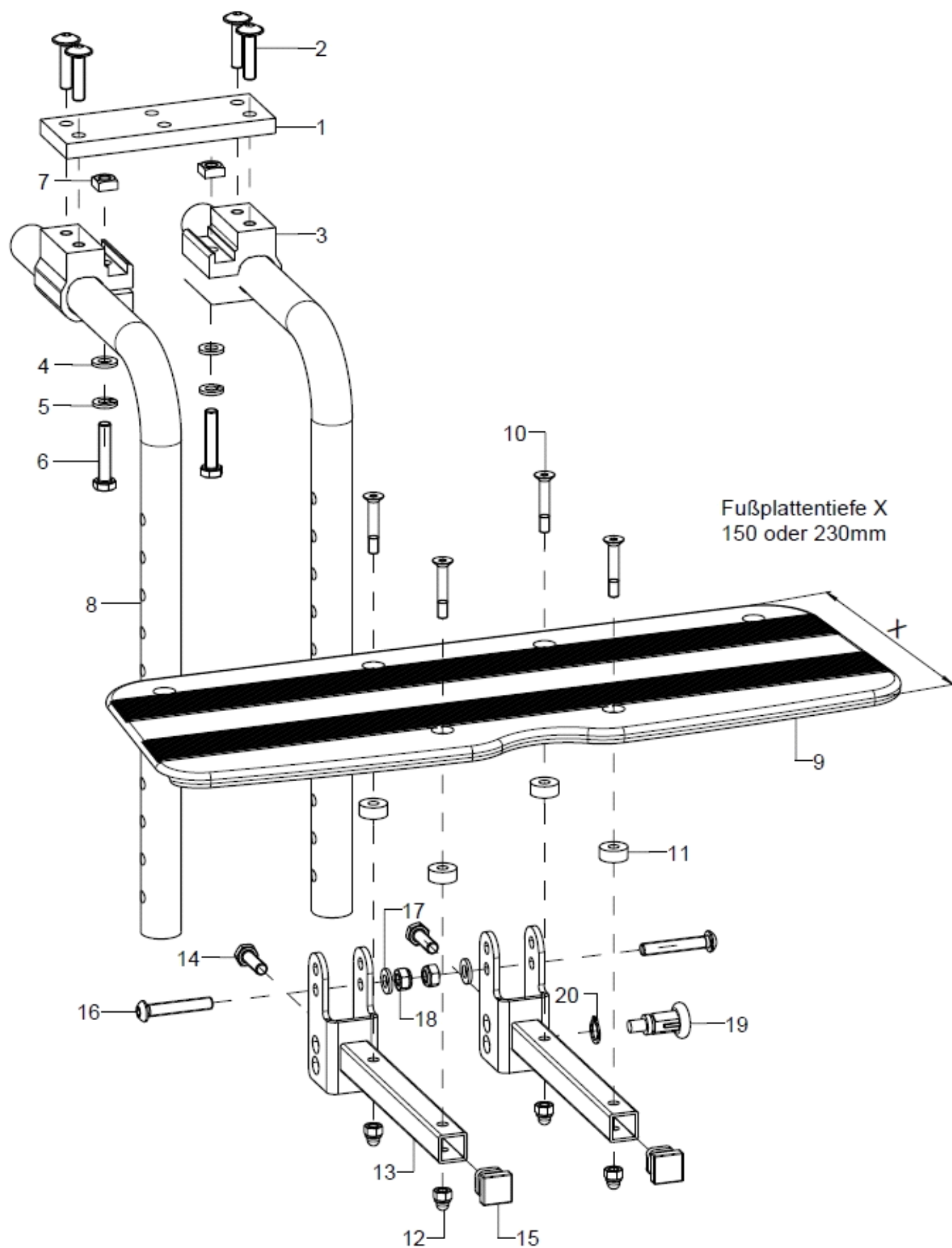



pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-18	1	405501			
o.B.	1	405502			
1	1	40550201			40.55.02.01.00.76
2	4		lens head screw with hexagon socket M6x25		01.43.08.56.65.00
3-7	2	40551003			40.55.10.03.00.76
4	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
5	1		spring ring DIN 127- A6		02.85.30.60.05.10
6	1		hexagon screw DIN 933 - M6x30		01.04.08.58.02.40
7	1		square nut DIN 557 M6		02.13.00.60.01.20
8	2	400102	knee angle leg support holder 90° 310mm		40.01.02.01.00.24
altern.	2	400111	knee angle leg support holder 90° 410 mm		40.01.02.01.00.28
altern.	2	400112			40.01.02.02.00.28
9-12	1	40551100	foot rest 150 mm continuous incl mounting set		40.55.11.00.00.76
altern.	1	40551200	foot rest 230 mm continuous incl. mounting set		40.55.12.00.00.76
10-12	1	40551199	mounting set foot rest		
10	1		counter sunk screw with hexagon socket DIN 7991 M5x30		01.65.07.98.05.00
11	4		bushing Ø14/5,2x6		40.55.00.01.00.76
12	4		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
13-18	2	400103	foot plate joint holder 100720 for continuous foot plate incl. mounting set		40.01.03.02.99.28
14	1				01.04.08.54.82.40
15	1		plugs for square tubes, black 15x15		04.63.11.15.00.10
16	1		lens head screw with hexagon socket ISO 7380 - M6x35		01.42.08.59.65.00
17	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
18	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
19	1	690101	latch ø6/M10x1x31 with lock		04.14.43.60.02.00
20	1		lock ring 10, DIN 471		03.01.01.00.93.00


Leg support

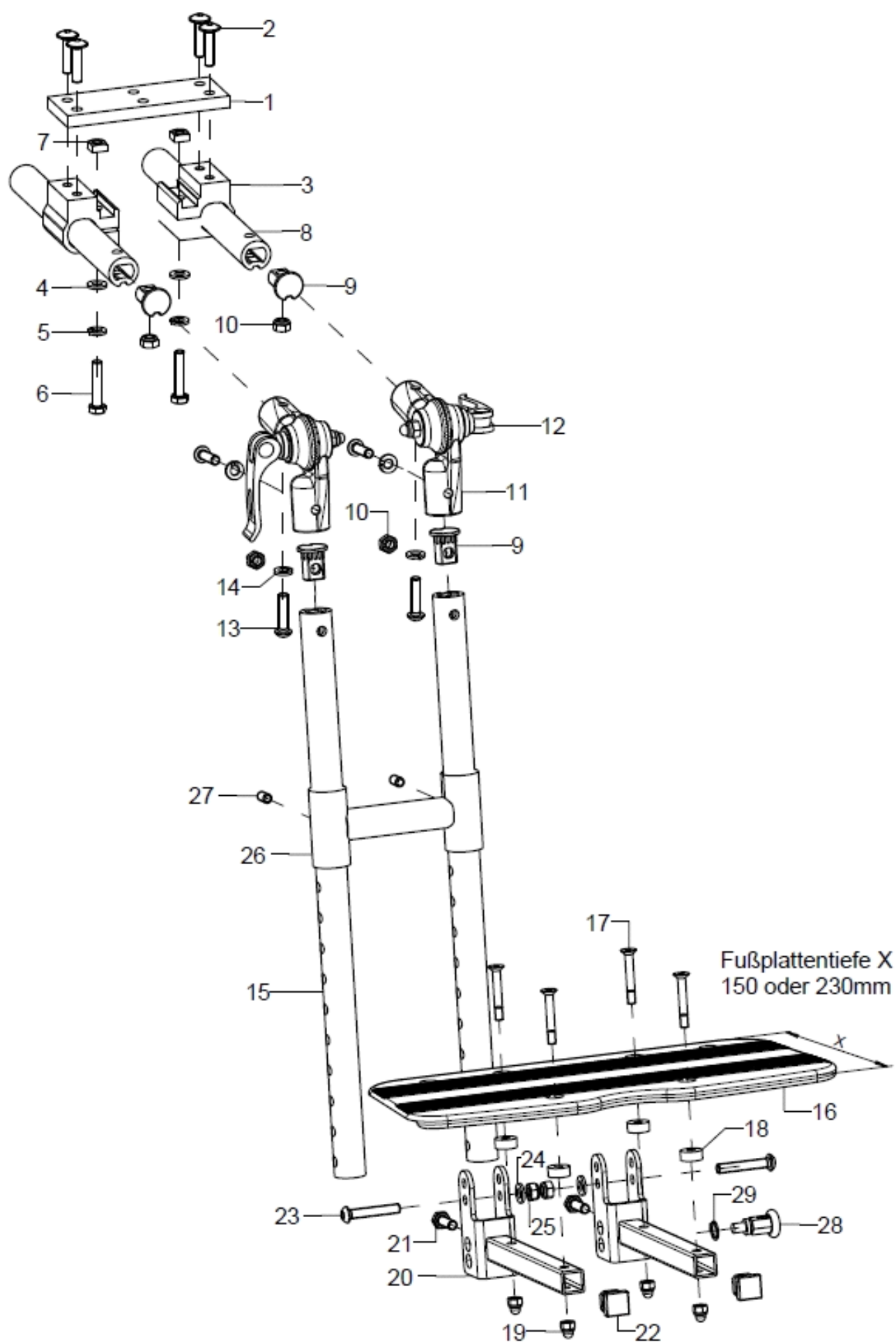



pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-27	1	405503			
o.B.	1	405504			
1a-2	1	40550201			40.55.02.01.00.76
2	3		lens head screw with hexagon socket M6x25		01.43.08.56.65.00
3-7	2	40551003			40.55.10.03.00.76
4	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
5	1		spring ring DIN 127- A6		02.85.30.60.05.10
6	1		hexagon screw DIN 933 - M6x30		01.04.08.58.02.40
7	1		square nut DIN 557 M6		02.13.00.60.01.20
8-10	2	40550003			40.55.00.03.00.76
altern.	2	40550004			40.55.00.04.00.76
9+10	4	653406	plug for nut M6 incl hexagon nut		65.22.20.10.00.00
10	4		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
11-14	2	652220	angle adjusting element metal 307° compl with eccentric clamp and mounting set		65.22.20.99.00.10
12	1	652224	eccentric clamp length 35 mm M6x20, black (incl. spring, nut, washer, bush)		65.22.24.99.00.76
13	2		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
14	2		spring ring DIN 127- A6		02.85.30.60.05.10
15	2	40550301			40.55.03.05.01.76
altern.	2	40550302			40.55.03.05.02.76
altern.	2	40550303			40.55.03.05.03.76
16-19	1	40551100	foot rest 150 mm continuous incl mounting set		40.55.11.00.00.76
altern.	1	40551200	foot rest 230 mm continuous incl. mounting set		40.55.12.00.00.76
17-19	1	40551199	mounting set foot rest		
17	4		counter sunk screw with hexagon socket DIN 7991 M5x30		01.65.07.98.05.00
18	4		bushing Ø14/5,2x6		40.55.00.01.00.76
19	4		hexagon cap nut low form with clamp DIN 986 - M5		02.33.00.60.21.20
20-25	2	400103	foot plate joint holder 100720 for continuous foot plate incl. mounting set		40.01.03.02.99.28
21	1				01.04.08.54.82.40
22	1		plugs for square tubes, black 15x15		04.63.11.15.00.10
23	1		lens head screw with hexagon socket ISO 7380 - M6x35		01.42.08.59.65.00
24	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
25	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
26	1	40521001			40.52.10.01.00.24
27	2		setscrew with hexagon socket DIN 913, M6x12		01.67.08.49.61.20
28+29	1	690101	latch ø6/M10x1x31 with lock		04.14.43.60.02.00
29	1		lock ring 10, DIN 471		03.01.01.00.93.00


Leg support

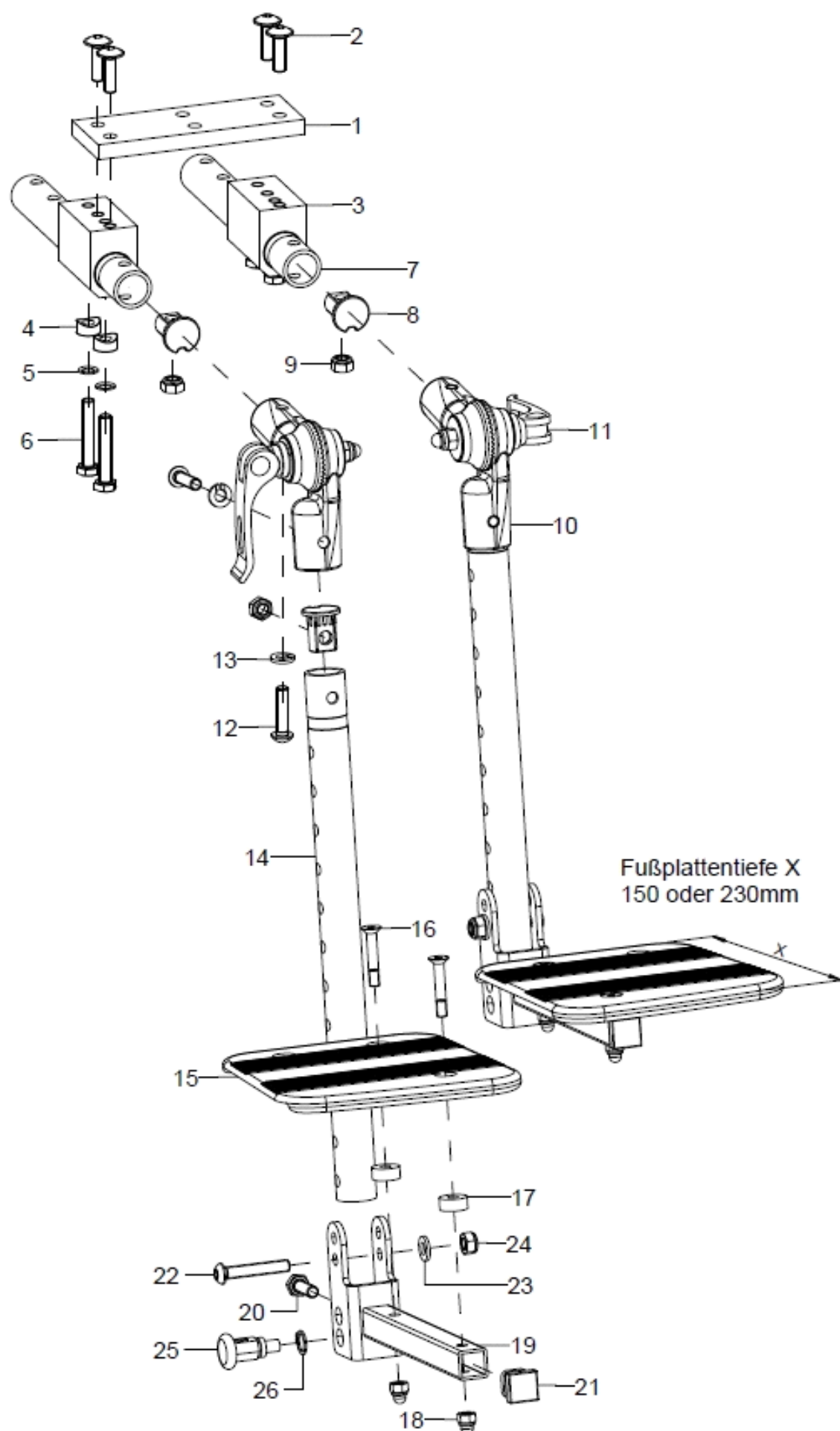



pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-24	1	405530			
o.B.	1	405531			
1a-2	1	40550201			40.55.02.01.00.76
2	4		lens head screw with hexagon socket M6x25		01.43.08.56.65.00
3-6	2	40550308			40.55.03.08.00.76
4	2		saddle washer Ø12x3x7		40.13.01.03.00.24
5	2		lock washer S6		02.99.10.60.01.10
6	2		hexagon screw DIN 933 - M6x35		01.04.08.59.02.40
7-9+12+13	2	40550307			40.55.03.07.00.76
8+9	4	653406	plug for nut M6 incl hexagon nut		65.22.20.10.00.00
9	4		hexagon nut DIN 934 - M6		02.19.00.60.01.20
10-13	2	652220	angle adjusting element metal 307° compl with eccentric clamp and mounting set		65.22.20.99.00.10
11	1	652224	eccentric clamp length 35 mm M6x20, black (incl. spring, nut, washer, bush)		65.22.24.99.00.76
12	2		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
13	2		spring ring DIN 127- A6		02.85.30.60.05.10
14	2	40550304			40.55.03.06.01.76
altern.	2	40550305			40.55.03.06.02.76
altern.	2	40550306			40.55.03.06.03.76
15-18	1	40551300	foot rest 150 mm divided, incl. mounting set		40.55.13.00.00.76
altern.	1	40551400	foot rest 230 mm divided, incl. mounting set		40.55.14.00.00.76
16-18	1	40551199	mounting set foot rest		
16	4		counter sunk screw with hexagon socket DIN 7991 M5x30		01.65.07.98.05.00
17	4		bushing Ø14/5,2x6		40.55.00.01.00.76
18	4		hexagon cap nut low form with clamp DIN 986 - M5		02.33.00.60.21.20
19-24	2	400103	foot plate joint holder 100720 for continuous foot plate incl. mounting set		40.01.03.02.99.28
20	1				01.04.08.54.82.40
21	1	401107	end plug 15x15 black, 4 piece set		04.63.11.15.00.10
22	1		lens head screw with hexagon socket ISO 7380 - M6x35		01.42.08.59.65.00
23	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
24	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
25+26	1	690101	latch Ø6/M10x1x31 with lock		04.14.43.60.02.00
26	1		lock ring 10, DIN 471		03.01.01.00.93.00

Adjustable foot plate

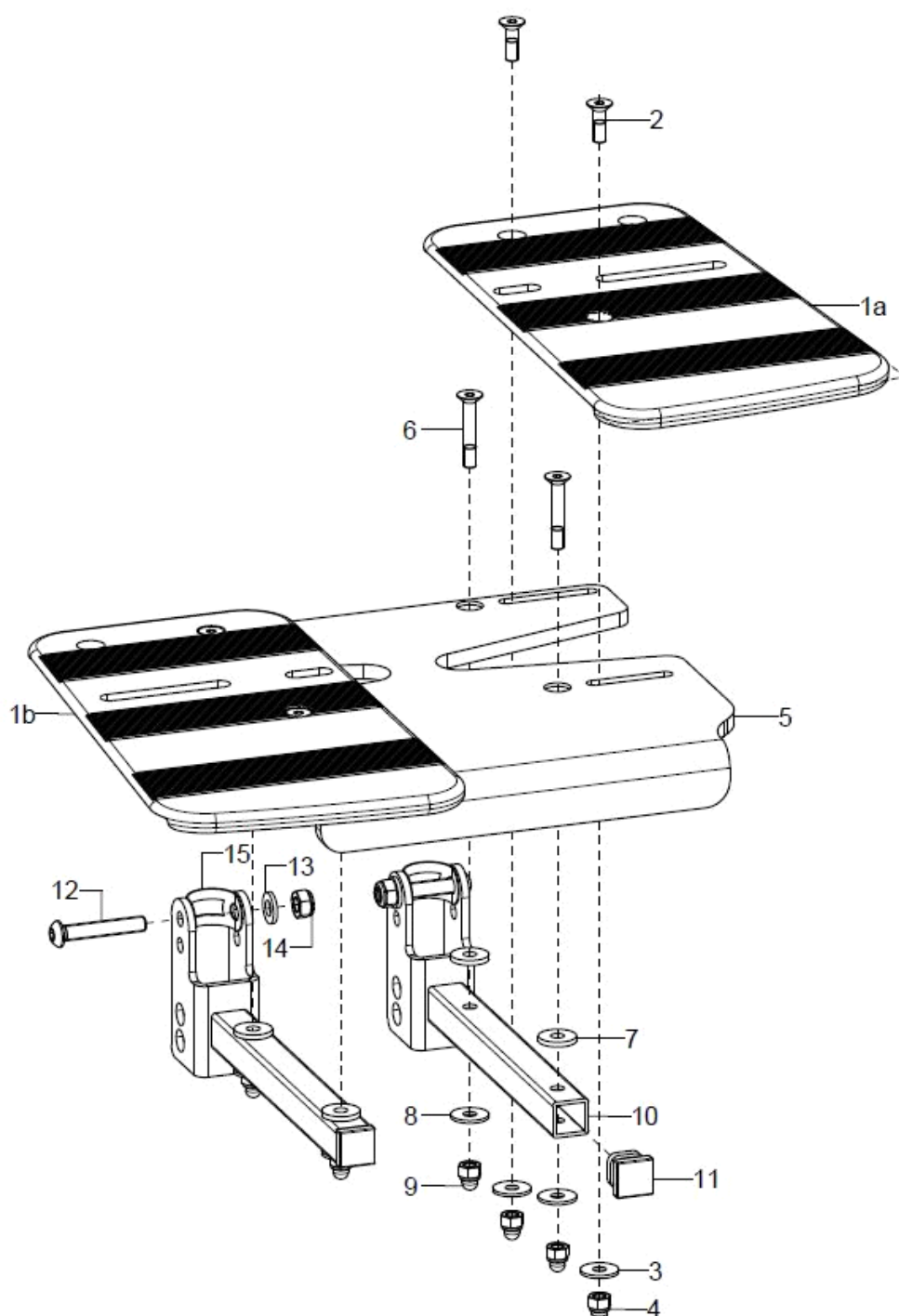


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-4	1	40551400	foot rest 230 mm divided, incl. mounting set		40.55.14.00.00.76
2	2		counter sunk screw with hexagon socket DIN 7991 M5x16		01.65.07.92.05.00
3	2		washer DIN 9021 - M5		02.79.05.31.20.20
4	2		hexagon cap nut low form with clamp DIN 986 - M5		02.33.00.60.21.20
5-9	1	40551405			40.55.16.00.00.76
6	4		counter sunk screw with hexagon socket DIN 7991 M5x30		01.65.07.98.05.00
7	4		washer DIN 7349 - 5,3		02.77.05.22.00.20
8	4		hexagon cap nut low form with clamp DIN 986 - M5		02.33.00.60.21.20
9	4		washer DIN 9021 - M5		02.79.05.31.20.20
10-15	2	400103	foot plate joint holder 100720 for continuous foot plate incl. mounting set		40.01.03.02.99.28



11	1	401107	end plug 15x15 black, 4 piece set	04.63.11.15.00.10
12	1		lens head screw with hexagon socket ISO 7380 - M6x35	01.42.08.59.65.00
13	1		washer DIN 125 - A 6,4	02.61.06.41.41.20
14	1		hexagon nut self-locking low form DIN 985 - M6	02.32.00.60.01.20
15	1			40.54.03.04.00.82

Leg support

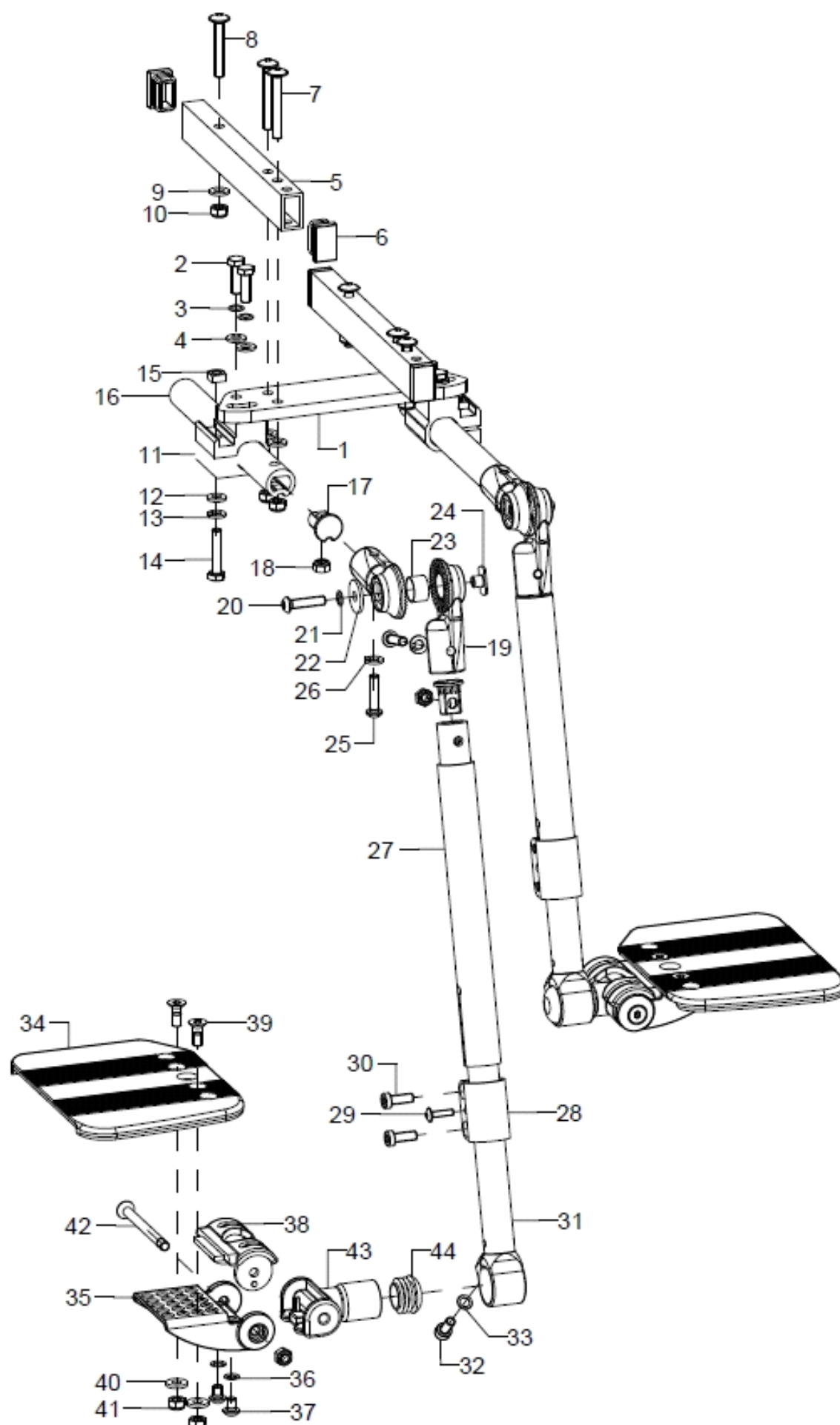




pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-44	1	405506			
1-15	1	40550601			
1-4	1	40550602			40.55.06.02.00.76
2	4		hexagon screw DIN 933 - M6x20		01.04.08.54.02.40
3	4		lock washer S6		02.99.10.60.01.10
4	4		washer DIN 125 - A 6,4		02.61.06.41.41.20
5-10	2	40550610			40.50.06.10.01.76
altern.	2	40550613			40.50.06.10.02.76
6	4		plugs for rectangle tubes, black, 25x15x1,5-2		04.63.12.25.15.10
7	4		lens head screw with hexagon socket M6x45		01.43.08.61.05.00
8	2				01.43.08.60.65.00
9	6		washer DIN 125 - A 6,4		02.61.06.41.41.20
10	6		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
11-15	2	40551003			40.55.10.03.00.76
12	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
13	1		spring ring DIN 127- A6		02.85.30.60.05.10
14	1		hexagon screw DIN 933 - M6x30		01.04.08.58.02.40
15	1		square nut DIN 557 M6		02.13.00.60.01.20
16-18	2	40550300			40.55.00.03.00.76
17-18	4	653406	plug for nut M6 incl hexagon nut		65.22.20.10.00.00
18	4		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
19-26	2	652221			65.22.20.02.00.47
20	1		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
21	1		lock washer S6		02.99.10.60.01.10
22	1				02.67.22.03.09.21
23	1		bushing Ø16,8x1x10		65.22.20.05.00.47
24	1		weld nut C- M6x7,5		02.49.34.03.00.10
25	2		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
26	2		spring ring DIN 127- A6		02.85.30.60.05.10
27	2	40550605			40.55.06.05.00.76
28-30	2		clamp BMD		40.55.06.08.00.76
29	2		lens head screw with hexagon socket M4x16		01.43.07.31.65.00
30	4		cylinder screw, low head DIN 7984-M6x16		01.24.08.52.05.00
31-33	2	40550611	leg support tube glued 195 mm left, incl. mounting screw		40.55.06.11.01.76
altern.	2	40360121	leg support tube glued 235 mm left, incl. mounting screws		40.36.01.15.21.74
altern.	2	40360131	leg support tube glued 305 mm left, incl. mounting screws		40.36.01.15.31.74
32	1		cylinder screw DIN 912 M6x16		01.22.08.52.62.40
33	1		lock washer S6		02.99.10.60.01.10
34a	1	40361100	foot plate 8mm divided left		40.36.10.XX.02.74
34b	1	40361000	foot plate 8mm divided right		40.36.10.XX.02.74
35-38	2	40361020	adjustment unit divided incl. grid		40.36.10.30.00.74
36	2		lock washer S6		02.99.10.60.01.10
37	2		lens head screw with hexagon socket ISO 7380 - M6x8		01.42.08.47.65.00
38	1	40360102	raster part for foot plate		40.36.01.02.00.74
39-42	2	40360599	mounting set adjustment piece		XXXXXXXXXXXX
39	2		counter sunk screw with hexogon socket DIN 7991 M6x16		01.65.08.52.02.40
40	2		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
41	3		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
42	1		counter sunk screw with hexagon socket DIN 7991 M6x75/8		01.65.08.67.02.41



43-44	2	40360205	joint for divided leg support incl. plug	40.36.10.31.00.74
44	2		plugs for tube ø25x1-2, black	04.63.10.25.00.10


Leg support

Possible from SD40. Not possible in combination with frame arches for closed frames.

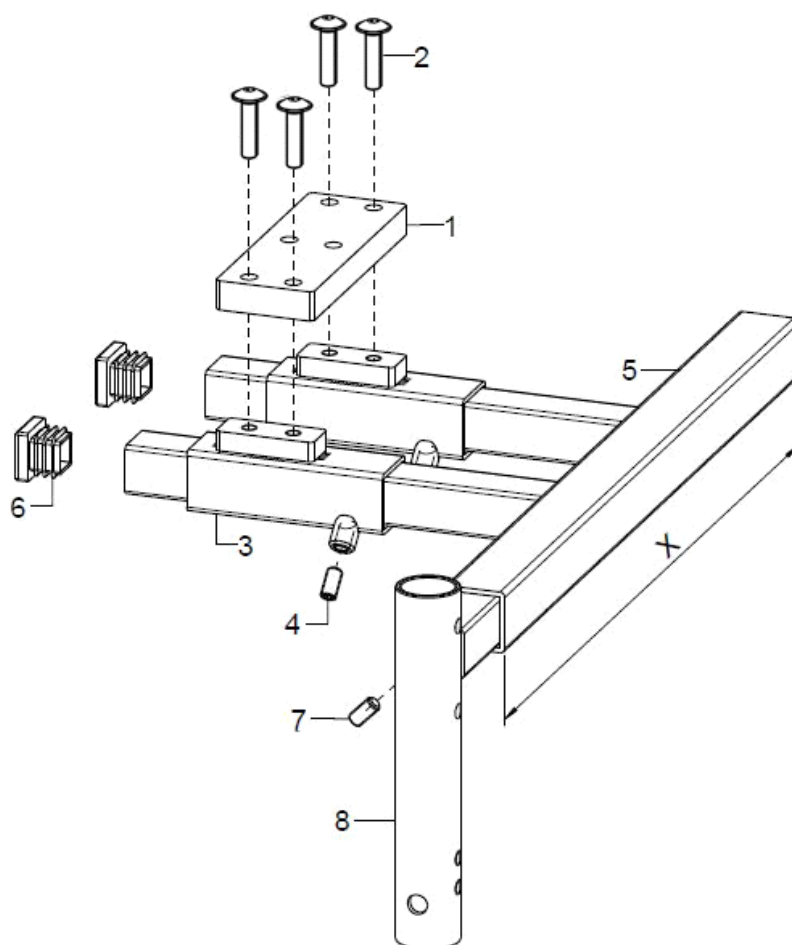

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-10	1	40550401			
1-2	1	40550405			40.55.04.04.02.76
2	4		lens head screw with hexagon socket M6x25		01.43.08.56.65.00
3-4	2	40550404			40.55.04.04.00.76
4	1		setscrew with hexagon socket DIN 913, M6x12		01.67.08.49.61.20
5-7	1	40550411			40.55.04.01.00.76
5-7	1	40550412			40.55.04.01.00.76
6	2		plugs 20x20		04.63.11.20.00.10
7	2		setscrew with hexagon socket DIN 913, M6x12		01.67.08.49.61.20
8	2	40550403			40.55.04.03.00.76
o.B.	1				40.55.01.01.02.80
o.B.	1				40.55.01.01.01.80


Leg support

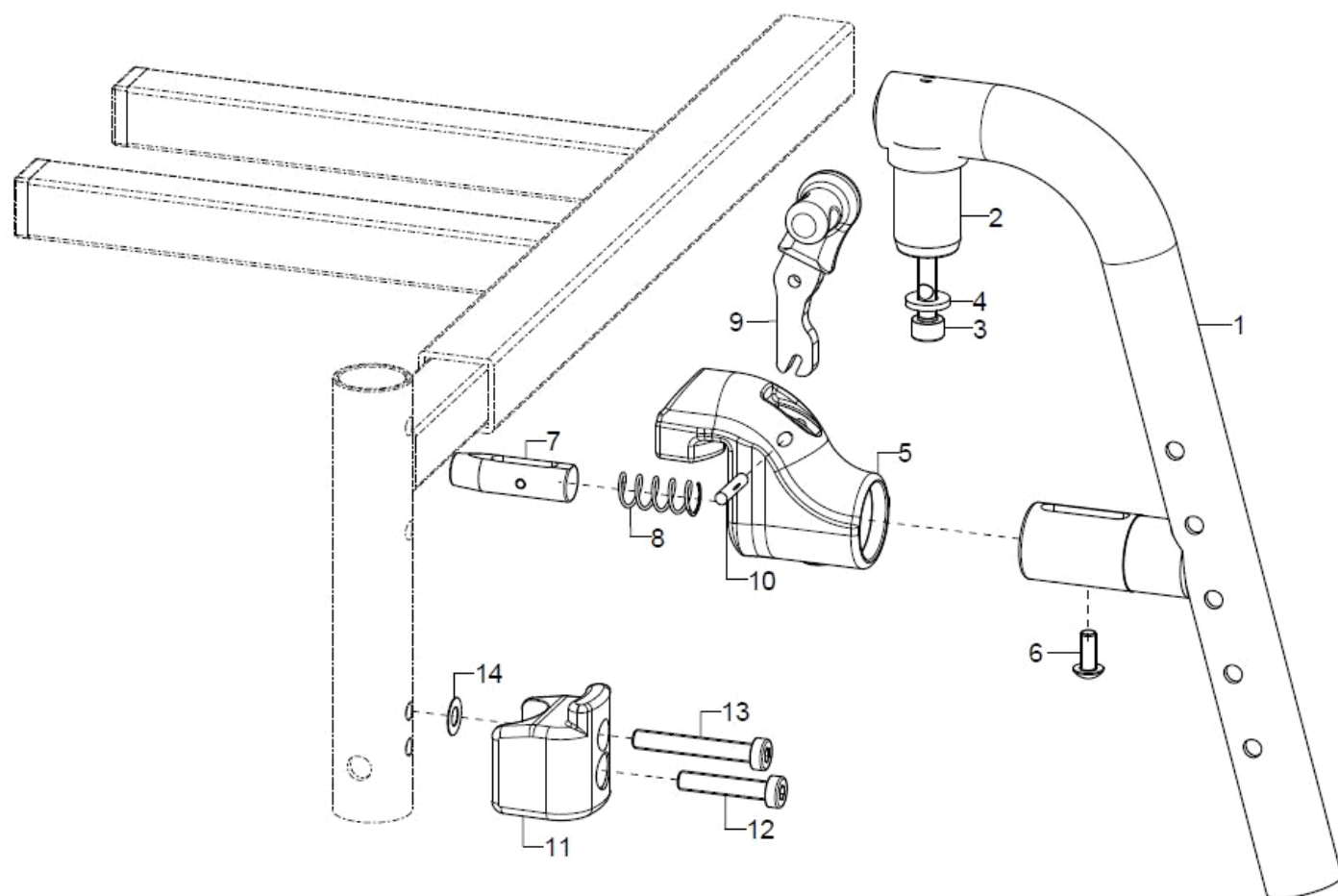

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1a-4	1	40370802			40.37.08.00.02.75
o.B.	1	40370801			40.37.08.00.01.75
2-4	1	40370103			40.37.01.03.00.75
3	1				01.22.08.58.62.40
4	1		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
5-10	1	40380808			
o.B.	1	40380807			
5	2				40.37.08.02.00.75
6	1				01.43.07.89.65.01
7	1				40.37.08.04.00.75
8	1				04.21.01.10.26.00
9	1				40.37.08.05.90.75
10	1				03.36.04.20.00.01
11-14	2	40370810			40.37.08.03.00.75
12	1				01.24.08.58.05.01
13	1				01.24.08.60.05.01
14	1				02.68.06.00.30.30

Leg support


Possible from SD40. Not possible in combination with frame arches for closed frames.

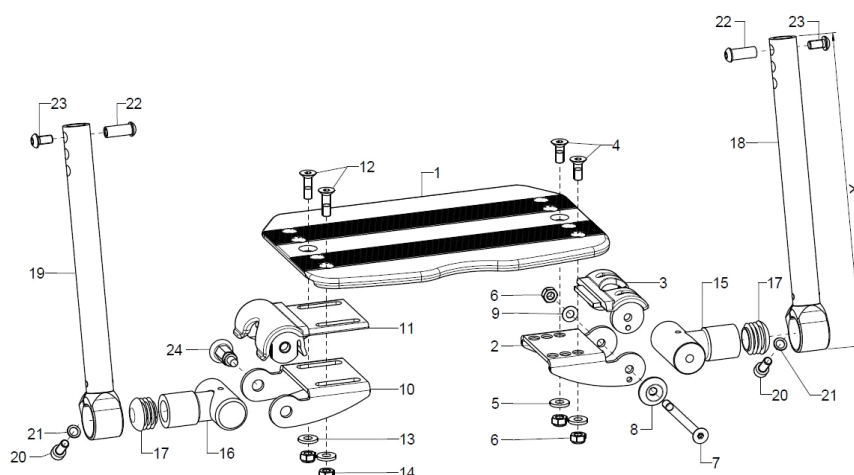

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-23	1	403610	leg support continuous, folds up sideways, outer attachment complete		XXXXXXXXXXXX
1	1	40280100	continuous foot plate 8 mm		40.28.01.01.00.10
2-3	1	40360105	adjustment part left incl. raster piece		40.36.01.05.00.74
3	1	40360102	raster part for foot plate		40.36.01.02.00.74
4-9	1	40360299	mounting set adjustment piece		XXXXXXXXXXXX
4	2		counter sunk screw with hexagon socket DIN 7991 M6x16		01.65.08.52.02.40
5	2		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
6	3		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
7	1		counter sunk screw with hexagon socket DIN 7991 M6x75/8		01.65.08.67.02.41
8	1		rosette ring Ø22/6x4,5, black		04.14.70.60.09.00
9	1		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
10-11	1	40360106	adjustment part right incl. raster piece		40.36.01.06.00.74
11	1	40360101	locking piece		40.36.01.01.00.74
12-14	1	40360599	mounting set adjustment piece		XXXXXXXXXXXX
12	2		counter sunk screw with hexagon socket DIN 7991 M6x20		01.65.08.54.02.40
13	2		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
14	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
15+17	1	40360111	joint D6 incl. plug		40.36.01.11.50.74
16-17	1	40360110	joint incl. plug		40.36.01.10.50.74
o.B.	1	40360112	joint for latch incl. plug		40.36.01.12.50.74
17	2		plugs for tube Ø25x1-2, black		04.63.10.25.00.10
18+20+21	1	40360121	leg support tube glued 235 mm left, incl. mounting screws		40.36.01.15.21.74
o.B.	1	40360131	leg support tube glued 305 mm left, incl. mounting screws		40.36.01.15.31.74
19-21	1	40360122	leg support tube glued 235 mm right, incl. mounting screws		40.36.01.15.22.74
o.B.	1	40360132	leg support tube glued 305 mm right, incl mounting screws		40.36.01.15.32.74
20	1		cylinder screw DIN 912 M6x16		01.22.08.52.62.40
21	1		lock washer S6		02.99.10.60.01.10
22-23	2	40390299	mounting set leg support tube		40.39.02.99.90.00
22					40.39.01.05.50.69
23	1		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00
24	1	690101	latch Ø6/M10x1x31 with lock		04.14.43.60.02.00
10+16+24	1	400902	lock complete		

Leg support


Possible from SD40. Not possible in combination with frame arches for closed frames.

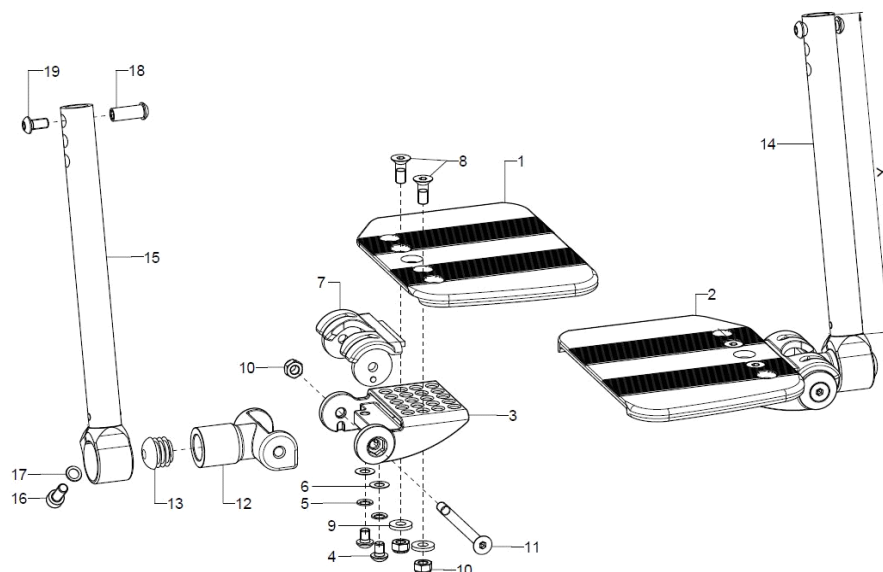

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-19	1	403612	leg support divided, folds up sideways, outer attachment, complete		XXXXXXXXXXXX
1	1	40361000	foot plate 8mm divided right		40.36.10.XX.02.74
2	1	40361100	foot plate 8mm divided left		40.36.10.XX.02.74
3-7	2	40361020	adjustment unit divided incl. grid		40.36.10.30.00.74
4	2		lens head screw with hexagon socket ISO 7380 - M6x8		01.42.08.47.65.00
5	2		lock washer S6		02.99.10.60.01.10
6	2		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
7	1	40360102	raster part for foot plate		40.36.01.02.00.74
8-11	1	40360299	mounting set adjustment piece		XXXXXXXXXXXX
8	2		counter sunk screw with hexagon socket DIN 7991 M6x16		01.65.08.52.02.40
9	2		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
10	3		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
11	1		counter sunk screw with hexagon socket DIN 7991 M6x75/8		01.65.08.67.02.41
12-13	2	40360205	joint for divided leg support incl. plug		40.36.10.31.00.74
13	1		plugs for tube Ø25x1-2, black		04.63.10.25.00.10
14+16+17	1	40360121	leg support tube glued 235 mm left, incl. mounting screws		40.36.01.15.21.74
o.B.	1	40360131	leg support tube glued 305 mm left, incl. mounting screws		40.36.01.15.31.74
15-17	1	40360122	leg support tube glued 235 mm right, incl. mounting screws		40.36.01.15.22.74
o.B.	1	40360132	leg support tube glued 305 mm right, incl. mounting screws		40.36.01.15.32.74
16	1		cylinder screw DIN 912 M6x16		01.22.08.52.62.40
17	1		lock washer S6		02.99.10.60.01.10
18-19	2	40390299	mounting set leg support tube		40.39.02.99.90.00
18	1				40.39.01.05.50.69
19	1		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00

Leg support

Possible from SD40. Not possible in combination with frame arches for closed frames.

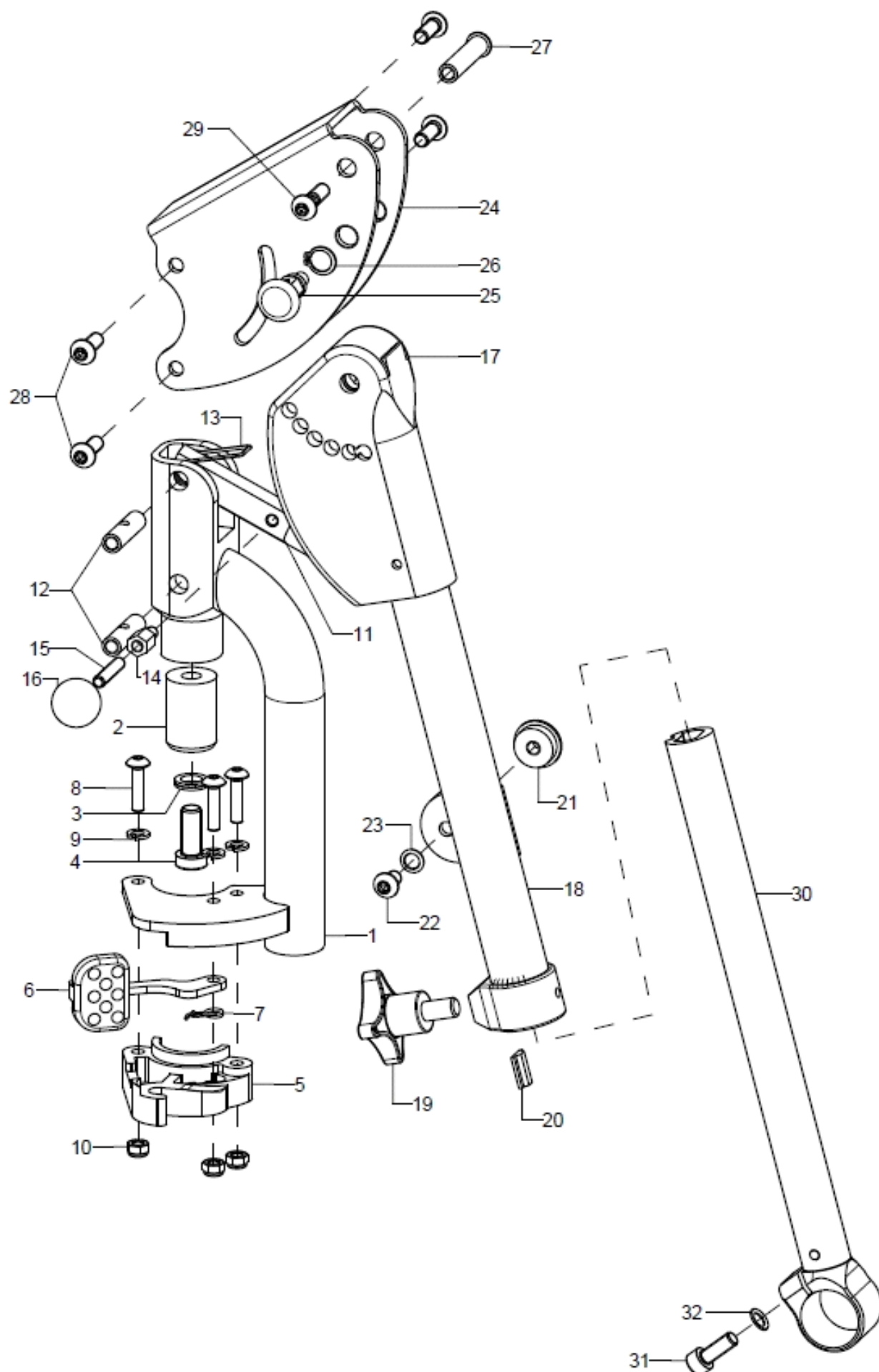




pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-30	1	40370502			
o.B.	1	40370501			
1	1	40371002			40.37.06.10.02.75
o.B.	1	40371001			40.37.06.10.01.75
2-4	1	40370617			40.37.06.17.00.75
3	1				02.99.20.70.01.10
4	1				01.24.09.74.05.00
5-10	1	40370122			40.37.01.04.02.75
o.B.	1	40370121			40.37.01.04.01.75
6	1				40.37.01.08.00.75
7	1				40.37.01.09.02.75
o.B.	1				40.37.01.09.01.75
8	3		lens head screw with hexagon socket ISO 7380 - M5x20		01.42.07.94.65.00
9	3		spring ring DIN 127 - A 5		02.85.30.50.05.10
10	3		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
11-16	1	40370623			40.37.06.23.00.75
12	2		thread bushing M6x29,5		40.37.06.24.00.75
13	1		leg spring double arm		40.37.06.26.00.75
14	1		distance hexagon thread 6/8 M5x10		02.47.20.78.72.10
15	1		setscrew with hexagon socket DIN 913, M6x8		01.67.08.47.61.20
16	1		ball knob DIN 319 Ø20/M5, form C, black		04.06.12.00.50.40
17-20	1	40370627			
17	1	40370622			40.37.06.22.00.75
18-20	1	4037060502			40.37.06.01.02.75
o.B.	1	4037060501			40.37.06.01.01.75
altern.	1				40.37.06.05.02.75
o.B.	1				40.37.06.05.01.75
19-20	1	40370521			
19	1		cross handle M8x14		04.05.24.73.14.40
20	1		clamp piece		40.37.05.21.00.75
21-23	1				40.30.10.04.00.75
22	1		lens head screw with hexagon socket ISO 7380 - M6x10		01.42.08.48.65.00
23	1		lock washer S6		02.99.10.60.01.10
24-29	1	40370621			40.37.06.21.00.75
25	1	690101	latch ø6/M10x1x31 with lock		04.14.43.60.02.00
26	1		lock ring 10, DIN 471		03.01.01.00.93.00
27	1		bushing M6x36		40.37.06.25.99.75
28	6		lens head screw with hexagon socket ISO 7380 - M6x16		01.42.08.52.65.00
29	1		lens head screw with hexagon socket ISO 7380 - M6x22 Tuflock spot		01.66.08.55.92.60
30	1	40370526	leg support tube glued, long(BSH)		40.37.05.25.23.75
altern.	1	40370525			40.37.05.25.30.75
31	1		cylinder screw DIN 912 M6x16		01.22.08.52.62.40
32	1		lock washer S6		02.99.10.60.01.10


Die geeigneten Fußauflagen inklusive Halterungen finden Sie unter dem Kapitel „Beinstütze geteilt, nach außen abschenkbare und abnehmbar“

Leg support

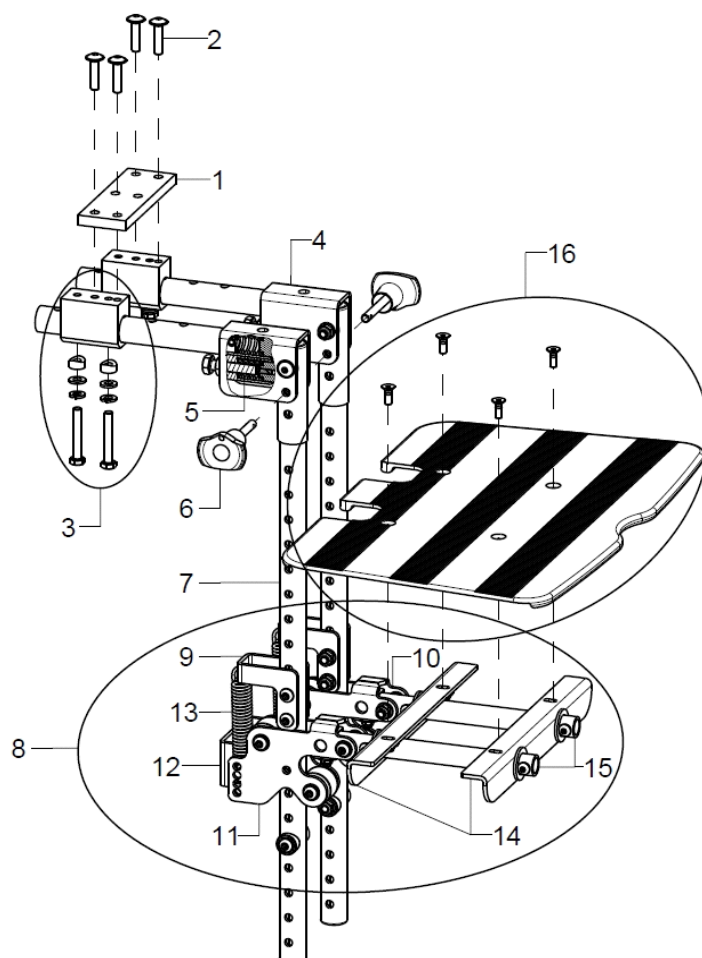


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-16	1	4053	leg support dynamic Genex - sprung on the flexible knee angle		
1	1	40550201			40.55.02.01.00.76
2-3	2	40550308			40.55.03.08.00.76
2	4		lens head screw with hexagon socket M6x25		01.43.08.56.65.00
4-5	2	405303	leg support tube black, with spring intake and springs in the knee angle for Genex leg support		40.53.02.02.00.24
5	4	405312	pressure spring 2,5x12,5x22 in the knee angle for dynamic leg support		04.21.01.50.22.00
6	2	405310	ball lock pin 6x30 with red button		04.14.10.60.33.10
7	2	405302	leg support tube 500mm, with stainless steel bushing for Genex leg support		40.53.02.01.00.24
8	1	405201	holder for foot plate complete for dynamic leg support		
9	2	405205	spring holder top for dynamic leg support		40.52.05.00.00.24
10	1	40520601	intake left, stainless steel for dynamic leg support		40.52.06.00.01.24
11	1	40520602	intake right, stainless steel for dynamic leg support		40.52.06.00.02.24
12	1	40520801	connection black for dynamic leg support incl. mounting set		40.52.08.01.00.24
13	2	405212	tension spring 2,2x15x74,2 for dynamic leg support		04.22.01.50.74.00
14	2	40520902	holder for foot plate dynamic leg support		40.52.09.02.00.24
15	2	405207	foot plate tube, black for dynamic leg support		40.52.07.00.00.24
16	1	405203			40.53.05.00.00.00

Heel edge

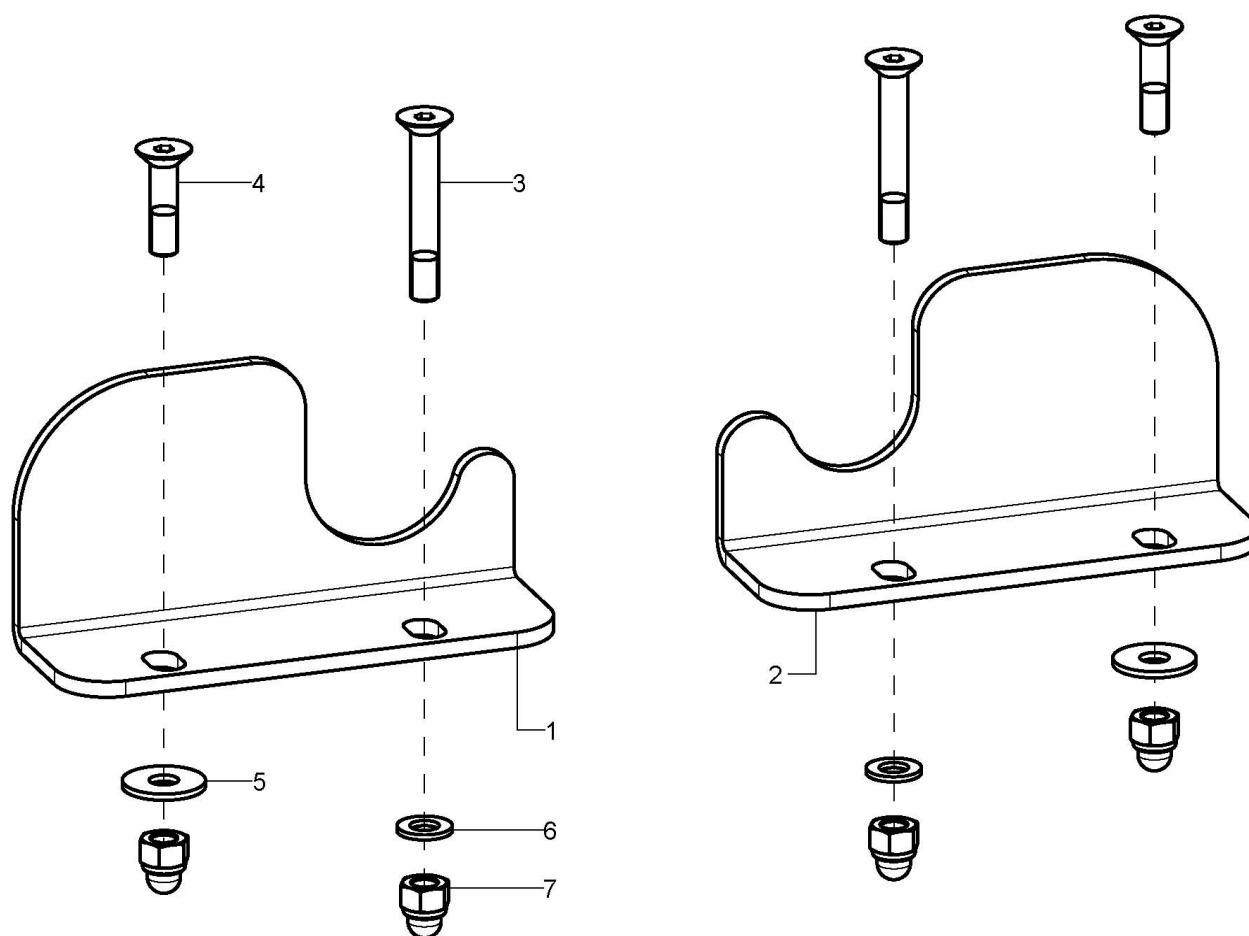


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-7	1P	405509			
1+3-7	1	40551012			40.55.10.01.12.76
2-7	1	40551011			40.55.10.01.11.76
o.B.	1	40551022			40.55.10.01.22.76
o.B.	1	40551021			40.55.10.01.21.76
o.B.	1	40551032			40.55.10.01.32.76
o.B.	1	40551031			40.55.10.01.31.76
o.B.	1	40551042			40.55.10.01.42.76
o.B.	1	40551041			40.55.10.01.41.76
3	1		counter sunk screw with hexagon socket DIN 7991 M5x35		01.65.07.99.05.00
4	1		counter sunk screw with hexogon socket DIN 7991 M5x20		01.65.07.94.05.00
5	1		washer DIN 125 - A 5,3		02.61.05.31.01.20
6	1		washer DIN 9021 - M5		02.79.05.31.20.20
7	2		hexagon cap nut low form with clamp DIN 986 - M5		02.33.00.60.21.20

Heel support

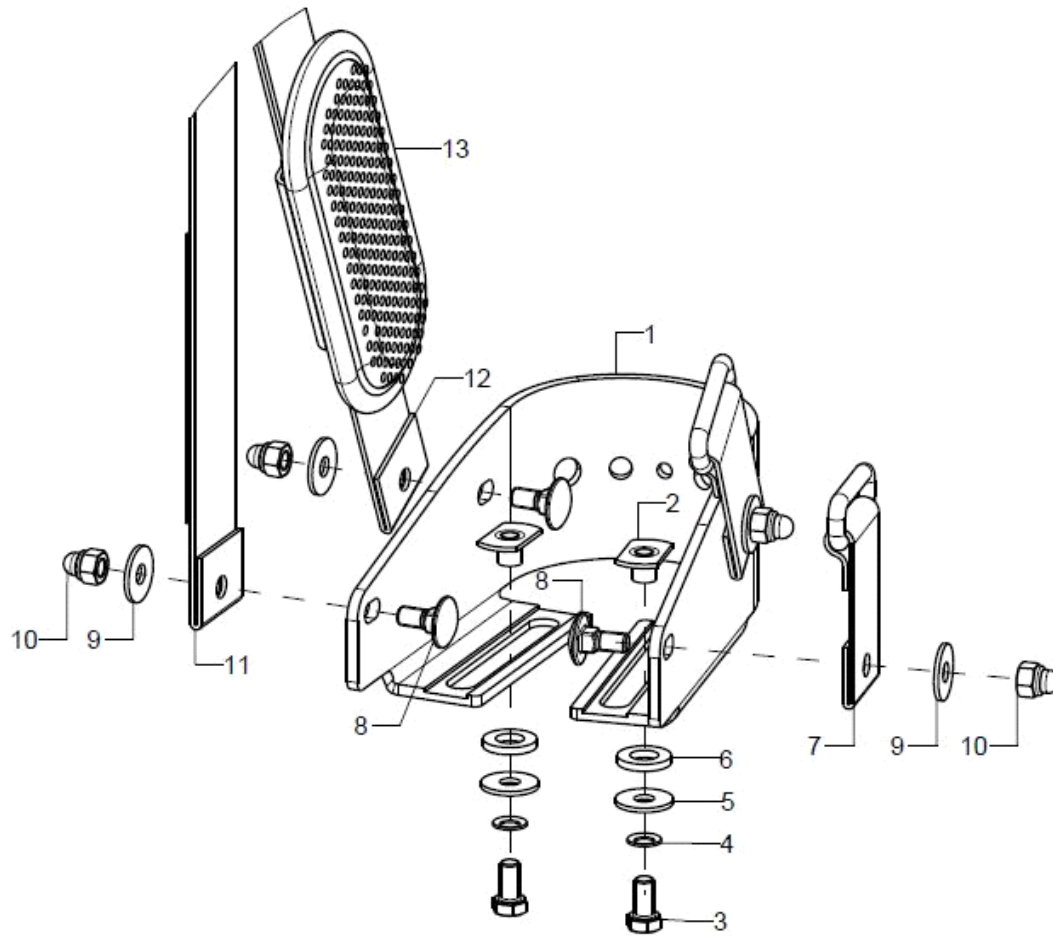


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-6	2	40551501			40.55.15.01.00.76
altern.	2	40551502			40.55.15.02.00.76
altern.	2	40551503			40.55.15.03.00.76
2	1		weld nut C- M5x7,5		02.49.33.03.00.10
3	1				01.04.07.88.02.40
4	1		lock washer S5		02.99.10.50.01.10
5	1		washer DIN 9021 -6,4		02.79.06.41.60.20
6	1				02.83.07.42.00.80
7-13	2	40201501			
altern.	2	40201502			
altern.	2	40201503			

Calf rest

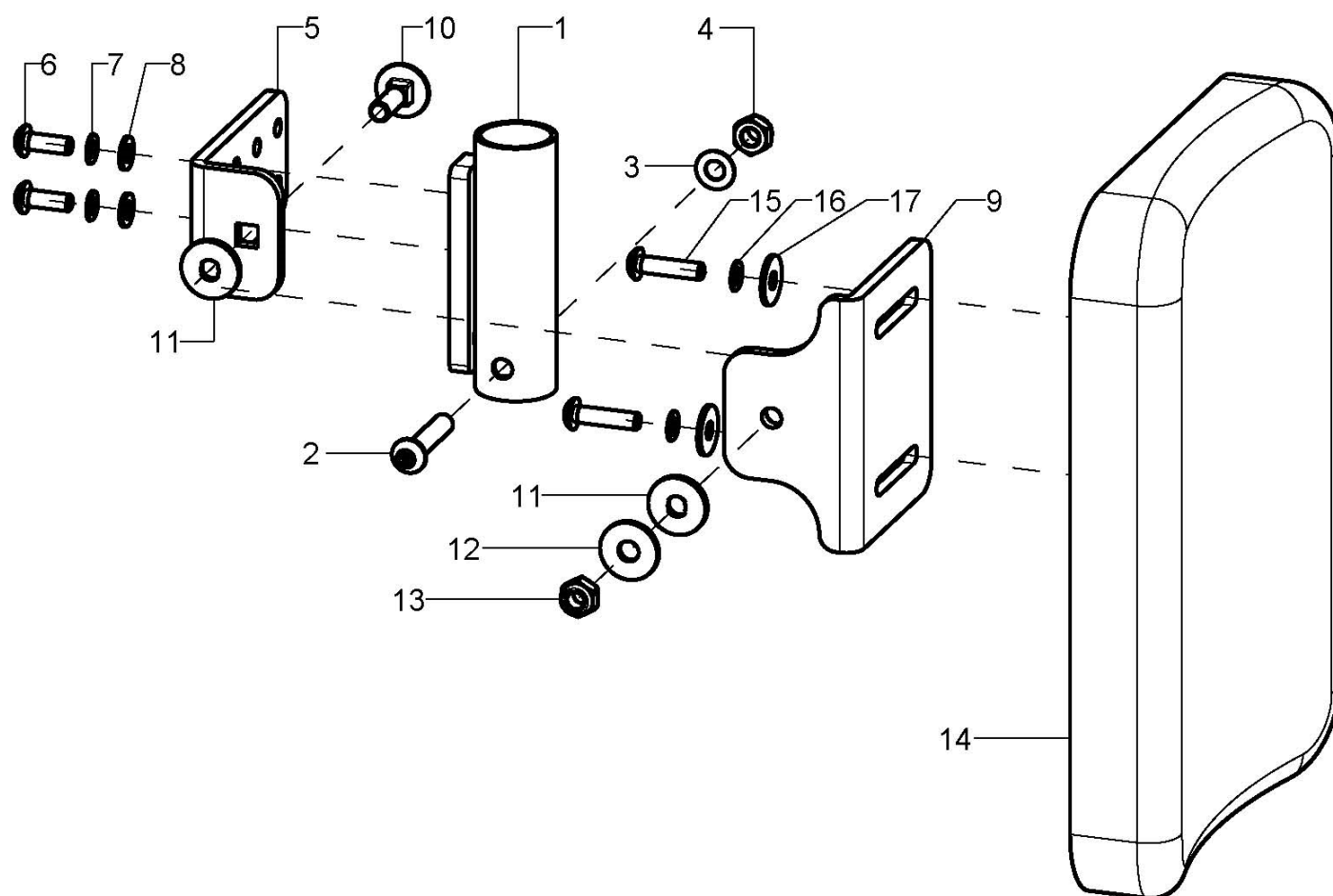


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-17	1	403005			XXXXXXXXXXXX
1-4	1	40300602			40.30.06.02.00.10
2	1		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
3	1		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
4	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
5-8	1	40300603	angle adjustment incl. mounting set		40.30.06.03.00.10
6	2		lens head screw with hexagon socket ISO 7380 - M5x14		01.42.07.90.65.00
7	2		lock washer S5		02.99.10.50.01.10
8	2		washer DIN 125 - A 5,3		02.61.05.31.01.20
9-13	1	40300604	angle holder incl. mounting set		40.30.06.04.00.10
10	1		round-head screw DIN 6003 M6x20		01.11.08.54.00.40
11	2		washer DIN 9021- 6,4- PA		02.79.06.41.60.70
12	1		washer DIN 9021 -6,4		02.79.06.41.60.20
13	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
14-17	1	40300610	calf rest 150x120x30 PU black incl. mounting set		04.65.09.15.13.20
o.B.	1	40300611	calf rest 230x170x30 PU black incl. mounting set		04.65.09.22.15.20
15	2		lens head screw with hexagon socket ISO 7380 - M5x20		01.42.07.94.65.00
16	2		lock washer S5		02.99.10.50.01.10
17	2		washer DIN 9021 - M5		02.79.05.31.20.20

Calf rest

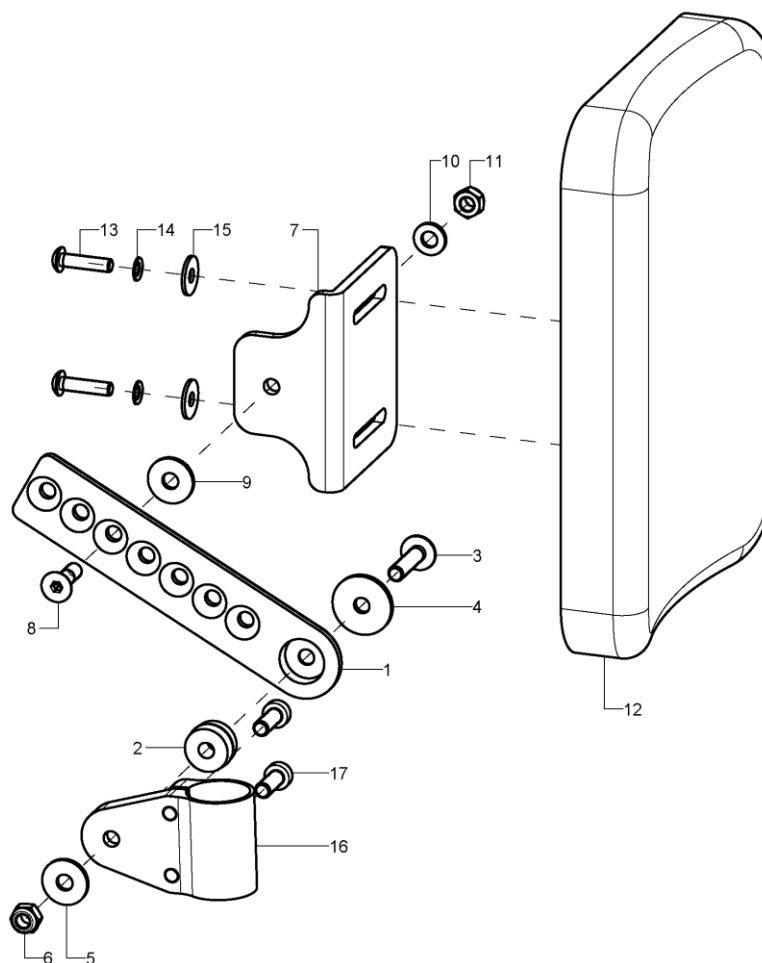


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-15	1	403006			
1-6	1	40301003			40.30.10.01.00.75
2	1				40.24.01.02.00.54
3	1		lens head screw with hexagon socket M6x25		01.43.08.56.65.00
4	1		washer 23,5x6,2x2 black anodized		58.67.01.07.00.67
5	1		washer DIN 9021 -6,4		02.79.06.41.60.20
6	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
7-11	1	40300604	angle holder incl. mounting set		40.30.06.04.00.10
8	1		counter sunk screw with hexagon socket DIN 7991 M6x20		01.65.08.54.02.40
9	1		washer DIN 9021- 6,4- PA		02.79.06.41.60.70
10	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
11	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
12-15	1	40300610	calf rest 150x120x30 PU black incl. mounting set		04.65.09.15.13.20
o.B.	1	40300611	calf rest 230x170x30 PU black incl. mounting set		04.65.09.22.15.20
13	2		lens head screw with hexagon socket ISO 7380 - M5x20		01.42.07.94.65.00
14	2		lock washer S5		02.99.10.50.01.10
15	2		washer DIN 9021 - M5		02.79.05.31.20.20
16-17	1	40550609			40.55.06.09.00.76
17	2		cylinder screw, low head DIN 7984-M6x16		01.24.08.52.05.00

Calf belt

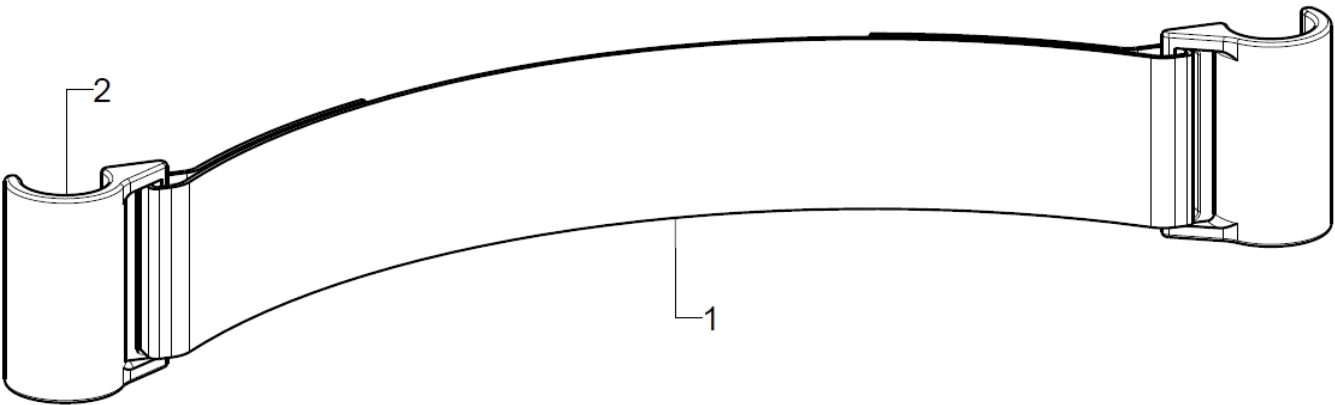

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-2	1	4041			
1	1	404108			40.41.03.XX.00.10
2	2	404106			40.41.06.01.00.10


Seat

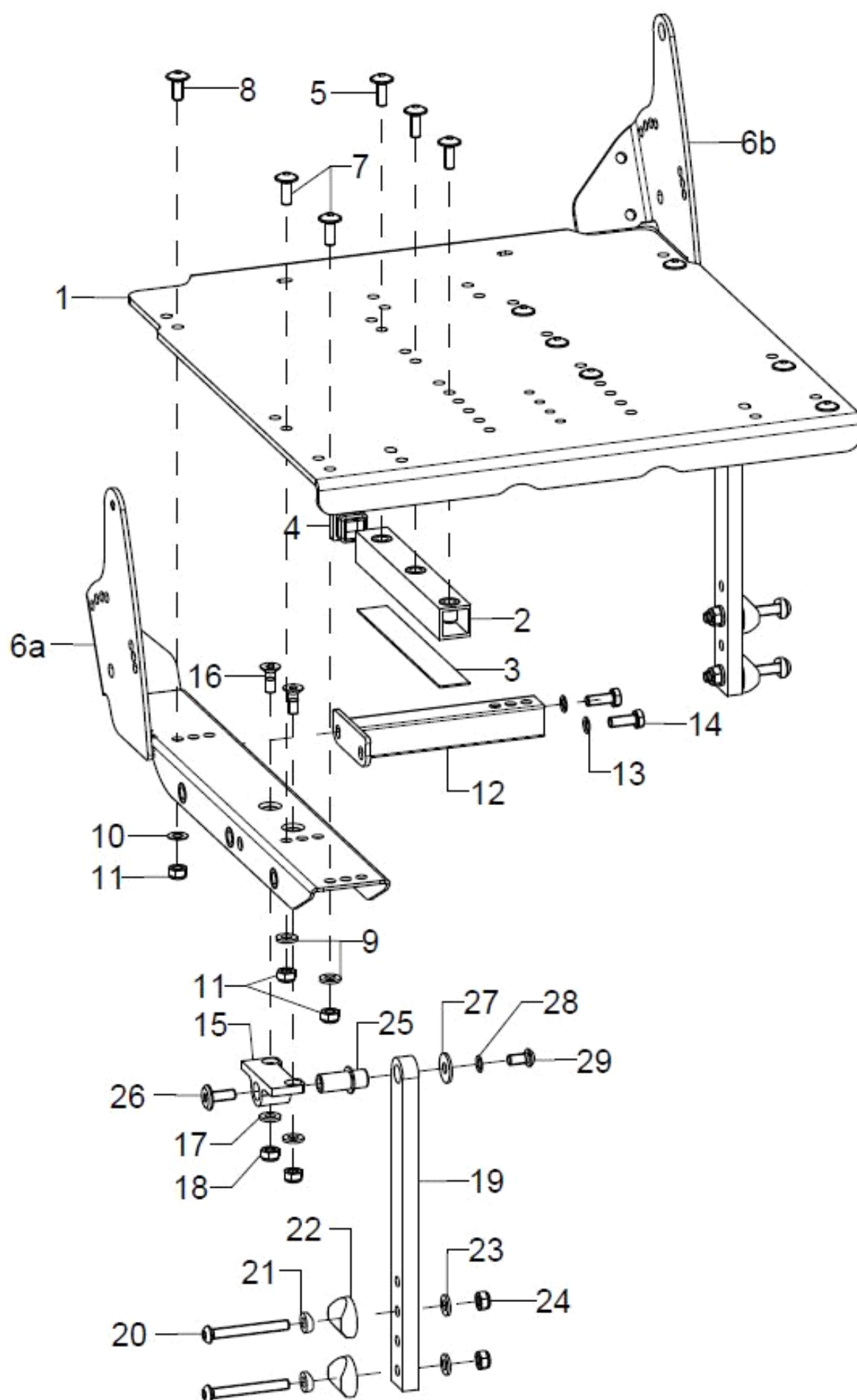



pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	19271100			19.27.11.00.00.80
2-5	2	19270501			19.27.05.01.00.76
o.B.	2	19270502			19.27.05.02.00.76
o.B.	2	19270503			19.27.05.03.00.76
3	1				19.27.05.10.01.76
o.B.	1				19.27.05.10.02.76
o.B.	1				19.27.05.10.03.76
4	1		plugs 20x20		04.63.11.20.00.10
5	3		lens head screw with hexagon socket M6x16		01.43.08.52.65.00
6a-9	1	19271302			19.27.13.00.02.80
6b-9	1	19271301			19.27.13.00.01.80
7	2		lens head screw with hexagon socket M6x16		01.43.08.52.65.00
8	1				01.43.08.50.65.00
9	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
10	1		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
11	3		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
12-14	2	19271000			19.27.10.00.00.80
13	2		lock washer S6		02.99.10.60.01.10
14	2		hexagon screw DIN 933 - M6x16		01.04.08.52.02.40
15-18	2	19271006			19.27.10.06.00.80
16	2		counter sunk screw with hexagon socket DIN 7991 M6x16		01.65.08.52.02.40
17	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
18	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
19-24	2	19270310			19.76.03.01.00.76
20	2		lens head screw with hexagon socket ISO 7380 - M6x50		01.42.08.62.65.00
21	2		saddle washer Ø12/6x2,5		35.04.16.01.00.10
22	2		saddle washer black ø 24/6		04.63.39.24.06.30
23	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
24	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
25-29	2	19271005			19.27.10.05.00.80
26	1				01.25.08.52.04.80
27	1		washer DIN 9021 -6,4		02.79.06.41.60.20
28	1		lock washer S6		02.99.10.60.01.10
29	1		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00

Seat extension

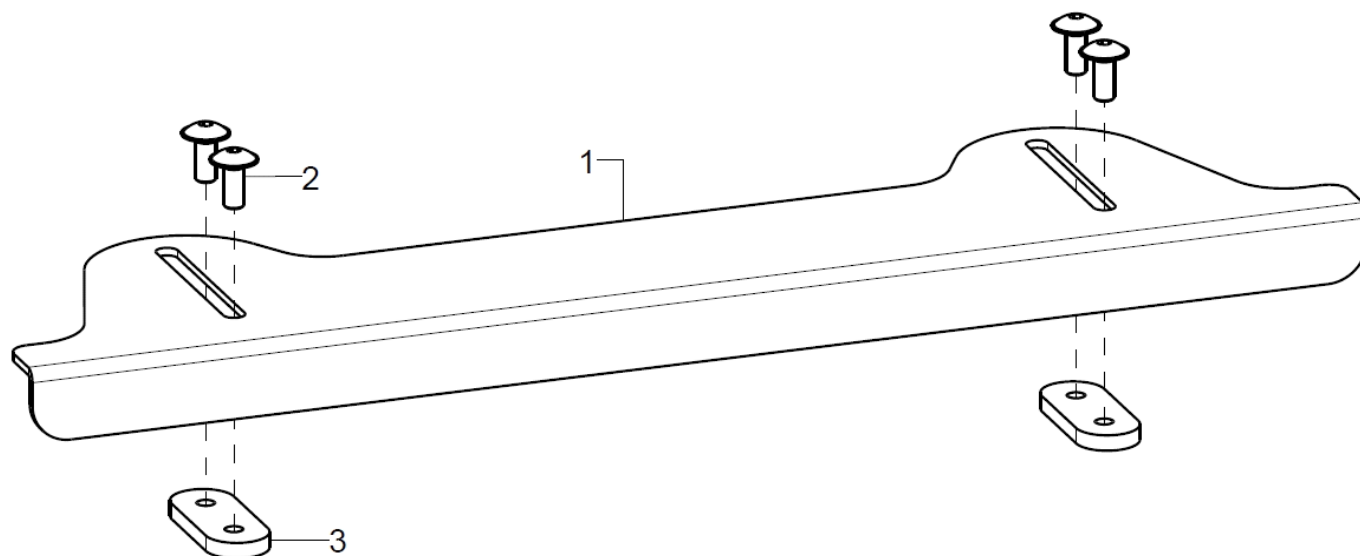

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-4	1	19271200			19.27.12.00.00.80
2	4		lens head screw with hexagon socket M5x12		01.43.07.89.65.00
3	2				19.02.05.01.00.24




Seat cushion

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	1	891002	anatomical seat cushion		XXXXXXXXXXXXX
o.B.	1	891001			
o.B.	1	891003	cover for anatomical seat cushion		XXXXXXXXXXXXX
o.B.	1	891004	incontinence cover for anatomical seat cushion		XXXXXXXXXXXXX
o.B.	1	8003	seat cushion, synthetic foam, 3cm thick		XXXXXXXXXXXXX
o.B.	1	8005	seat cushion, synthetic foam, 5cm thick		XXXXXXXXXXXXX
o.B.	1	8103	seat cushion, memory foam, 3cm thick		XXXXXXXXXXXXX
o.B.	1	8105	seat cushion, memory foam, 5cm thick		XXXXXXXXXXXXX
o.B.	1	800050			
o.B.	1	8603	incontinence protection for 3cm cushion (non refundable)		
o.B.	1	8605	incontinence protection for 5cm cushion (non refundable)		

Stoffmuster	XX
schwarz	30
grau	31
nachtblau	32
ultramarinblau	33
petrol	34
applegreen	35
bordeauxrot	36
rot	37
pink	38


Gas pressure spring seat tilt

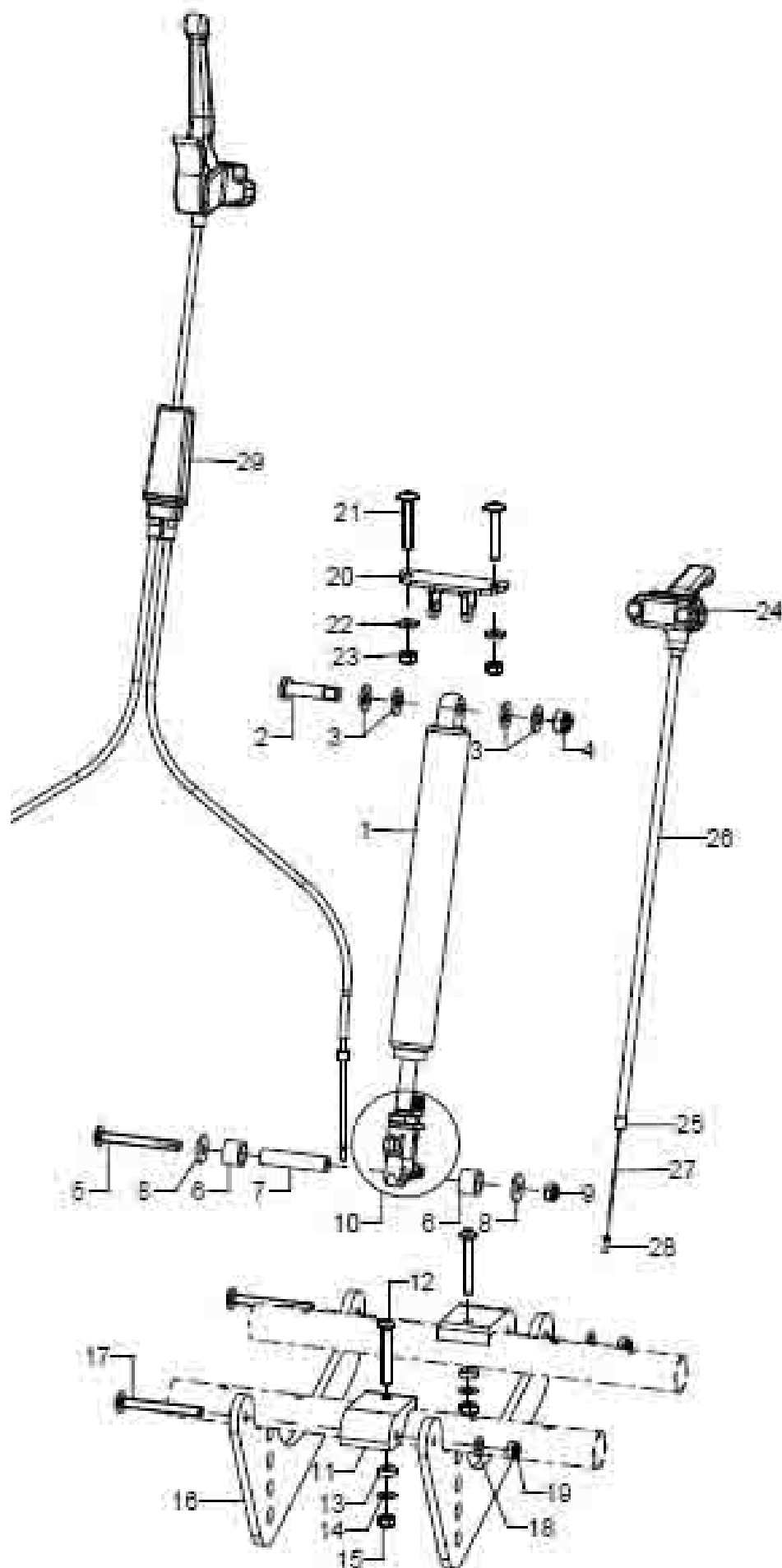




pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1+10	1	192501			04.27.09.20.00.60
2-4	1	190503	fitted bolt M8x35 SW13		19.05.03.10.00.28
3	4		washer DIN 125 - A 8,4 - PA black		02.61.08.41.61.71
4	1		hexagon nut self-locking low form DIN 985 - M8		02.32.00.80.01.20
5-9	1	192504			
5	1		lens head screw with hexagon socket M6x55		01.43.08.63.65.00
6	2				19.22.03.07.00.47
7	1				19.25.01.02.00.76
8	2		washer DIN 9021 -6,4		02.79.06.41.60.20
9	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
10	1	190521	gas pressure spring release knob		04.27.00.99.81.00
11-18	1	192505			XXXXXXXXXX
11-15	2(1)	19250104			19.25.01.04.00.76
o.B.	1				19.25.01.06.00.76
12	2		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
13	2		saddle washer Ø12/6x2,5		35.04.16.01.00.10
14	2				02.68.06.00.30.30
15	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
16-19	2	19250103			19.25.01.03.00.76
17	2(1)		lens head screw with hexagon socket ISO 7380 - M6x55		01.42.08.63.65.00
o.B.	2				01.43.08.47.62.50
18	2(1)		washer DIN 125 - A 6,4		02.61.06.41.41.20
19	2(1)		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
20-23	1	192702			19.27.02.00.00.76
21	2		lens head screw with hexagon socket M6x35		01.43.08.59.65.00
22	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
23	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
24-28	1	190512	gas pressure spring release incl. mounting set		19.05.12.99.00.10
24	1	190513	hand release lever for gas pressure spring, not lockable		04.54.92.12.20.10
25-28	1	190514	bowden cable incl. cover and connection parts		19.05.14.99.00.71
25	2	352412	end bush metal ø5		04.54.91.50.10.41
26	1	352013	outer shell for Bowden cable black 150mm		04.54.91.21.53.00
27	1	190517	inner cable for gas pressure spring 1.3x2000		04.54.91.11.31.30
28	1	352005	end cap for inner Bowden cable ø1-1,5 mm		04.54.91.50.10.40
29-30	1	190515			
29	1	190502			04.54.91.80.40.11

Abduction wedge

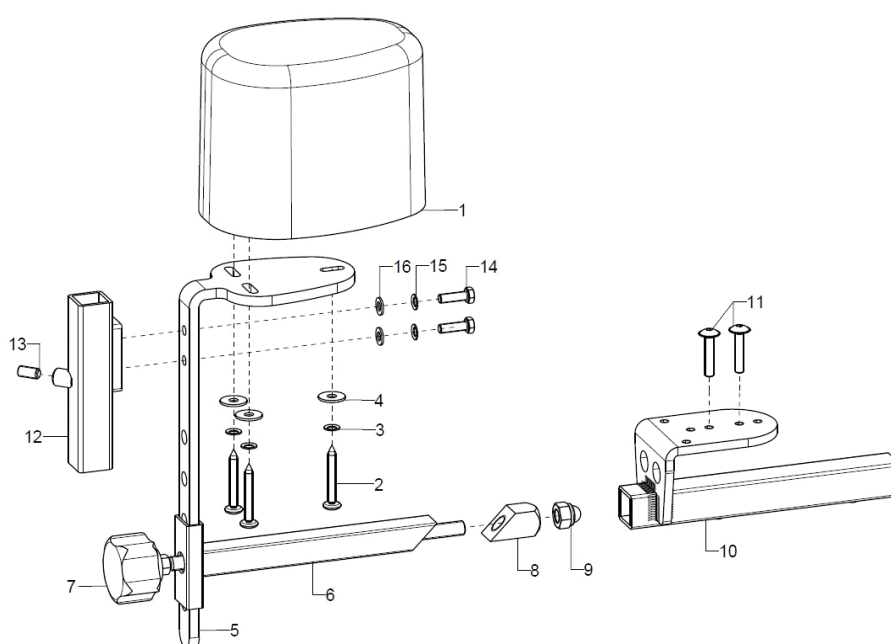


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-10	1	1952	abduction wedge size s		XXXXXXXXXXXX
o.B.	1	1953	abduction wedge size m		XXXXXXXXXXXX
o.B.	1	1954	abduction wedge size l		XXXXXXXXXXXX
1	1	195002	abduction wedge small		04.65.17.09.01.20
o.B.	1	195003	abduction wedge medium		04.65.17.11.03.20
o.B.	1	195004	abduction wedge large		04.65.17.13.02.20
2-4		19500299	mounting set abduction wedge on holder angle		XXXXXXXXXXXX
2	3				01.88.07.99.00.80
3	3		lock washer S5		02.99.10.50.01.10
4	3		washer DIN 9021 - M5		02.79.05.31.20.20
5-11	1	1951			
5	1	195101	bracket for abduction wedge		19.50.03.02.00.10
6	1	195103	weldment clamber		19.50.02.01.00.10
7	1	684004	clamping set for posterior truss pad or abduction wedge		XXXXXXXXXXXX
7-9	1				19.05.03.01.99.82
8	1				68.40.04.04.00.51
9	1				02.33.00.80.41.20
10-11	1	195102	weldment holder abduction wedge		19.50.01.01.00.10
11	2				01.43.07.94.62.60
o.B.	2		lens head screw with hexagon socket M5x25		01.43.07.96.65.00
12-16	1	400105	universal leg support socket for combination abduction wedge with therapy table incl. mounting set		40.01.01.05.00.28
13	1		setscrew with hexagon socket DIN 913, M6x12		01.67.08.49.61.20
14	2		hexagon screw DIN 933 - M5x16		01.04.07.92.02.40
15	2		lock washer S5		02.99.10.50.01.10
16	2		washer DIN 125 - A 5,3		02.61.05.31.01.20

Therapy table, version 1, hinged sidewards

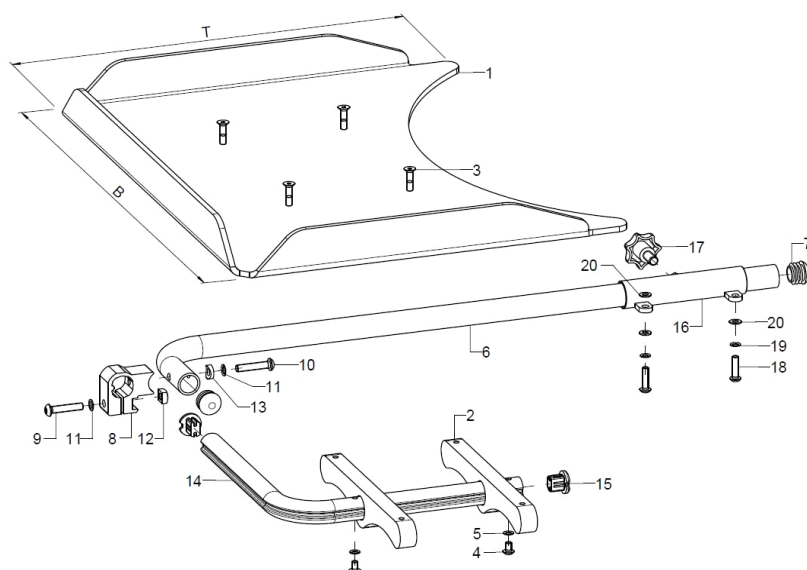


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-20	1	9204XX	therapy table version 1 swingaway to the side complete		XXXXXXXXXXXX
1	1	920030	therapy table size 1 (B45xT36) polycarbonate		92.00.10.01.00.51
o.B.	1	920031	therapy table size 2 (B59xT45) polycarbonate		92.00.11.01.00.51
o.B.	1	920032	therapy table size 3 (B70xT50) polycarbonate		92.00.12.01.00.51
o.B.	1	920033	therapy table size 1 (B50xT40) wood		92.00.13.01.00.10
o.B.	1	920034	therapy table size 2 (B60xT50) wood		92.00.14.01.00.10
2-20	1	9204	mounting set for therapy table version 1 complete without table		
2	2	920403	foldable supporting frame 15x35x200		92.04.01.01.00.10
3-5	1	92040399	mounting set support bracket		XXXXXXXXXXXX
3	4		counter sunk screw with hexagon socket DIN 7991 M5x20		01.65.07.94.05.00
4	2		lens head screw with hexagon socket ISO 7380 - M5x8		01.42.07.87.65.00
5	2		lock washer S5		02.99.10.50.01.10
6+7	1	920402	holding tube, incl. plug		92.04.02.00.00.10
7	2		plugs for tube Ø20x1-2, black		04.63.10.20.00.10
8	1	92040403	clamp unit incl. mounting set		92.04.04.03.00.10
9-13	1	92040499	mounting set clamp unit		XXXXXXXXXXXX
9	1		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
10	1				01.42.08.59.62.60
11	2		lock washer S6		02.99.10.60.01.10
12	1		square nut DIN 557 M6		02.13.00.60.01.20
13	1				40.04.16.01.00.10
14+15	1	920404	foldable bail tube incl tube end plug		92.04.04.00.00.10
15	2	620105	plug for profile tube		62.01.05.00.00.74
16	2	920407	tube adapter, weldment incl. mounting set		92.04.05.01.00.10
17-20	2	92040599	mounting set tube intake		XXXXXXXXXXXX
17	1		7-star knob Ø32 - M8x15 external thread, black		04.05.47.14.15.40
18	2		lens head screw with hexagon socket ISO 7380 - M5x20		01.42.07.94.65.00
19	2		lock washer S5		02.99.10.50.01.10
20	3		washer DIN 125 - A 5,3		02.61.05.31.01.20

Therapy table, version 2, fixed to the arm pad

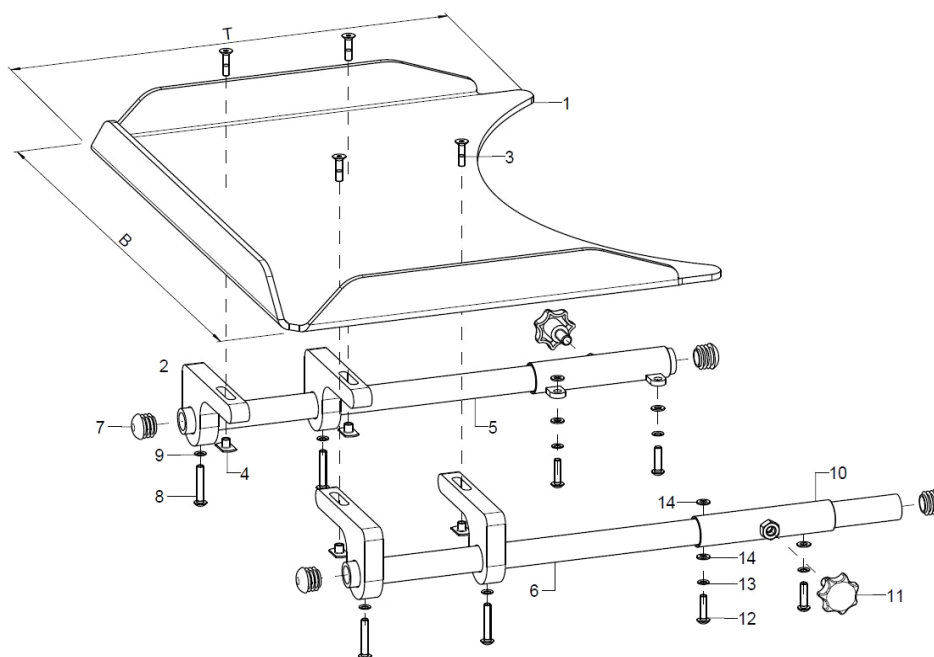


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-14	1	9207XX	therapy table version 2 on arm pad complete		XXXXXXXXXXXX
1	1	920030	therapy table size 1 (B45xT36) polycarbonate		92.00.10.01.00.51
o.B.	1	920031	therapy table size 2 (B59xT45) polycarbonate		92.00.11.01.00.51
o.B.	1	920032	therapy table size 3 (B70xT50) polycarbonate		92.00.12.01.00.51
o.B.	1	920033	therapy table size 1 (B50xT40) wood		92.00.13.01.00.10
o.B.	1	920034	therapy table size 2 (B60xT50) wood		92.00.14.01.00.10
2-14	2	9207	mounting set for therapy table version 2 on arm rest		
2	4	920705	plugable supporting frame 15x61x120		92.07.05.01.00.51
o.B.	4				92.07.05.02.00.00
3-4	1	92070599	mounting set support bracket		XXXXXXXXXXXX
3	4		counter sunk screw with hexagon socket DIN 7991 M5x20		01.65.07.94.05.00
4	4		weld nut C- M5x7,5		02.49.33.03.00.10
5+7-9	1	920702	holder tube 20x3x400 incl. mounting set		92.07.02.02.00.29
6-9	1	920704	holder tube 20x3x450 incl. mounting set		92.07.02.04.00.29
7	2		plugs for tube ø20x1-2, black		04.63.10.20.00.10
8	2		lens head screw with hexagon socket ISO 7380 - M5x30		01.42.07.98.65.00
9	2		lock washer S5		02.99.10.50.01.10
10-14	1	920405	tube adapter, weldment, right incl. mounting set		92.07.01.01.02.10
10-14	1	920406	tube adapter, weldment, left incl. mounting set		92.07.01.01.01.10
12	2		lens head screw with hexagon socket ISO 7380 - M5x20		01.42.07.94.65.00
11-14	2	92040599	mounting set tube intake		XXXXXXXXXXXX
11	1		7-star knob ø32 - M8x15 external thread, black		04.05.47.14.15.40
13	2		lock washer S5		02.99.10.50.01.10
14	3		washer DIN 125 - A 5,3		02.61.05.31.01.20

Back, adjustable in angle by raster

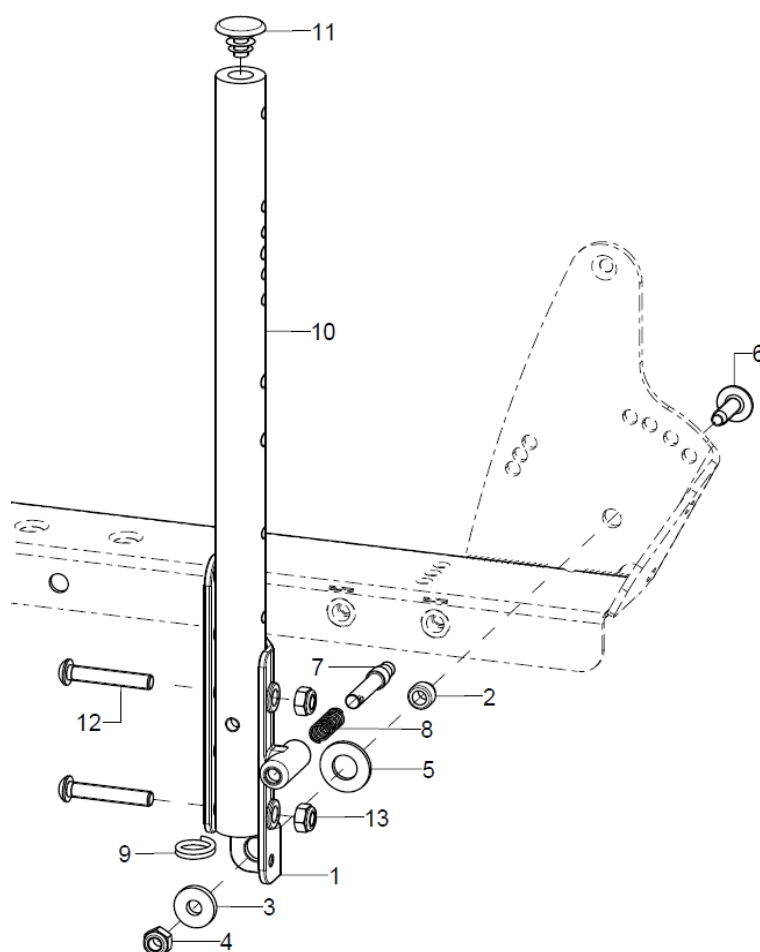

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-6	1	62370204	U-part 137 mm, right		61.09.05.01.02.00
o.B.	1	62370203	U-part 137 mm, left		61.09.05.01.01.00
2-6	2	62370298	mounting set U-part 137 mm		XXXXXXXXXXXX
2	1				61.03.02.01.00.18
o.B.	1				02.68.10.00.50.30
o.B.	1				02.68.10.00.50.30
3	1		washer DIN 7349 - 6,4		02.77.06.53.00.20
4	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
5	1		washer DIN 125 - A 10,5 - PA black		02.61.10.52.01.71
6	1		lens head screw with hexagon socket M6x20		01.43.08.54.65.00
7-9	2	610109	locking bolt incl. mounting set (spring and ring)		61.01.02.04.00.24
8	1		pressure spring DIN 2098 0,7x7,65/9,05x21		04.21.00.80.21.00
9	1		wire form ring 2x16		04.29.04.42.20.00
o.B.	1	61130601			61.13.06.01.00.83
11	1				04.63.10.20.01.10
10+11	2	623304XX			62.33.02.02.00.00


Back with gas pressure spring adjustment

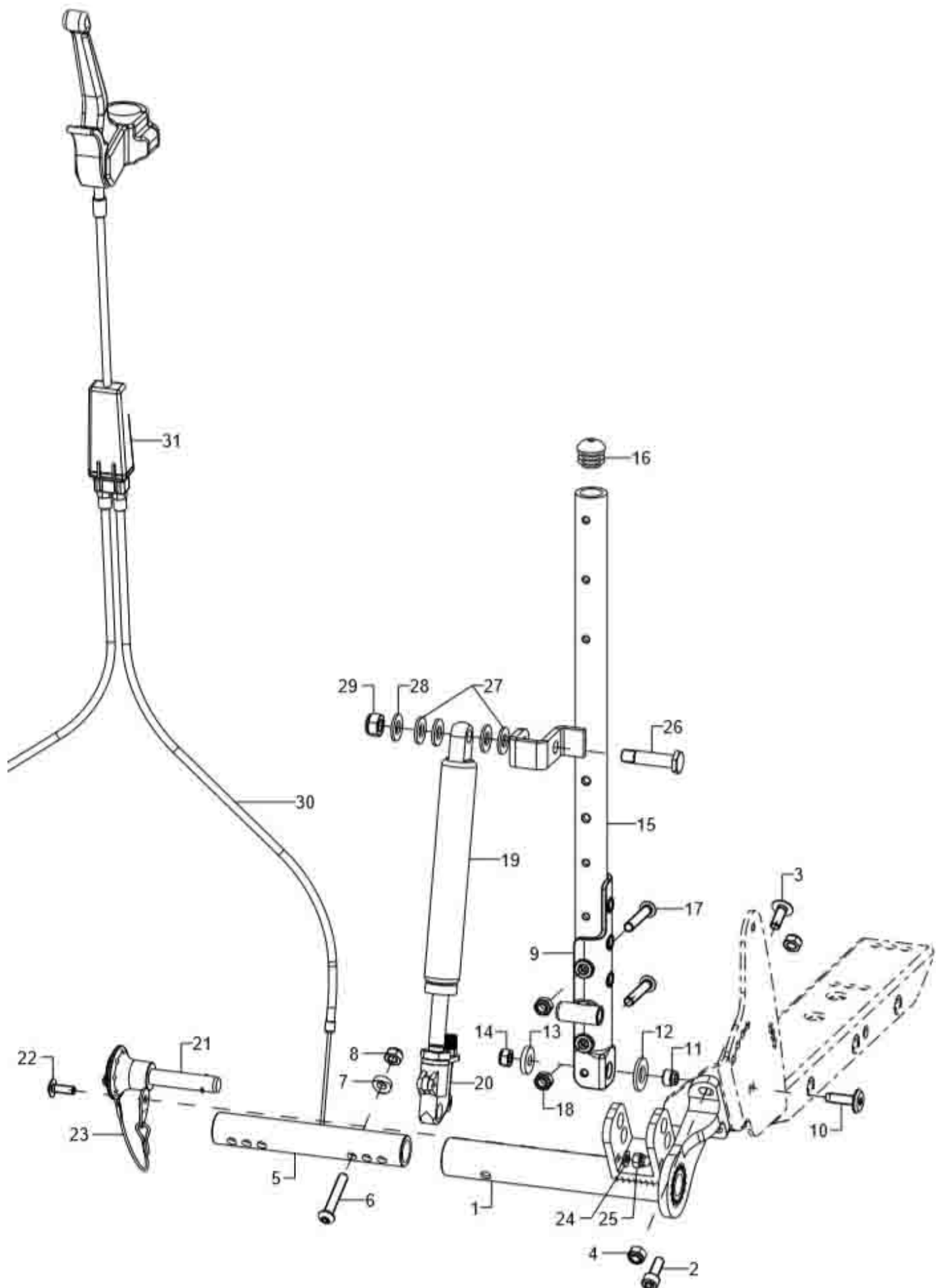



pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-32	1	61800001			
1-3	1	61370202			61.37.02.00.02.80
o.B.	1	61370201			61.37.02.00.01.80
2	1		cylinder screw, low head DIN 7984-M6x16		01.24.08.52.05.00
3	1		lens head screw with hexagon socket M6x16		01.43.08.52.65.00
4	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
5-8	1	61370301			61.37.03.01.00.80
6	2		lens head screw with hexagon socket ISO 7380 - M6x35		01.42.08.59.65.00
7	2		saddle washer Ø12/6x2,5		35.04.16.01.00.10
8	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
9	1	62370204	U-part 137 mm, right		61.09.05.01.02.00
o.B.	1	62370203	U-part 137 mm, left		61.09.05.01.01.00
10-14	2	62350199	mounting set U-part 137 mm		XXXXXXXXXXXX
10	1				01.67.08.54.60.84
11	1				61.03.02.01.00.18
12	1		washer DIN 125 - A 10,5 - PA black		02.61.10.52.01.71
13	1		washer DIN 7349 - 6,4		02.77.06.53.00.20
14	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
15+16	1	623502RH	back tube-gas pressure spring right, incl. cap		62.35.02.02.00.71
o.B.	1	623501RH	back tube-gas pressure spring left, incl. cap		62.35.02.01.00.71
16	1		plugs for tube Ø18x1-2, black		04.63.10.18.00.10
17+18	2	62330099	mounting set back tube		XXXXXXXXXXXX
17	2				01.42.08.59.62.60
18	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
19+20	2	190505			04.27.04.10.00.50
o.B.	2	190507	gas pressure spring 200/40 complete with release knob		04.27.04.20.00.60
20	1	190521	gas pressure spring release knob		04.27.00.99.81.00
21-25	2	613509	ball lock bolt complete		04.14.10.80.35.10
22	1		lens head screw with hexagon socket M5x20		01.43.07.92.05.00
23	1				04.14.10.80.35.11
24	1		washer DIN 125 - A 5,3		02.61.05.31.01.20
25	1		hexagon cap nut low form with clamp DIN 986 - M5		02.33.00.60.21.20
26-28	2	190503	fitted bolt M8x35 SW13		19.05.03.10.00.28
27	4		washer DIN 125 - A 8,4 - PA black		02.61.08.41.61.71
28	1		washer DIN 125 - A 8,4		02.61.08.41.41.20
29	1		hexagon nut self-locking low form DIN 985 - M8		02.32.00.80.01.20
30-32	1	190515			
30	2	190514	bowden cable incl. cover and connection parts		19.05.14.99.00.71
31	1	190502			04.54.91.80.40.11
o.B.	1	352014	hand brake lever adjustable incl. childproof		XXXXXXXXXXXX

ERGO back shell

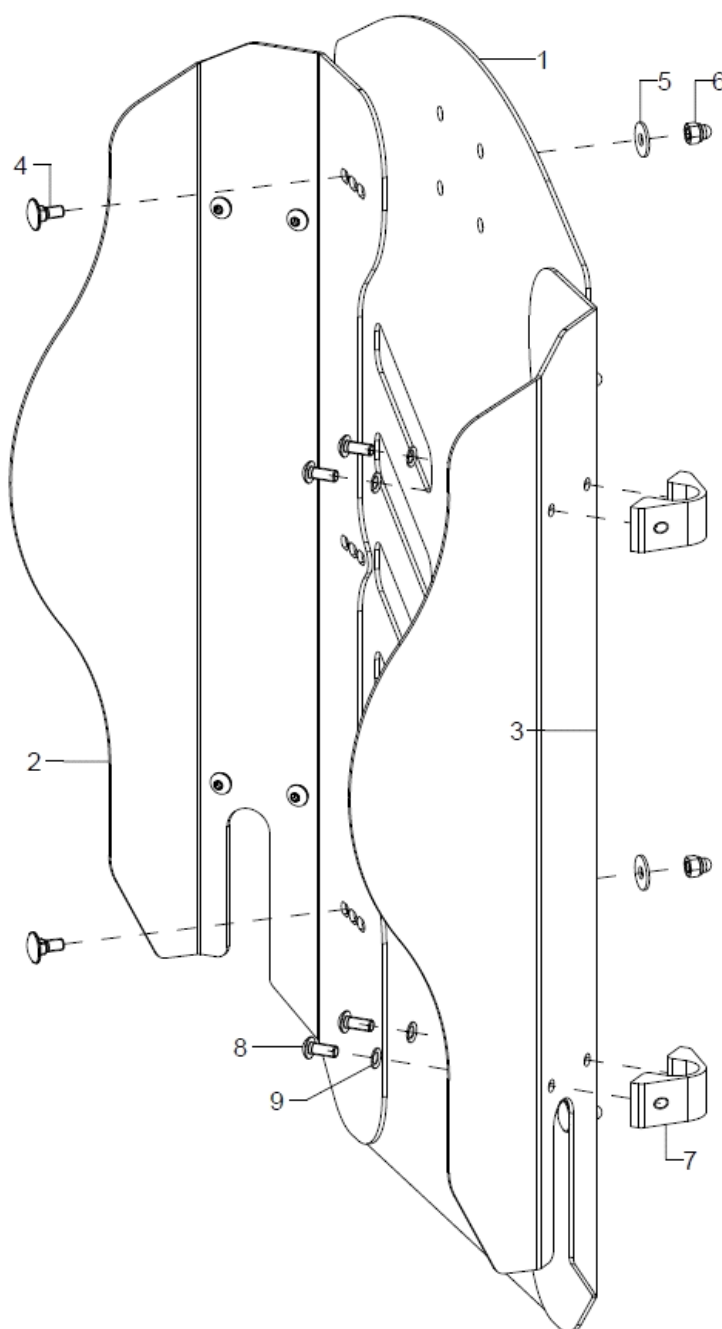

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-10	1	69800000			
1	1	69800101			69.80.01.01.00.80
2+4-6	1	69800202			69.80.02.00.02.80
3-6	1	69800201			69.80.02.00.01.80
4	2		round-head screw DIN 6003 M5x12		01.11.07.89.00.40
5	2		washer DIN 9021 - M5		02.79.05.31.20.20
6	2		hexagon cap nut low form with clamp DIN 986 - M5		02.33.00.60.21.20
7-10	4	69800301			69.80.03.01.00.80
8	2		lens head screw with hexagon socket ISO 7380 - M5x12		01.42.07.89.65.00
9	2		lock washer S5		02.99.10.50.01.10




Back cushion

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	1	8517	back cushion 3cm with zipper		XXXXXXXXXXXXX
o.B.	1	8517XX	cover for back cushion		XXXXXXXXXXXXX
o.B.	1	89120100			
o.B.	1	89120101			89.12.01.00.00.80
o.B.	1	89120201			



Stoffmuster	XX
schwarz	30
grau	31
nachtblau	32
ultramarinblau	33
petrol	34
applegreen	35
bordeauxrot	36
rot	37
pink	38

Curved back plate

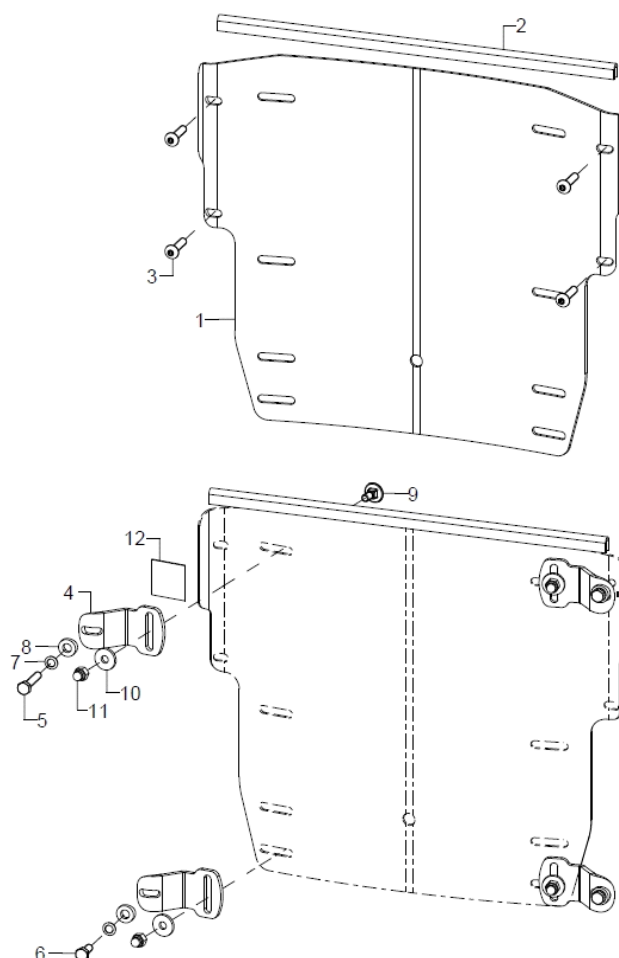

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	6950XX			69.50.XX.00.00.00
o.B.	1	6931XX			69.31.XX.00.00.00
o.B.	1	6932XX			69.32.XX.00.00.00
o.B.	1	6953XX			69.53.XX.00.00.00
o.B.	1	6954XX			69.54.XX.00.00.00
2	1	450030	edge protection-steel clamp band, 1-2 mm black		00.45.07.10.99.10
3	4				01.42.07.95.65.00
4-12	2	692593			XXXXXXXXXXXX
4-12	2	692598	mounting set curved back plate		XXXXXXXXXXXX
5-11	2	692599			
4	1	692511			
o.B.	1	692510	screwed on holding plate		69.25.05.01.00.69
5	1		hexagon screw DIN 933 - M5x25		01.04.07.96.02.40
6	1		hexagon screw DIN 933 - M5x16		01.04.07.92.02.40
7	1		lock washer S5		02.99.10.50.01.10
8	1				40.04.16.01.00.10
9	1		round-head screw DIN 6003 M5x12		01.11.07.89.00.40
10	1		washer DIN 7349 - 5,3		02.77.05.22.00.20
11	1		hexagon cap nut low form with clamp DIN 986 - M5		02.33.00.60.21.20
12	2		Velcro strap 25 mm		00.42.61.02.50.11

Übersicht Muldenrücken

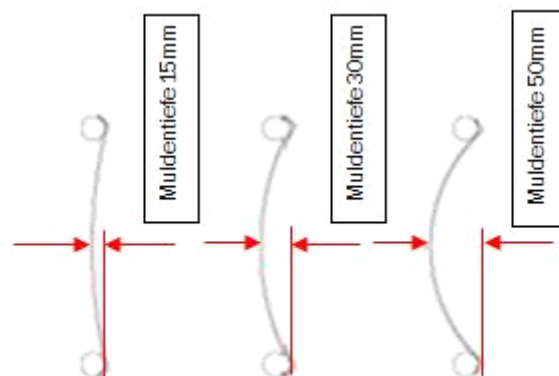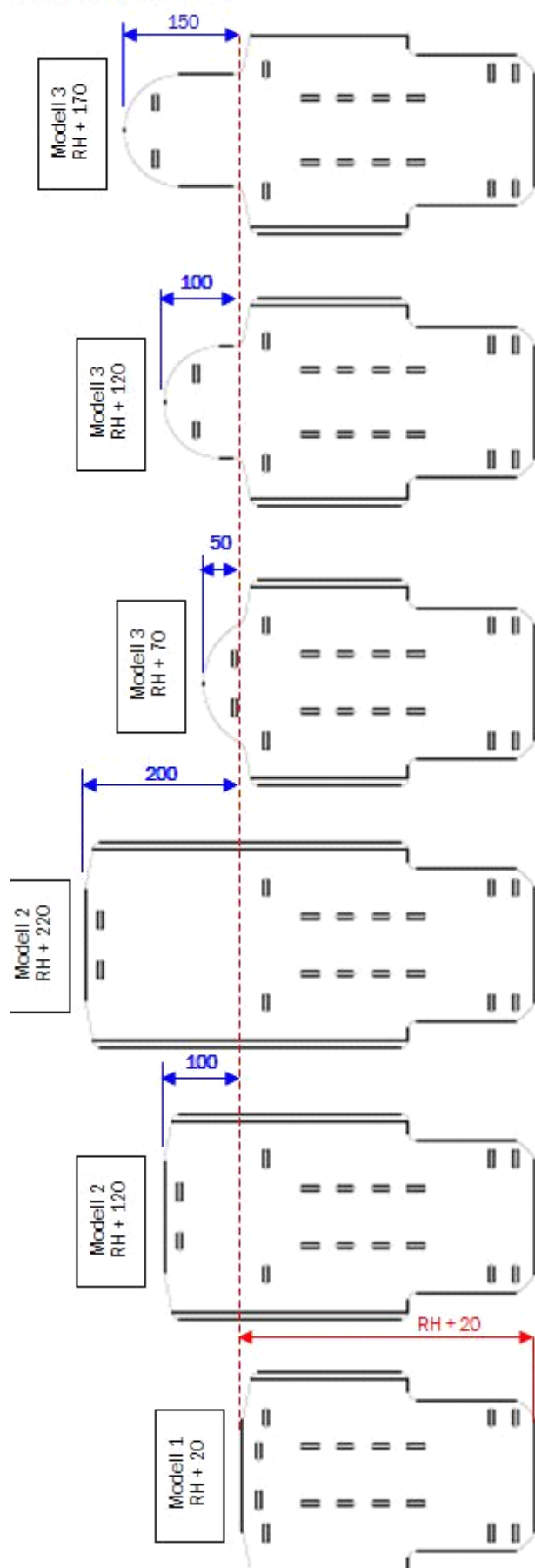


Back extension

Curved back version 1: only from BH 35 cm


Curved back version 3: only from BH 30 cm, only with BH+5cm

If the curved back is attached with the fitting set for solid back, a back extension is only possible from SW28.


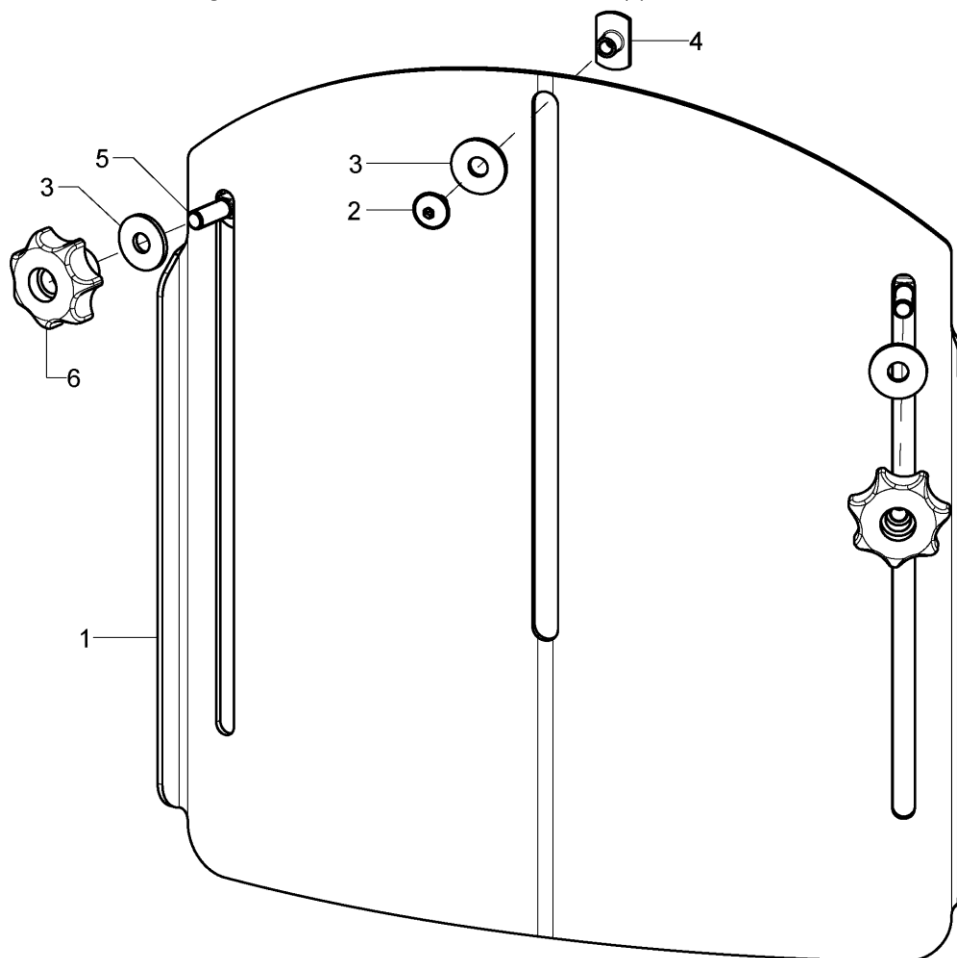


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	686092			68.60.02.00.00.00
2-6	1	68607299	mounting set back extension		XXXXXXXXXXXX
2	1		lens head screw with hexagon socket M5x8		01.43.07.87.62.60
3	3		washer DIN 9021- 6,4- PA		02.79.06.41.60.70
4	1		weld nut C- M5x7,5		02.49.33.03.00.10
5	2		round-head screw DIN 6003 M6x20		01.11.08.54.00.40
6	2		7-star knob ø32 - M6 internal thread continuous, black		04.05.37.64.20.30

Seat cover, adjustable

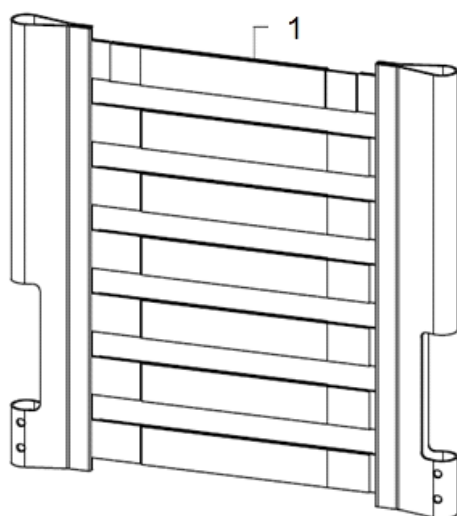


Stoffmuster	XX
Betex schwarz	20
Betex blau	21
Betex rot	22
Betex grau	23
Betex grün	24
Betex gelb	25
Betex orange	27

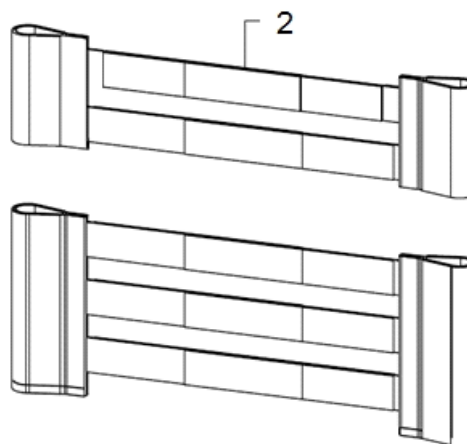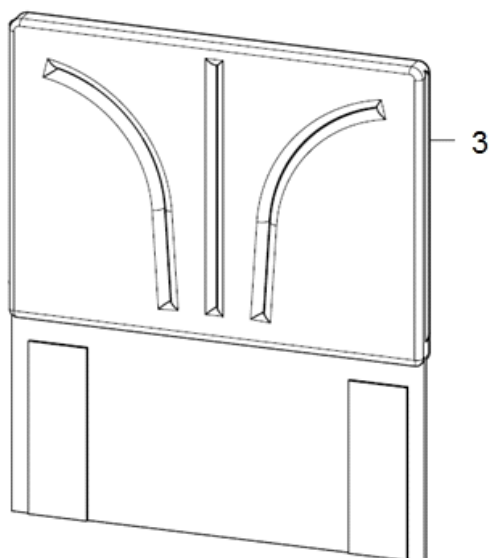


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1+3	1	670103XX			XX.XX.XX.XX.XX.XX
2+3	1	610105XX			XX.XX.XX.XX.XX.XX
1	1	670500	back cover adjustable		XXXXXXXXXXXX
2	1	670505			
3	1	670599	back cover with cushion 2 cm		XX.XX.XX.XX.XX.XX
o.B.	2		counter sunk screw with hexagon socket DIN 7991 M5x16		01.65.07.92.05.00
o.B.	2		rosette ø13/6x2, black		04.14.70.60.02.00

Stabilizer bar + push bar holder

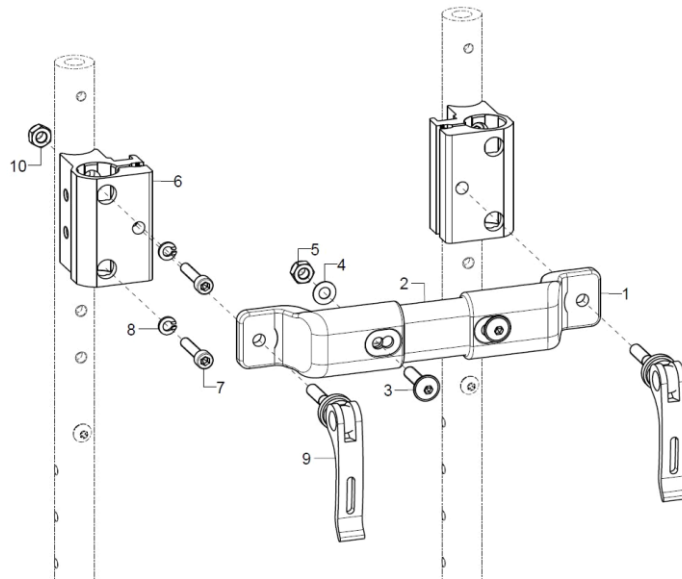

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5	1	64090302			
1	2	64090103			64.09.01.03.00.00
2-5	1	64090110			64.09.01.07.00.00
3	2		lens head screw with hexagon socket M6x25		01.43.08.56.65.00
4	2		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
5	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
6-10	2	650061	push bail holder R10		65.00.51.02.00.10
7	2		cylinder screw DIN 912 M5x25		01.22.07.96.62.40
8	2		spring ring DIN 127 - A 5		02.85.30.50.05.10
9	1	652225			04.13.40.60.35.10
10	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20

SB	L Stabilisierungsrohr
30	175
32	195
34	215
36	235
38	255
40	275
42	295
44	315
46	335
48	355

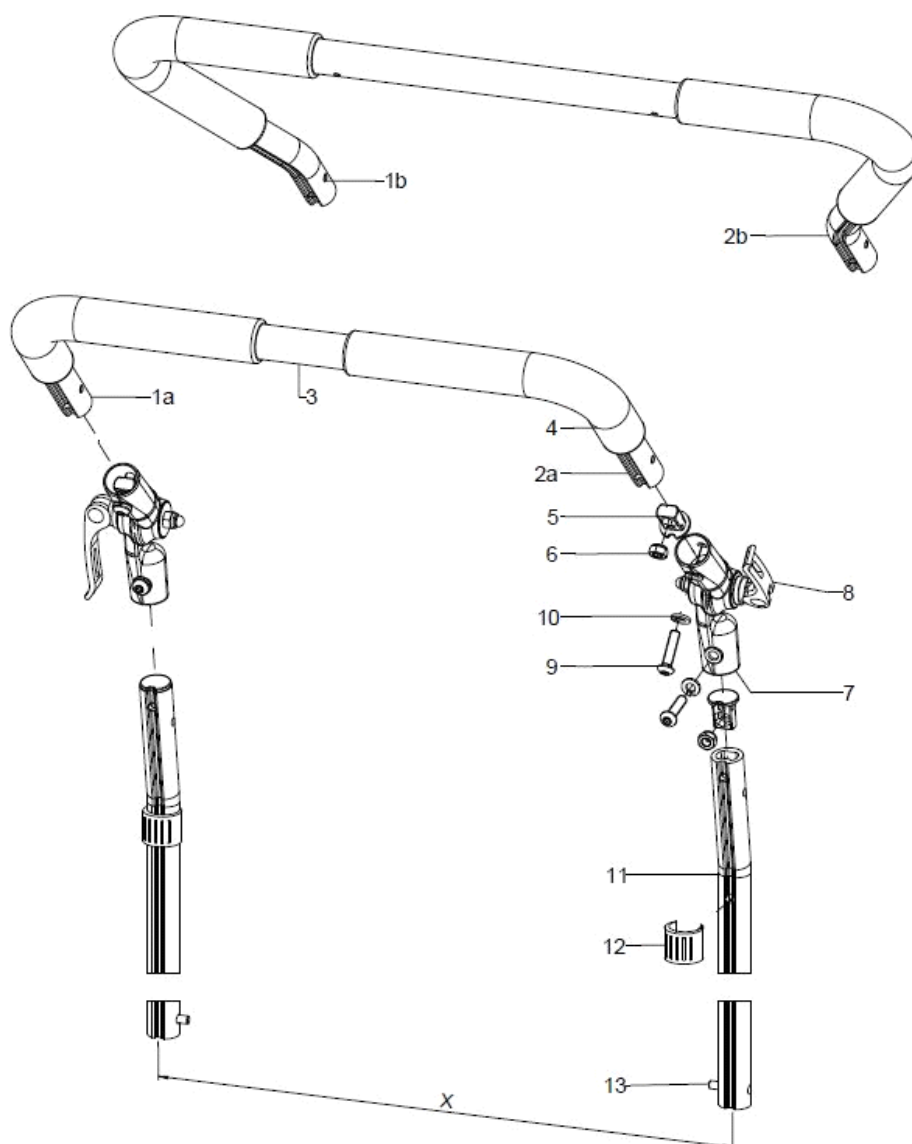


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-13	1	6532	push bail extendable by 20 cm, 307° angle adjustment metal joint, complete		XXXXXXXXXXXX
1a+2a+3-4	1	653001	push bail 90° pre-assembled for SW 40 – 46 when push handle clamp holder is mounted to the back tube		XXXXXXXXXXXX
1b+2b+3+4	1	653002	push bail cropped pre-assembled		XX.XX.XX.XX.XX.XX
1a	1	653009	push bail, bend part- 20x150x150, left		65.34.00.02.01.66
1b	1	653006	push bail left part		65.30.01.02.01.47
2a	1	653010	push bail, bend part- 20x150x150, right		65.34.00.02.02.66
2b	1	653007	push bail right part		65.30.01.02.02.47
3	1	653004	steel tube incl. riveting		65.30.01.04.00.24
4	2	653005	tube cover - rubber 260 mm		65.30.03.01.00.47
5+6	4	653406	plug for nut M6 incl hexagon nut		65.22.20.10.00.00
6	4		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
7-10	2	652220	angle adjusting element metal 307° compl with eccentric clamp and mounting set		65.22.20.99.00.10
8	1	652224	eccentric clamp length 35 mm M6x20, black (incl. spring, nut, washer, bush)		65.22.24.99.00.76
9	2		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
10	2		spring ring DIN 127- A6		02.85.30.60.05.10






11	2	653202	profile tube support 200	65.31.02.01.00.24
12	2	046350200010	clamp conductor black, ø 18-20	04.63.50.20.00.10
13	2	650160	locking unit for push handles	65.01.30.03.00.10


Head rest

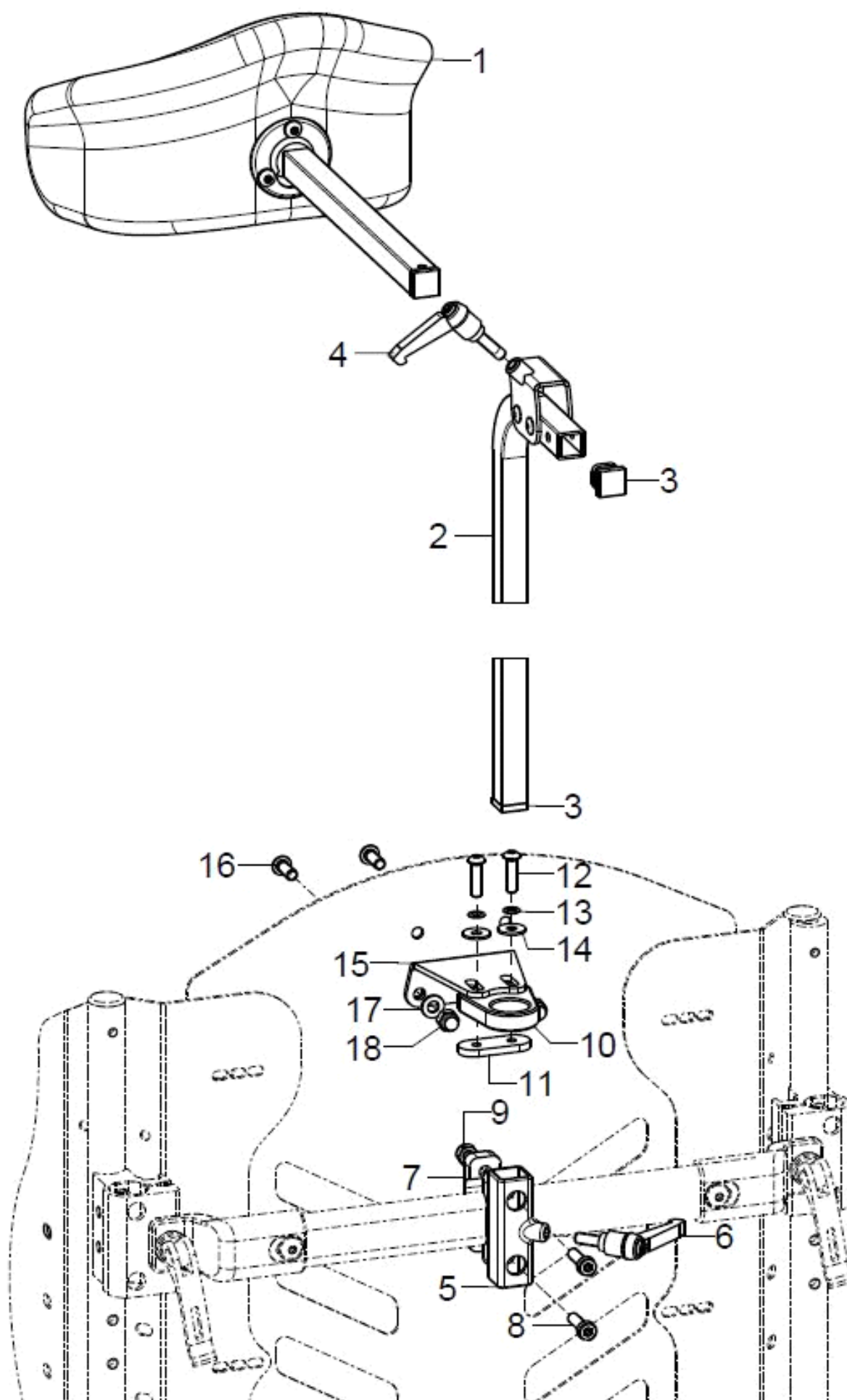

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-13	1	686021			
altern.	1	686023			
altern.	1	686024			
1-5	1	686031	head rest cushion model 1, 213x145 mm shell form complete		68.60.15.99.00.10
altern.	1	686033	head rest model 3, 225x100 curved complete		68.60.16.99.00.10
altern.	1	686034	head rest model 4, 300x150 curved complete		68.60.19.99.00.10
o.B.	1	68605001			
2	1				04.56.01.22.00.10
3	2		plugs for square tubes, black 15x15		04.63.11.15.00.10
4	1	403110	clamp lever adjustable M6x20, black		04.13.10.60.20.50
5-9	1	68600810			68.60.01.01.00.00
6	1	45320303			04.13.10.60.16.50
7	1				68.63.01.03.00.00
8	2				01.24.08.53.05.00
9	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
10-18	1	68600806			
10	1				68.60.08.06.00.80
11	1				19.02.05.01.00.24
12	2				01.42.07.64.65.00
13	2		lock washer S5		02.99.10.50.01.10
14	2		washer DIN 125 - A 5,3		02.61.05.31.01.20
15	1				68.60.08.07.00.80
16	2		lens head screw with hexagon socket ISO 7380 - M6x16		01.42.08.52.65.00
17	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
18	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20

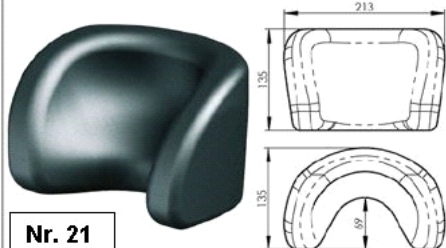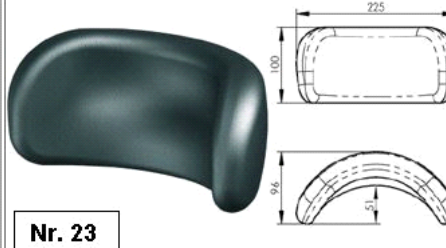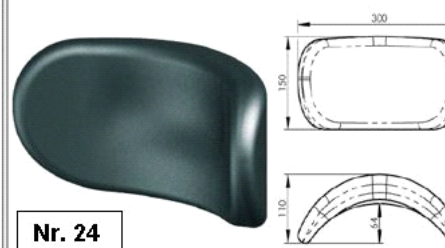
Modell 1	Modell 3	Modell 4
 Nr. 21	 Nr. 23	 Nr. 24
muschelförmig	bogenförmig - klein	bogenförmig - groß

Side truss pads STANDARD

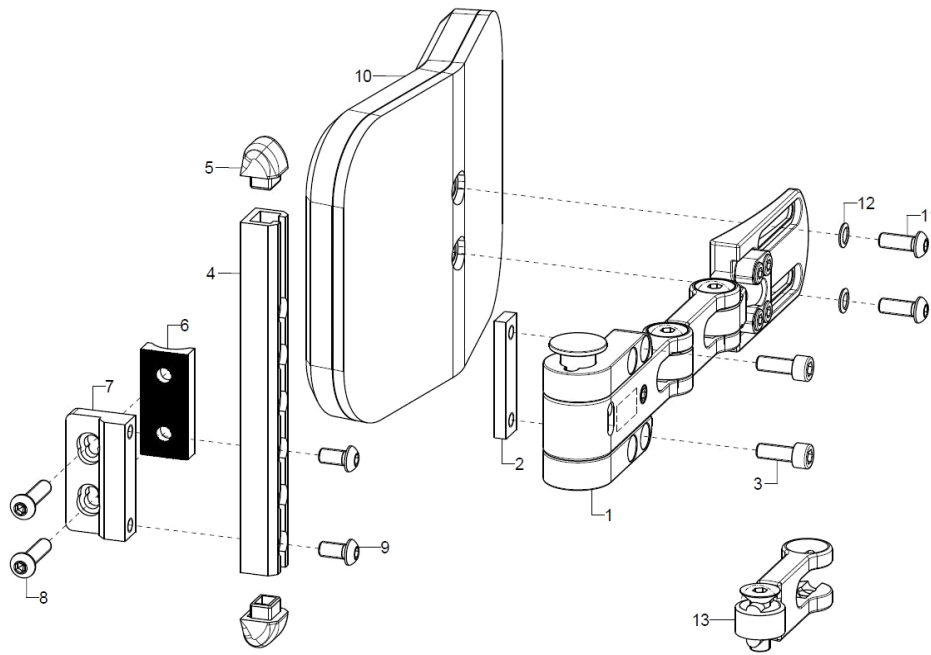


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	68590202			68.59.02.00.02.00
1	1	68590201			68.59.02.00.01.00
2-9	1	68590111			68.59.01.11.90.00
2-5+9	1	68590112			XX.XX.XX.XX.XX.XX
2	1				68.59.02.12.00.00
3	2		cylinder screw DIN 912 M6x16		01.22.08.52.62.40
4-5+9	2	685811			68.59.01.20.00.00
5	2				68.59.01.20.01.00
6	1				68.57.01.05.00.00
7	1				68.57.01.03.00.00
8	2		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
o.B.	2		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
9	2		lens head screw with hexagon socket ISO 7380 - M6x16		01.42.08.52.65.00
o.B.	2		counter sunk screw with hexagon socket DIN 7991 M5x16		01.65.07.92.05.00
10-12	1	68590106			68.59.01.26.00.00
o.B.	1	68590204			68.59.01.23.00.00
o.B.	1	68590205			68.59.01.24.00.00
11	2		lens head screw with hexagon socket ISO 7380 - M6x16		01.42.08.52.65.00
12	2		lock washer S6		02.99.10.60.01.10
20	1	68590203			68.59.02.15.00.00

Side truss pads ERGO

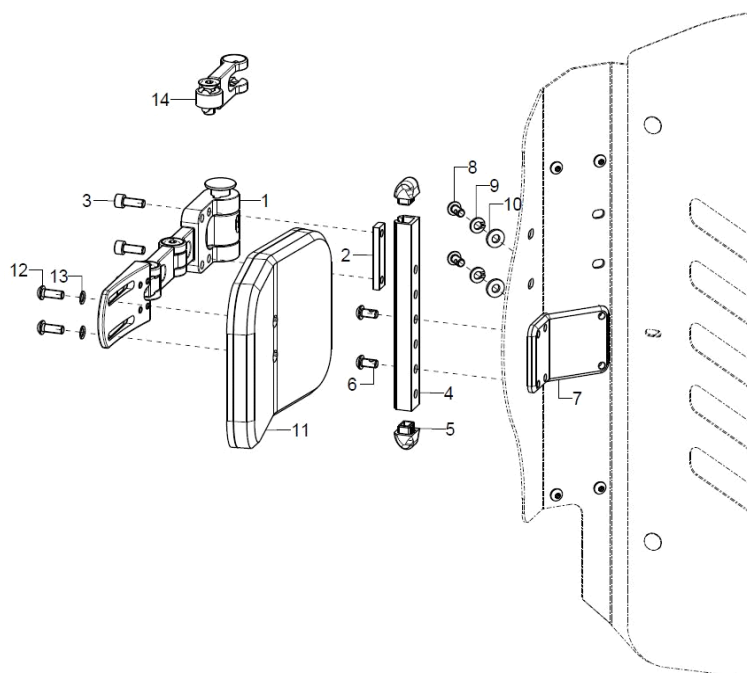


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	68590202			68.59.02.00.02.00
o.B.	1	68590201			68.59.02.00.01.00
2-10	1	68590118			
2	1				68.59.02.12.00.00
3	2		cylinder screw DIN 912 M6x16		01.22.08.52.62.40
4	1				68.59.01.20.00.00
5	2				68.59.01.20.01.00
6	2				01.42.08.49.65.01
7	1				68.59.04.01.00.80
8	2				01.42.08.49.65.01
9	2		spring ring DIN 127- A6		02.85.30.60.05.10
10	2		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
11-13	1	68590106			68.59.01.26.00.00
o.B.	1	68590204			68.59.01.23.00.00
o.B.	1	68590205			68.59.01.24.00.00
12	2		lens head screw with hexagon socket ISO 7380 - M6x16		01.42.08.52.65.00
13	2		lock washer S6		02.99.10.60.01.10
14	1	68590203			68.59.02.15.00.00

Side guard FIX for ERGO back

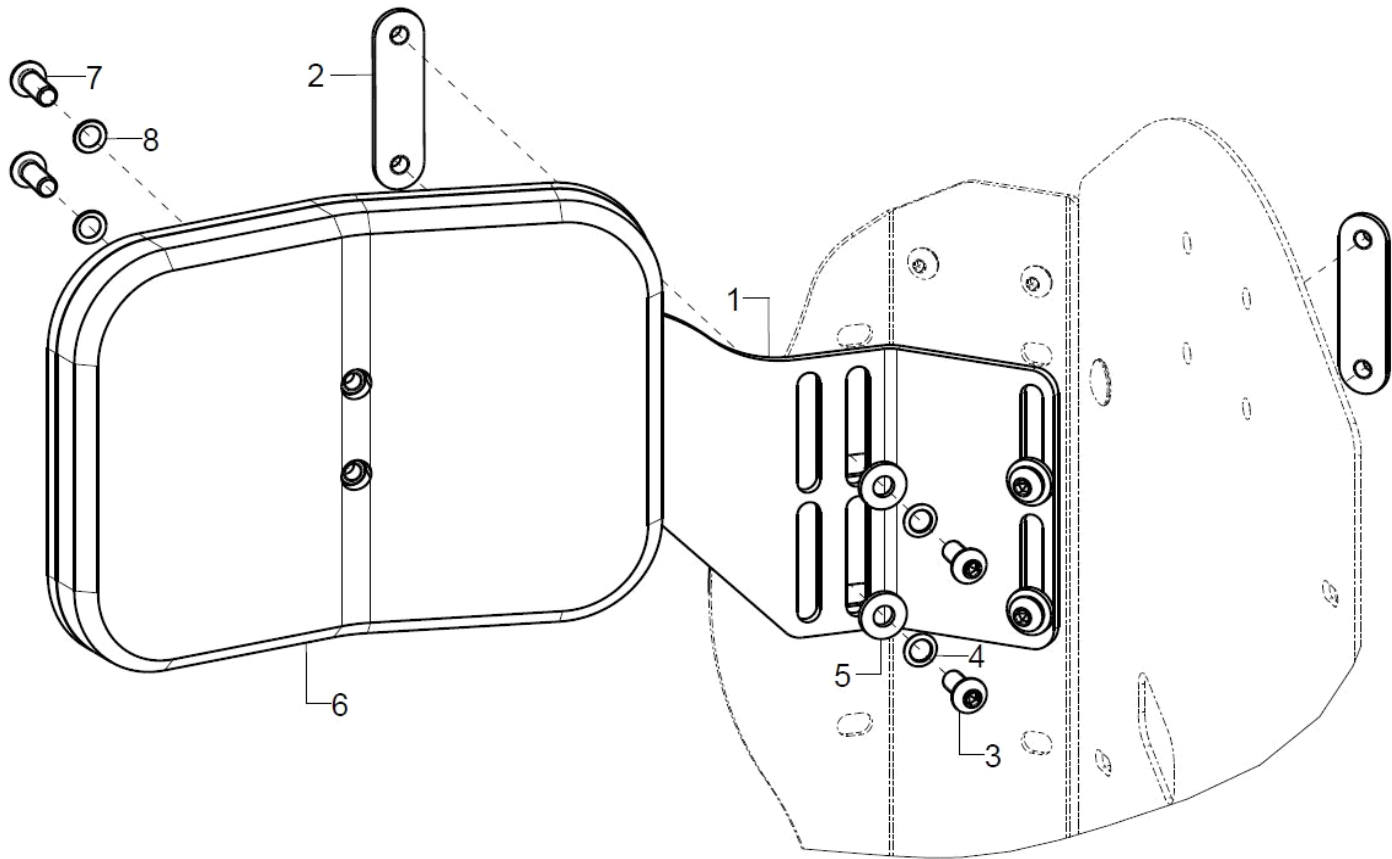


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5	1	68591002			68.59.04.03.11.80
o.B.	1	68591001			68.59.04.03.12.80
altern.	1	68591004			68.59.04.03.22.80
o.B.	1	68591003			68.59.04.03.21.80
2	2				68.59.04.04.00.80
3	4				01.42.08.50.65.00
4	4		lock washer S6		02.99.10.60.01.10
5	4		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
6-8	1	68590104			68.59.01.21.00.00
o.B.	1	68590105			68.59.01.22.00.00
o.B.	1	68590106			68.59.01.26.00.00
o.B.	1	68590204			68.59.01.23.00.00
o.B.	1	68590205			68.59.01.24.00.00
7	2		lens head screw with hexagon socket ISO 7380 - M5x16		01.42.07.92.65.00
o.B.	2		lens head screw with hexagon socket ISO 7380 - M6x16		01.42.08.52.65.00
8	2		lock washer S5		02.99.10.50.01.10
o.B.	2		lock washer S6		02.99.10.60.01.10

Connection for back shell

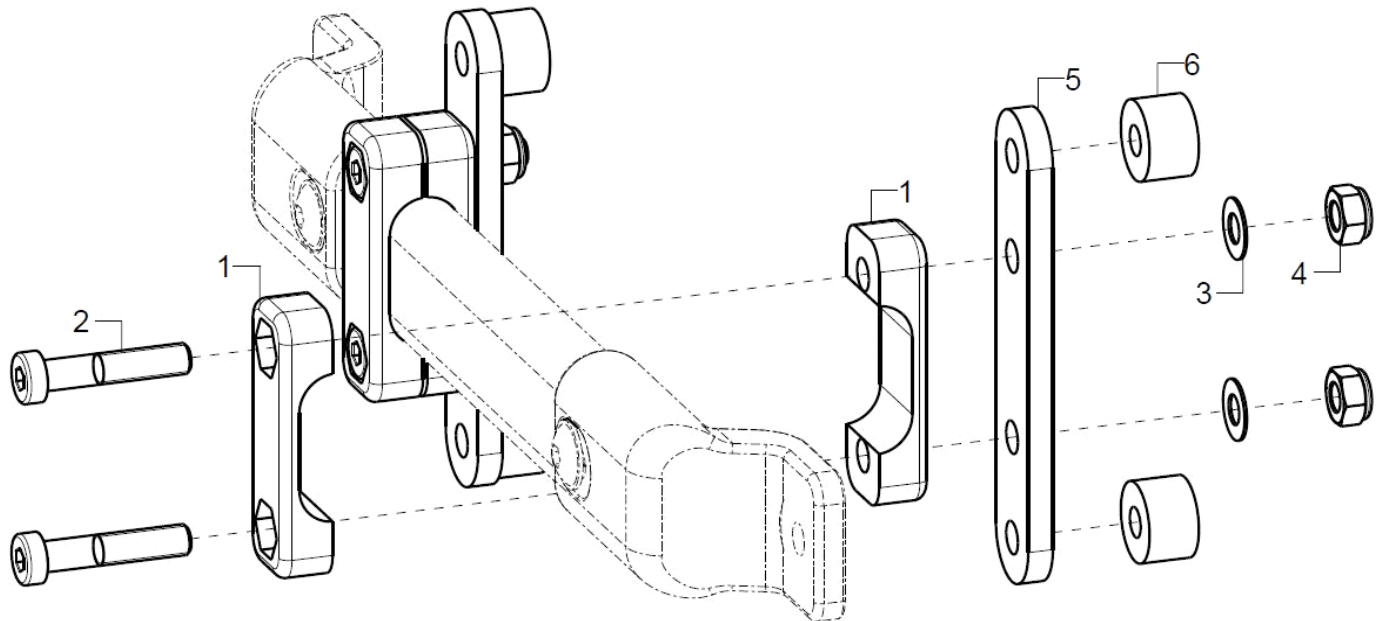


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-6	1/2				
1	2				68.63.01.03.00.00
2	2				01.24.08.58.05.00
3	2		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
4	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
5	1				69.80.04.01.00.80
6	2	35310209			45.31.01.03.00.72

Side guards Standard

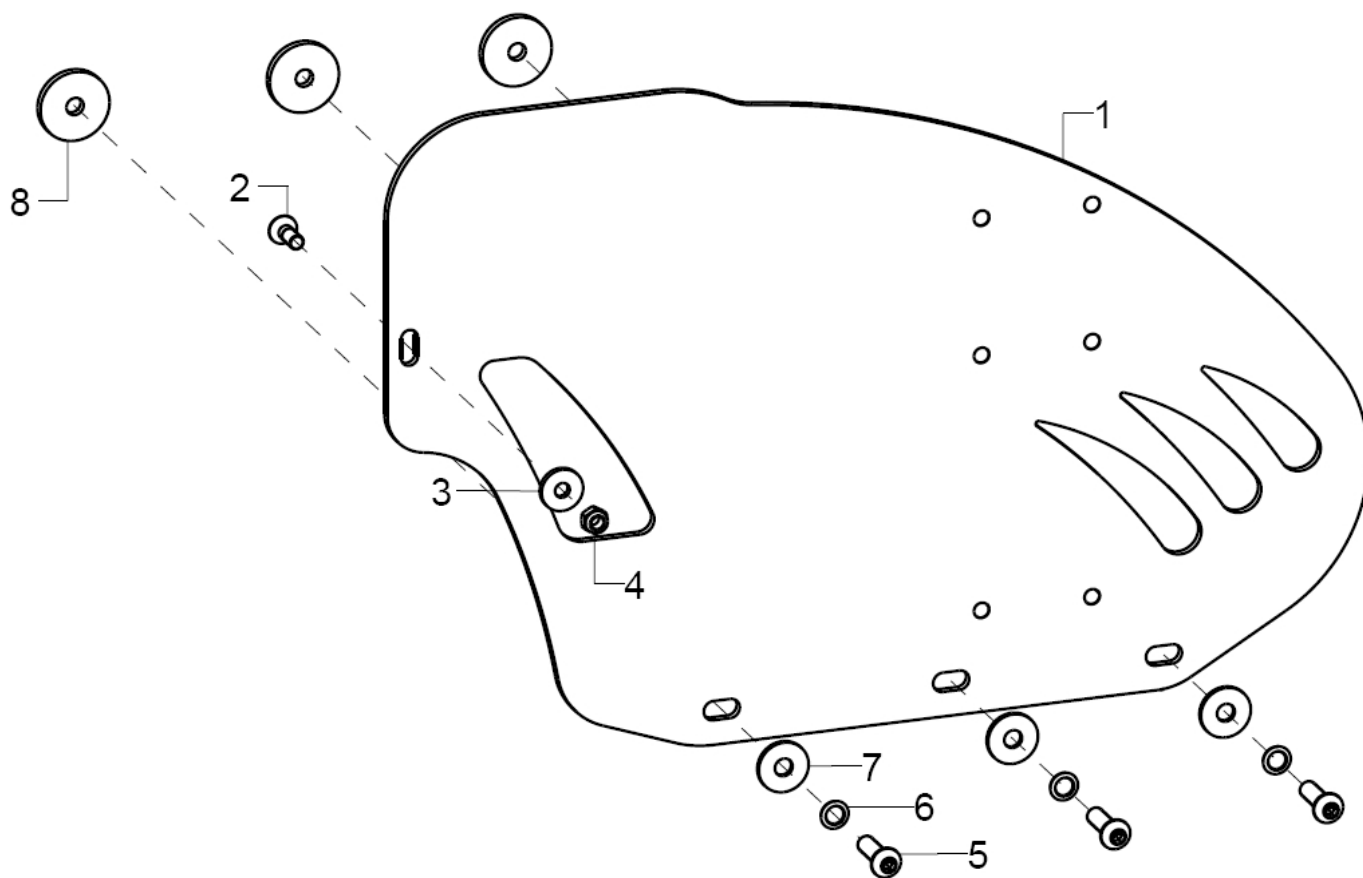


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-8	1	45270102			45.27.01.00.02.80
o.B.	1	45270101			45.27.01.00.01.80
altern.	1	45270202			45.27.02.00.02.80
o.B.	1	45270201			45.27.02.00.01.80
altern.	1	45270302			45.27.03.00.02.80
o.B.	1	45270301			45.27.03.00.01.80
altern.	1	45270402			45.27.04.00.02.80
o.B.	1	45270401			45.27.04.00.01.80
altern.	1	45270502			45.27.05.00.02.80
o.B.	1	45270501			45.27.05.00.01.80
2	1		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
3	1		washer DIN 7349 - 5,3		02.77.05.22.00.20
4	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
5	3		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
6	3		lock washer S6		02.99.10.60.01.10
7	3		washer DIN 9021 -6,4		02.79.06.41.60.20
8	3		washer D 25/d=6,2x3		28.00.21.01.00.10

Side guards with guidance

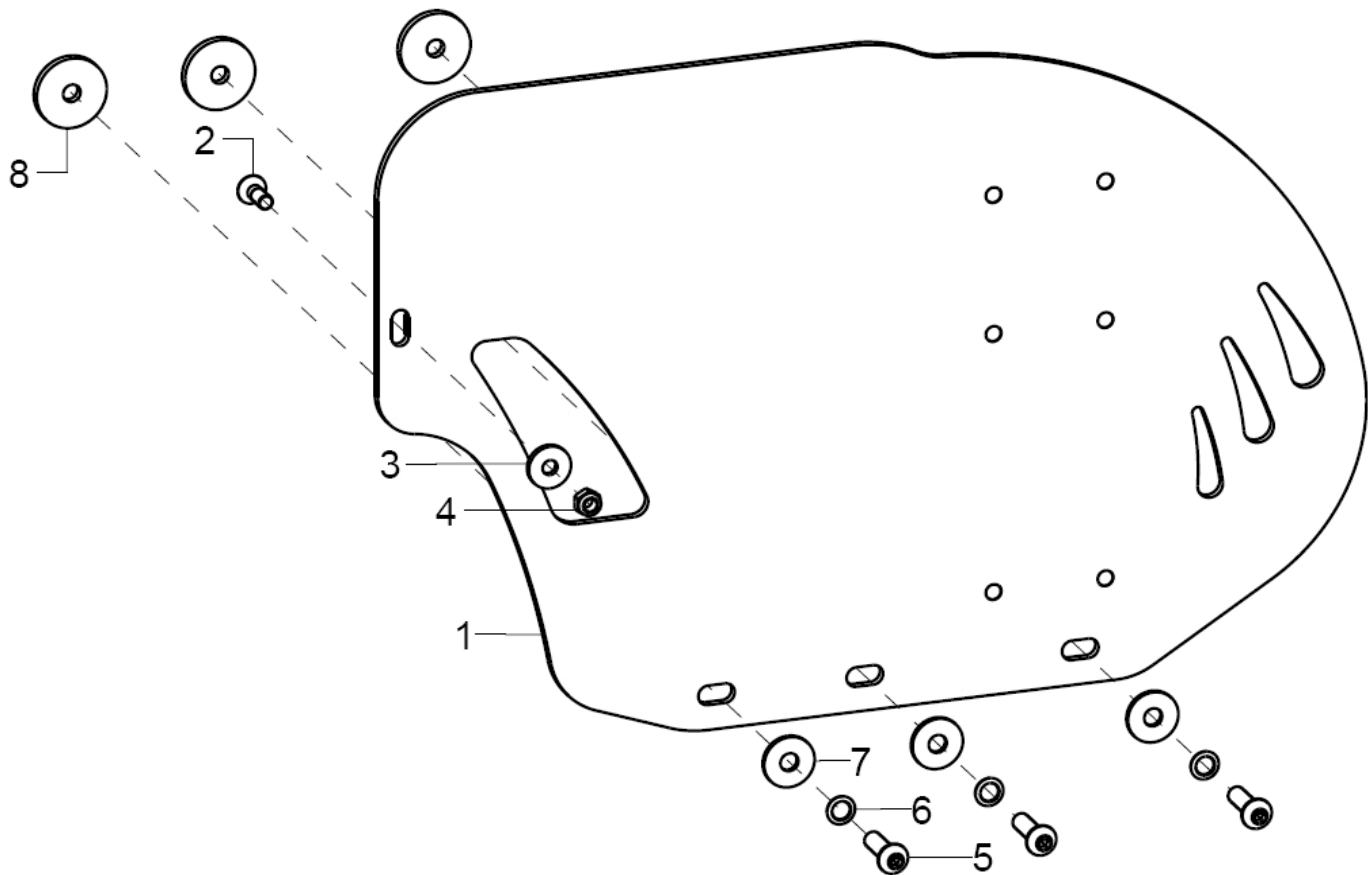

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-8	1	45280102			45.28.01.00.02.80
	1	45280101			45.28.01.00.01.80
altern.	1	45280202			45.28.02.00.02.80
o.B.	1	45280201			45.28.02.00.01.80
altern.	1	45280302			45.28.03.00.02.80
	1	45280301			45.28.03.00.01.80
altern.	1	45280402			45.28.04.00.02.80
o.B.		45280401			45.28.04.00.01.80
altern.	1	45280502			45.28.05.00.02.80
o.B.	1	45280501			45.28.05.00.01.80
2	1		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
3	1		washer DIN 7349 - 5,3		02.77.05.22.00.20
4	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
5	3		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
6	3		lock washer S6		02.99.10.60.01.10
7	3		washer DIN 9021 -6,4		02.79.06.41.60.20
8	3		washer D 25/d=6,2x3		28.00.21.01.00.10

Side guards ERGO

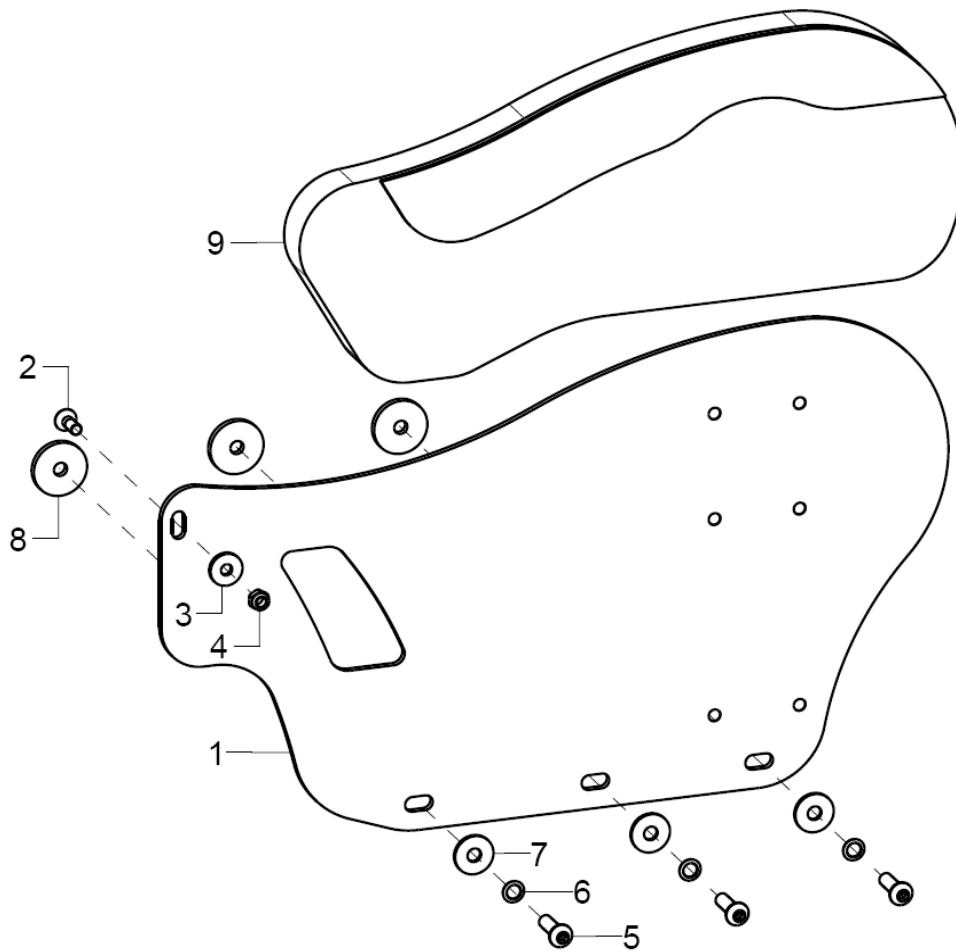


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-8	1	45290102			45.29.01.00.02.80
o.B.	1	45290101			45.29.01.00.01.80
altern.	1	45290202			45.29.02.00.02.80
o.B.	1	45290201			45.29.02.00.01.80
altern.	1	45290302			45.29.03.00.02.80
o.B.	1	45290301			45.29.03.00.01.80
altern.	1	45290402			45.29.04.00.02.80
o.B.	1	45290401			45.29.04.00.01.80
altern.	1	45290502			45.29.05.00.02.80
o.B.	1	45290501			45.29.05.00.01.80
2	1		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
3	1		washer DIN 7349 - 5,3		02.77.05.22.00.20
4	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
5	3		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
6	3		lock washer S6		02.99.10.60.01.10
7	3		washer DIN 9021 -6,4		02.79.06.41.60.20
8	3		washer D 25/d=6,2x3		28.00.21.01.00.10
9	1	45290602			45.29.05.0X.00.80
o.B.	1	45290601			45.29.05.0X.00.80
o.B.	1	45290603			
o.B.	1	45290604			

Stoffmuster	XX
Betex schwarz	20
Betex blau	21
Betex rot	22
Betex grau	23
Betex grün	24
Betex gelb	25
Betex orange	27

Alle Stoffmuster können unter www.sorgrollstuhltechnik.de angesehen werden.

Knee guidance for ERGO Side guards

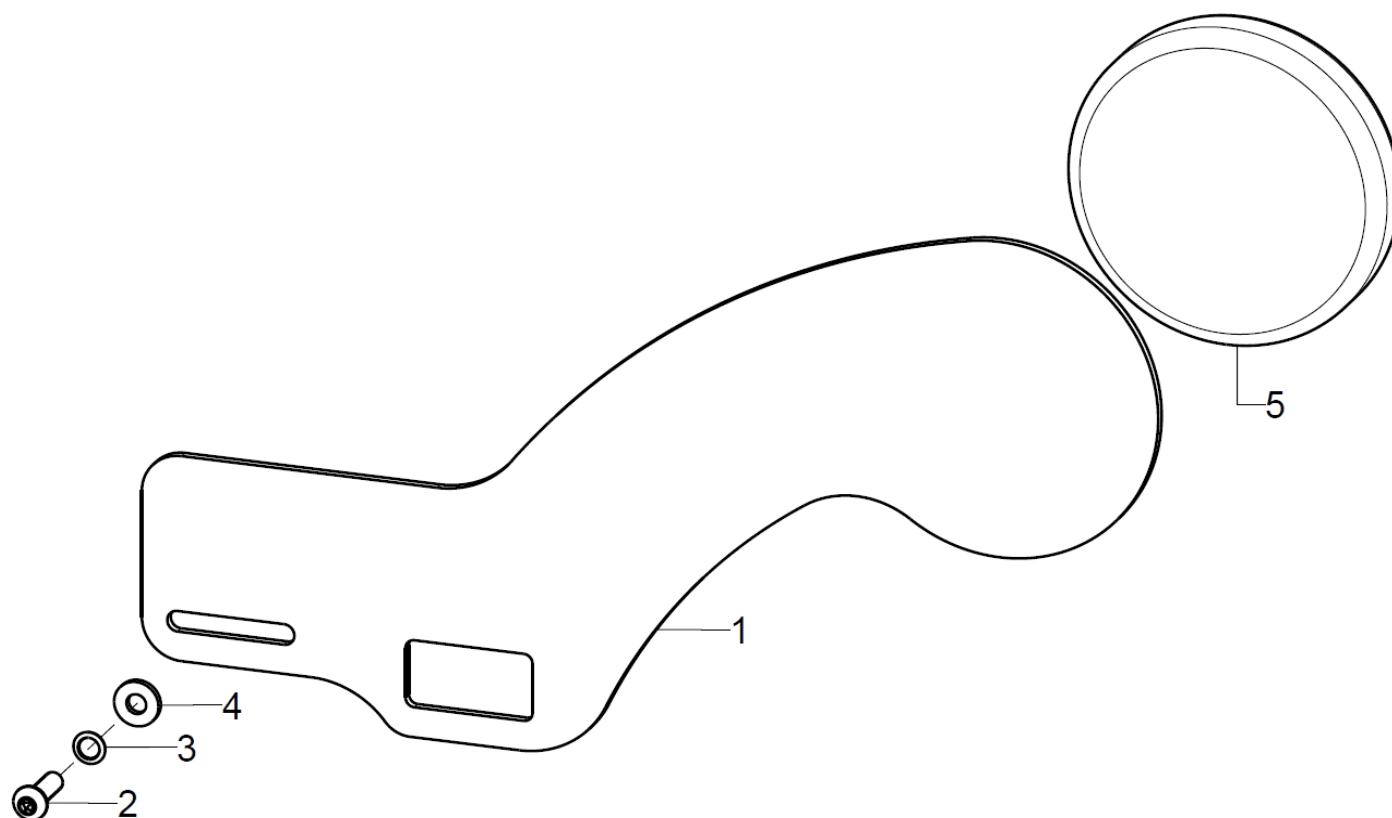


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-4	2	45290901			45.29.09.01.00.80
2	1		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
3	1		lock washer S6		02.99.10.60.01.10
4	1		washer reinforced $\varnothing 14/6,2 \times 2$		58.30.15.01.00.24
5	2	45290902			45.29.09.02.00.80

Stoffmuster	XX
Betex schwarz	20
Betex blau	21
Betex rot	22
Betex grau	23
Betex grün	24
Betex gelb	25
Betex orange	27


Alle Stoffmuster können unter www.sorgrollstuhltechnik.de angesehen werden.

Wheel cover Large

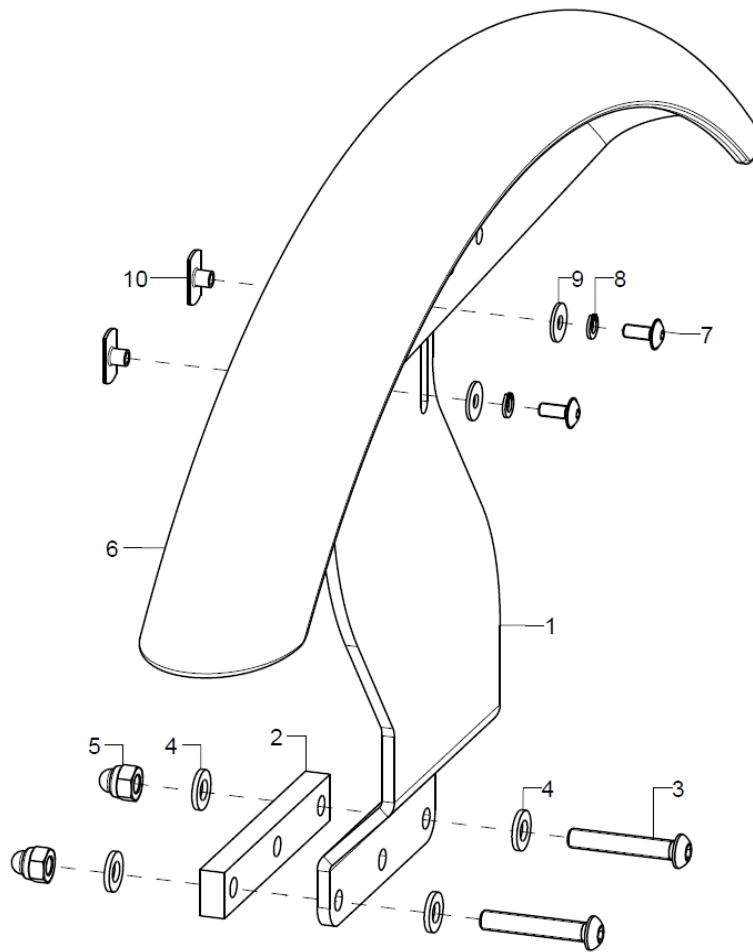


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5	1P	521002			52.10.01.01.00.76
altern.	1P	521004			52.10.01.02.00.76
2	1	52100202			52.10.01.06.00.76
3	2		lens head screw with hexagon socket ISO 7380 - M6x35		01.42.08.59.65.00
4	4		washer DIN 125 - A 6,4		02.61.06.41.41.20
5	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
6-10	1	5210			52.10.01.10.00.76
7	2		lens head screw with hexagon socket M4x10		01.43.07.28.65.00
8	2		spring ring DIN 127 - A 4		02.85.04.02.51.20
9	2		washer DIN 9021 - M4		02.79.04.31.00.20
10	2		weld nut C- M4x6		02.49.32.02.00.10

Wheel cover small

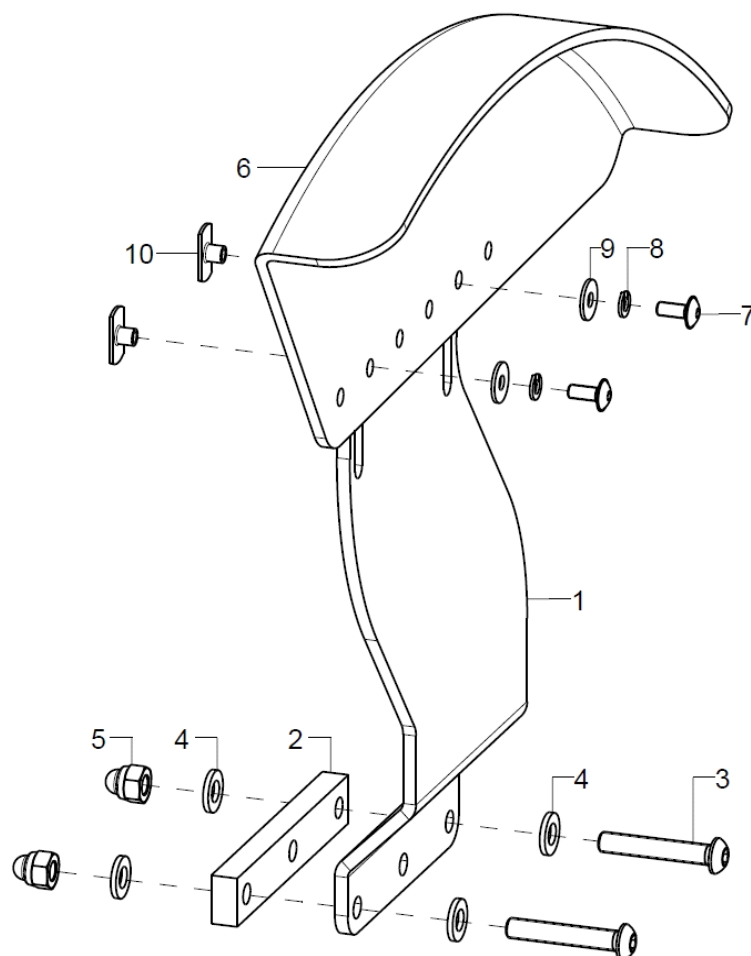


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5	1P	521002			52.10.01.01.00.76
altern.	1P	521004			52.10.01.02.00.76
2	1	52100202			52.10.01.06.00.76
3	2		lens head screw with hexagon socket ISO 7380 - M6x35		01.42.08.59.65.00
4	4		washer DIN 125 - A 6,4		02.61.06.41.41.20
5	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
6-10	1	52110102			52.11.01.01.02.80
o.B.	1	52110101			52.11.01.01.01.80
7	2		lens head screw with hexagon socket M4x10		01.43.07.28.65.00
8	2		spring ring DIN 127 - A 4		02.85.04.02.51.20
9	2		washer DIN 9021 - M4		02.79.04.31.00.20
10	2		weld nut C- M4x6		02.49.32.02.00.10

Wheel cover 16 inch

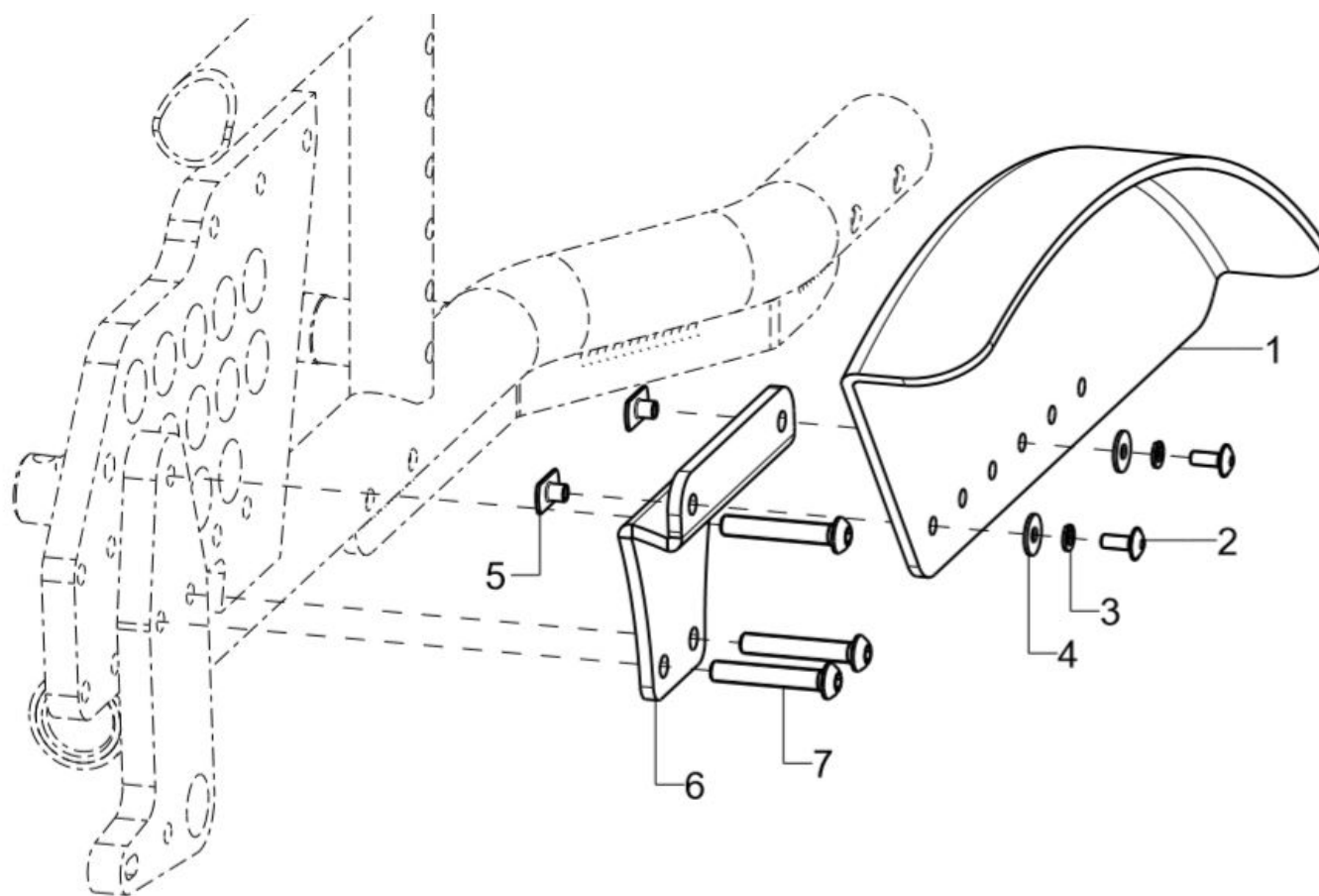


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5	1	52110102			52.11.01.01.02.80
o.B.	1	52110101			52.11.01.01.01.80
2	2		lens head screw with hexagon socket M4x10		01.43.07.28.65.00
3	2		spring ring DIN 127 - A 4		02.85.04.02.51.20
4	2		washer DIN 9021 - M4		02.79.04.31.00.20
5	2		weld nut C- M4x6		02.49.32.02.00.10
6-7	2	52100201			52.10.02.01.00.76
7	2				01.42.08.59.62.60

Arm rest 240x50mm up to SD32

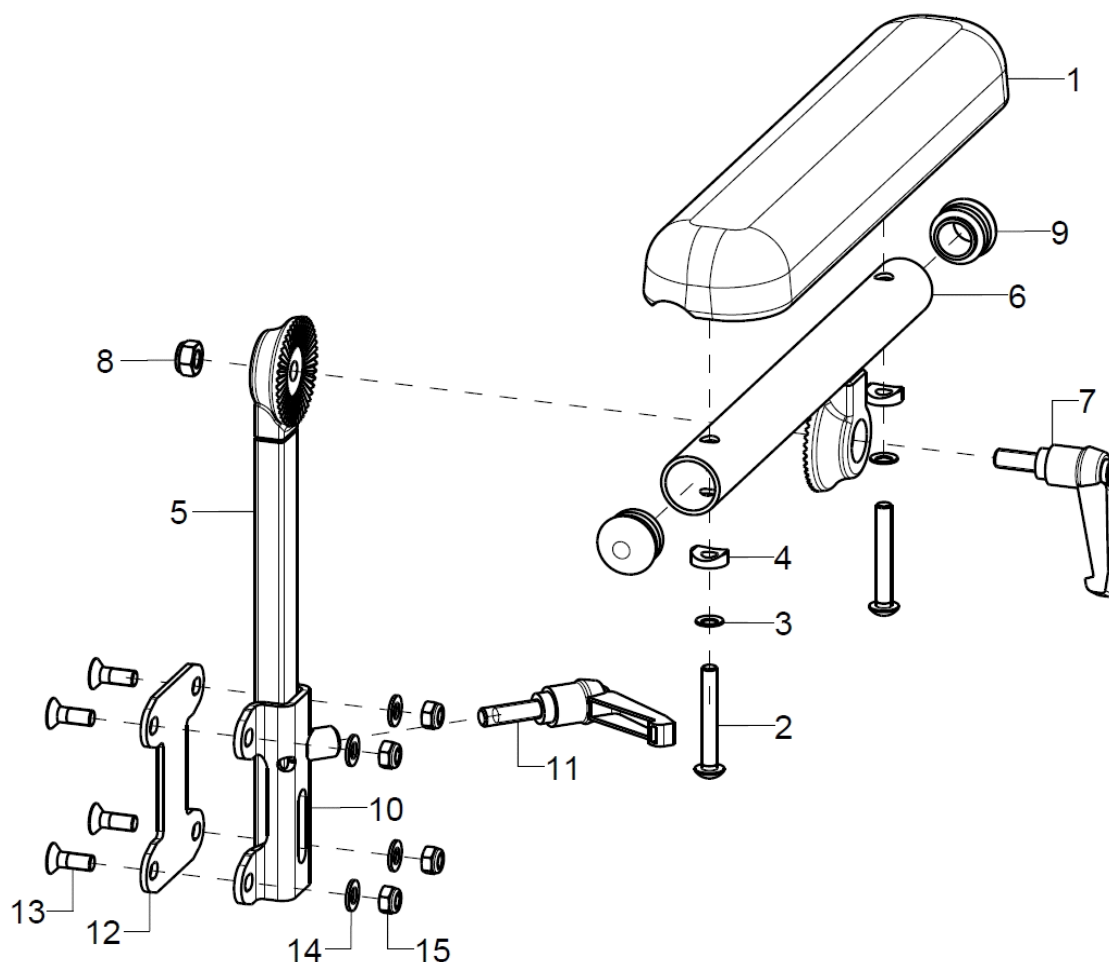


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-15	1	453202			
o.B.		453201			
1-4	1	450007	arm rest, straight black 240x50x26, 2xM5x164		04.65.01.24.00.10
2	2		lens head screw with hexagon socket ISO 7380 - M5x35		01.42.07.99.65.00
3	2		lock washer S5		02.99.10.50.01.10
4	2				40.04.16.01.00.10
5	1	45320201			45.15.18.02.00.80
6-9	1	45320302			45.15.18.01.02.80
o.B.	1	45320301			45.15.18.01.01.80
7	1		clamp lever adjustable M6x16, black		04.13.10.60.16.50
8	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
9	2		plugs for tube ø20x1-2, black		04.63.10.20.00.10
10-15	1	45320102			45.32.01.00.02.80
o.B.	1	45320101			45.32.01.00.01.80
11	1	403110	clamp lever adjustable M6x20, black		04.13.10.60.20.50
12	1				45.32.01.05.00.80
13	4		counter sunk screw with hexagon socket DIN 7991 M5x12		01.65.07.89.05.00
o.B.	4		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
14	4		washer DIN 125 - A 5,3		02.61.05.31.01.20
15	4		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20

Arm rest 240x50mm from SD34

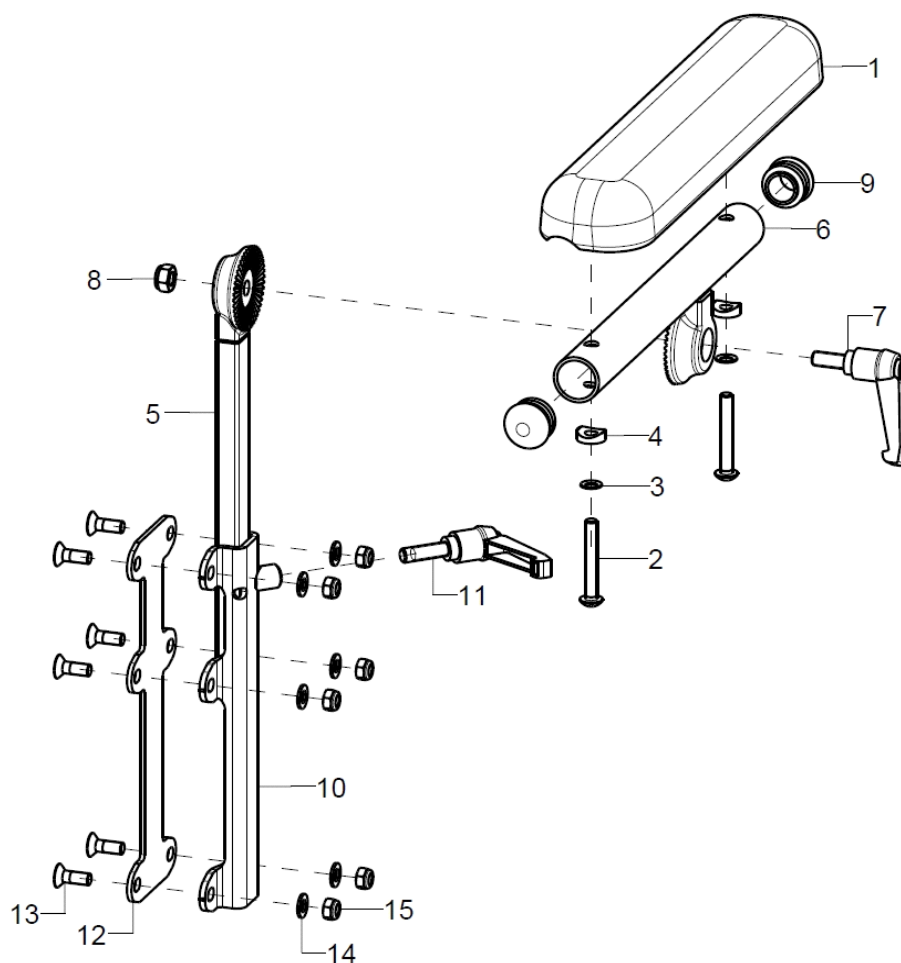


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-15	1	453204			
o.B.	1	453203			
1-4	1	450007	arm rest, straight black 240x50x26, 2xM5x164		04.65.01.24.00.10
2	2		lens head screw with hexagon socket ISO 7380 - M5x35		01.42.07.99.65.00
3	2		lock washer S5		02.99.10.50.01.10
4	2				40.04.16.01.00.10
5	1	45320201			45.15.18.02.00.80
6-9	1	45320302			45.15.18.01.02.80
o.B.	1	45320301			45.15.18.01.01.80
7	1		clamp lever adjustable M6x16, black		04.13.10.60.16.50
8	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
9	2		plugs for tube ø20x1-2, black		04.63.10.20.00.10
10-15	1	45320104			45.15.18.04.02.80
o.B.	1	45320103			45.15.18.04.01.80
11	1	403110	clamp lever adjustable M6x20, black		04.13.10.60.20.50
12	1				45.15.18.05.00.80
13	4		counter sunk screw with hexagon socket DIN 7991 M5x12		01.65.07.89.05.00
o.B.	6		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
14	6		washer DIN 125 - A 5,3		02.61.05.31.01.20
15	6		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20

Arm rest 260x80mm

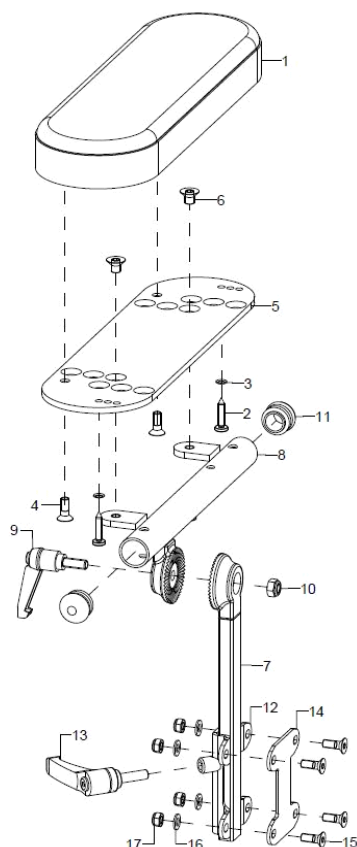

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-17	1	45330202			
o.B.	1	45330201			
1-4	1	450015			45.15.20.04.00.80
2	2				01.88.07.34.00.81
3	2		lock washer S4		02.99.10.40.01.10
4	2		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
5+6	1	45330205			45.15.20.03.00.80
6	2				01.65.08.47.02.40
7	1	45320201			45.15.18.02.00.80
8-11	1	45330204			45.15.20.01.02.80
o.B.	1	45330203			45.15.20.01.01.80
9	1		clamp lever adjustable M6x16, black		04.13.10.60.16.50
10	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
11	2		plugs for tube ø20x1-2, black		04.63.10.20.00.10
12-17	1	45320102			45.32.01.00.02.80
o.B.	1	45320101			45.32.01.00.01.80
o.B.	1	45320104			45.15.18.04.02.80
o.B.	1	45320103			45.15.18.04.01.80
13	1	403110	clamp lever adjustable M6x20, black		04.13.10.60.20.50
14	1				45.32.01.05.00.80
o.B.	1				45.15.18.05.00.80
15	4		counter sunk screw with hexagon socket DIN 7991 M5x12		01.65.07.89.05.00
o.B.	4		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
16	4		washer DIN 125 - A 5,3		02.61.05.31.01.20
17	4		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20

Caster wheel adapter

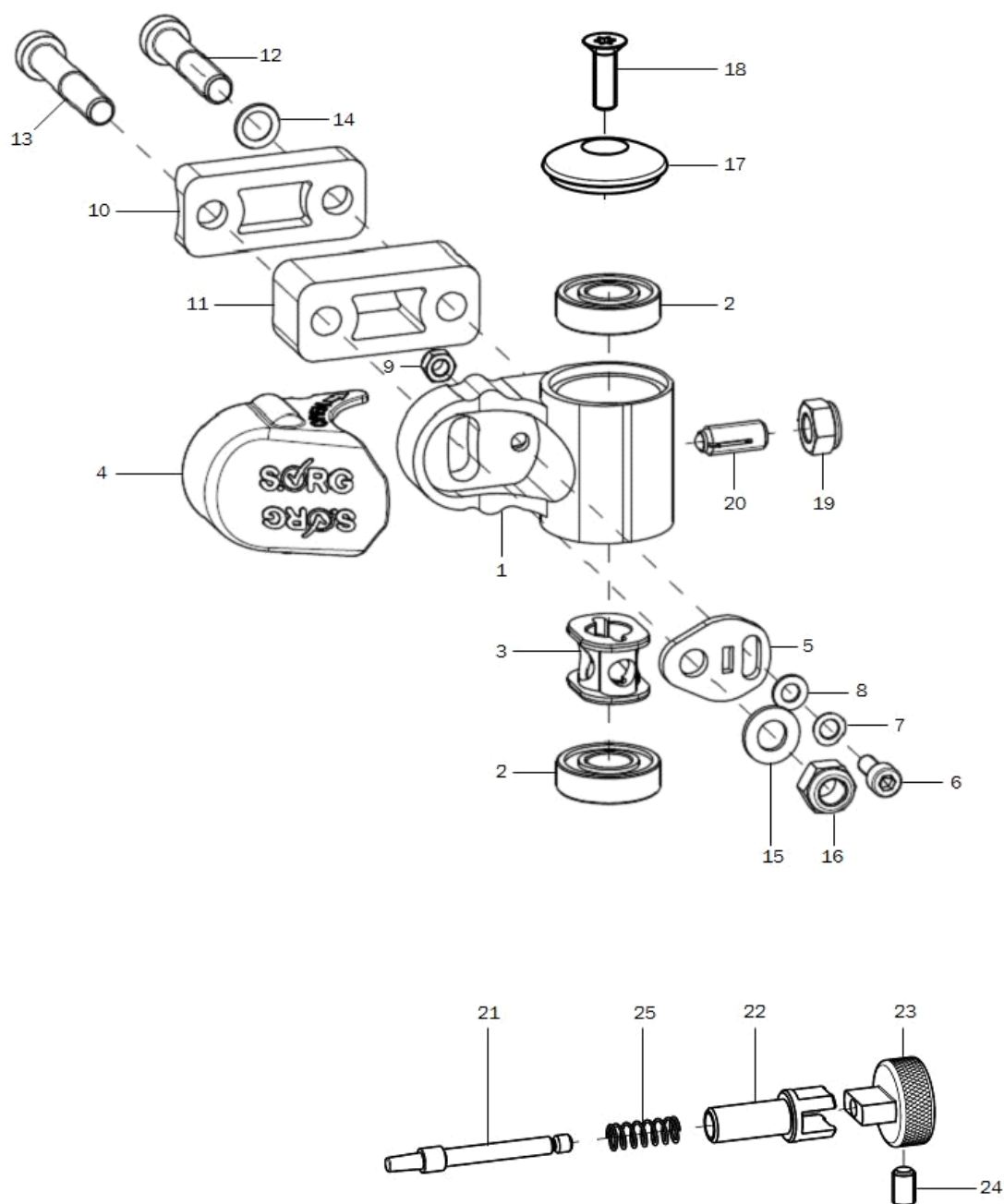


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-9	2	280506	caster adapter track fixation		XXXXXXXXXXXX
o.B.	2	280500	steering adapter aluminum complete		28.05.01.00.00.00
1	1	280507	caster adapter track fixation		28.05.01.01.00.00
altern.	1	280501	steering adapter aluminum		28.05.01.02.00.00
2	2	200011	groove ball bearing ø28/12 6001z		04.32.12.08.00.10
3	1	280504	damper element		28.05.01.04.00.00
4	1	280030	cover cap		28.00.10.03.00.10
5	1	280503	eccentric disc		28.05.01.03.00.00
6	1		cylinder screw DIN 912 M5x12		01.22.07.89.72.40
7	1		lock washer S5		02.99.10.50.01.10
8	1		washer DIN 125 - A 5,3		02.61.05.31.01.20
9	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
10+12-16	2	280520			XX.XX.XX.XX.XX.XX



10+12-16	2	280521		XX.XX.XX.XX.XX.XX
10+12-16	2	280522		XX.XX.XX.XX.XX.XX
10	1	280505		28.05.02.02.00.00
11	1	280513	track extention 20mm	28.05.02.03.00.00
altern.	1	280514	track extention 40mm	28.05.02.04.00.00
12	1		cylinder screw low head DIN 7984 - M8x45	01.24.09.81.62.40
altern.	1		cylinder screw low head DIN 7984 - M8x65	01.24.09.85.62.40
altern.	1		cylinder screw low head DIN 7984 - M8x85	01.24.09.89.62.40
13	1			01.24.09.82.62.40
altern.	1			01.24.09.86.62.40
altern.	1			01.24.09.90.62.40
14	1		lock washer S8	02.99.10.80.01.10
15	1		washer DIN 125 - A 8,4	02.61.08.41.41.20
16	1		hexagon nut self-locking low form DIN 985 - M8	02.32.00.80.01.20
17-18	1	282107	cap 35x10 black anodized incl. mounting screw	28.21.07.01.00.10
18	1		counter sunk screw with Torx ISO 14581 M6x20 vz, black	01.66.08.54.92.21
19	1		hexagon nut self-locking low form DIN 985 - M8	02.32.00.80.01.20
20	2	285103	spring pressure piece M8 complete	04.14.50.80.21.20
21-25	1	285003	track fixation complete	XXXXXXXXXXXX
21	1			28.50.01.01.00.10
22	1			28.50.01.02.00.10
23	1			28.50.01.03.00.10
24	1		setscrew with hexagon socket DIN 913,M5x8	01.67.07.87.61.20
25	1		pressure spring 0,55x5,5x23,7	04.21.05.52.37.10


Achtung: Wenn eine Lenk- und Schiebehilfe oder Rückhalteösen am Rollstuhl angebaut sind, bitte bei Bestellungen von Ersatzteilen angeben.

Adapter for lower seat heights

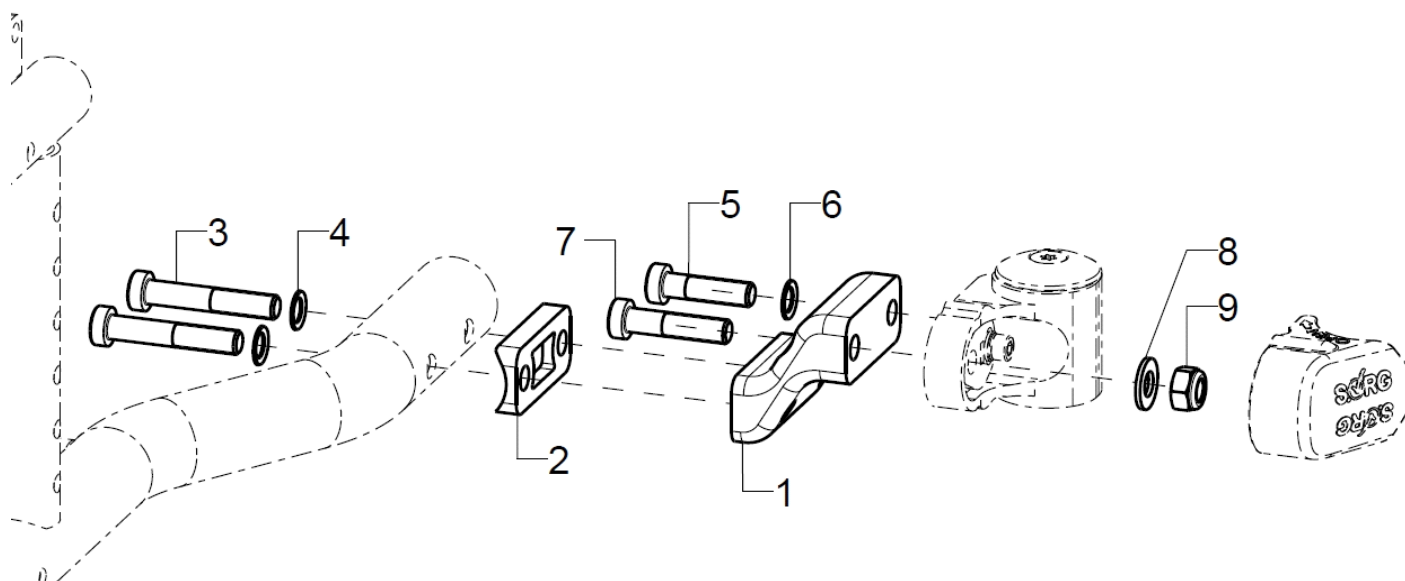

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	2	280525			28.05.03.01.00.00
2-8	2	280528			
2	1	280505			28.05.02.02.00.00
3	2		cylinder screw low head DIN 7984 - M8x45		01.24.09.81.62.40
4	2		lock washer S8		02.99.10.80.01.10
5	1				01.24.09.78.62.40
6	1		lock washer S8		02.99.10.80.01.10
7	1				01.24.09.79.62.40
8	1		washer DIN 125 - A 8,4		02.61.08.41.41.20
9	1		hexagon nut self-locking low form DIN 985 - M8		02.32.00.80.01.20

Caster wheel fork

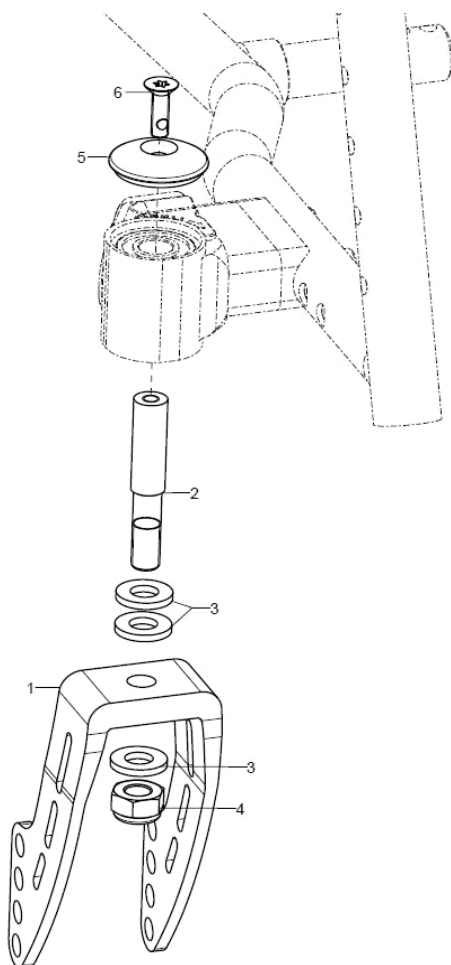


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	2	2841	caster fork aluminum 164x45		28.41.01.00.00.10
o.B.	2	2842	caster fork aluminum 126x45		28.42.01.00.00.10
o.B.	2	2845			28.00.02.03.00.00
2	2	282104	steering axle $\varnothing 12 \times 65$, 1		28.21.05.01.00.10
o.B.	2	285101	steering axle 65 for track fixation		28.50.00.03.00.10
o.B.	2	285001	steering axle 65 for locking		28.50.00.01.00.10
3+4	2	282120	mounting set caster axle on caster fork aluminum (washers, hexagon nut)		28.21.20.01.01.00
3	3		washer DIN 1440 -10x20x2,5		02.71.10.02.50.20
4	1		hexagon nut self-locking low form DIN 985 - M10		02.32.01.00.01.20
5-6	2	282107	cap 35x10 black anodized incl. mounting screw		28.21.07.01.00.10
6	2		counter sunk screw with Torx ISO 14581 M6x20 vz, black		01.66.08.54.92.21



Caster wheels


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	2	2805	caster 5" PU		04.52.42.05.00.30
o.B.	2	2809	caster 5" soft solid rubber		04.52.44.05.00.10
o.B.	2	2834	caster 5" PU, transparent with LED		04.52.45.05.00.40
o.B.	2	2836	caster 5" solid rubber black with aluminum rim		04.52.41.05.00.61
o.B.	2	2802	caster 5,5" solid rubber		
o.B.	2	2803	caster 6" PU		04.52.42.06.00.30
o.B.	2	2811	caster 6" solid rubber		04.52.41.06.00.30
o.B.	2	2838			04.52.41.06.00.70
o.B.	2	2808	caster 6" pneumatic wheels		04.52.43.06.00.11
o.B.	2	2813	caster 7" solid rubber		04.52.41.07.00.20
o.B.	2	2807	caster 7" pneumatic wheels		04.52.43.07.00.11
o.B.	2	289998			

Adaptation plate 12 inch

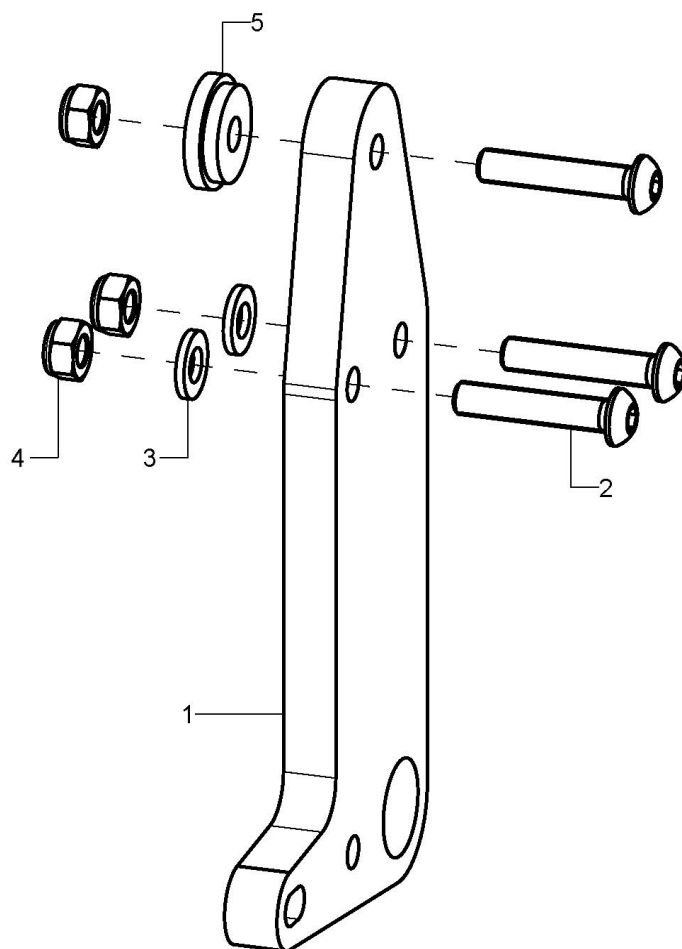


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5	2	20157610			20.15.76.10.01.76
o.B.	2	20157611			20.15.76.10.02.76
2	3		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
3	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
4	3		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
5	1				76.06.01.04.00.69

Adaptation plate 16 inch

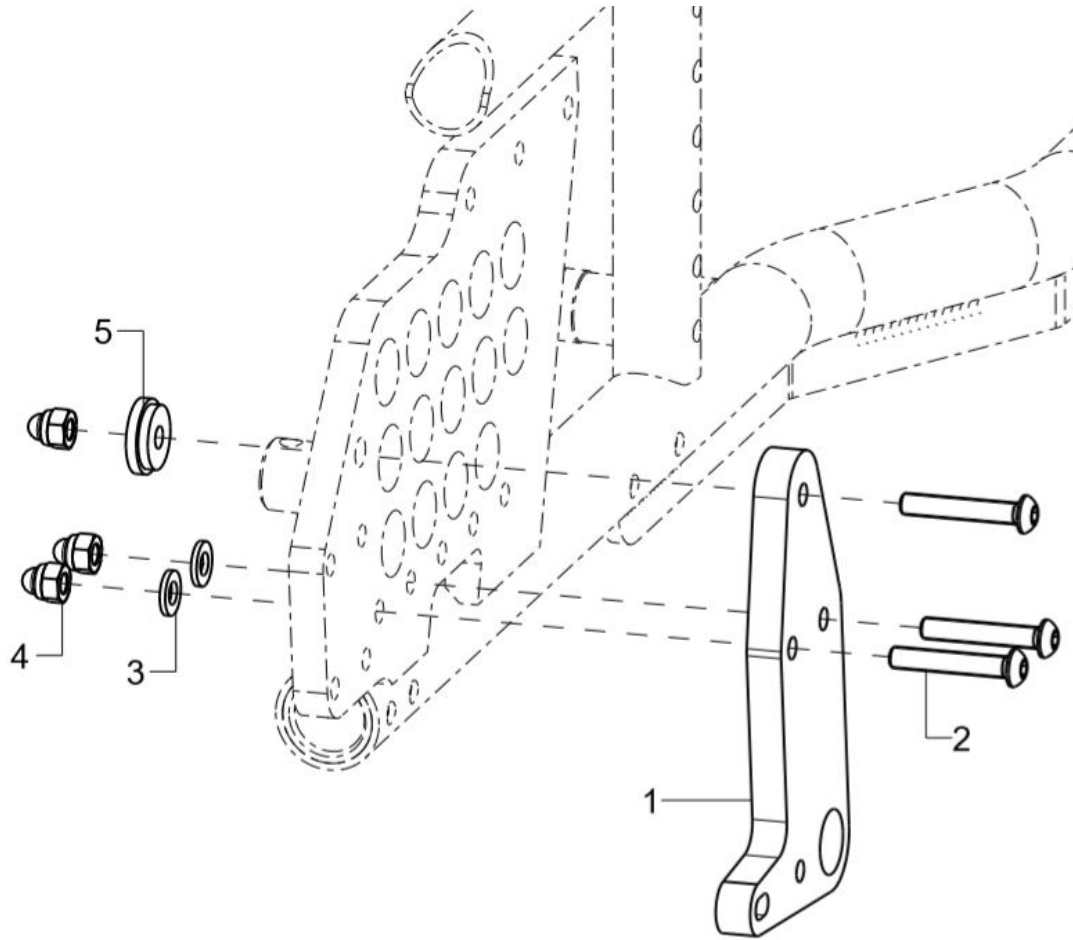


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5	2	20157616			20.15.76.16.00.76
2	3		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
o.B.	3				01.42.08.59.62.60
3	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
4	3		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
5	1				76.06.01.04.00.69

12 inch wheels

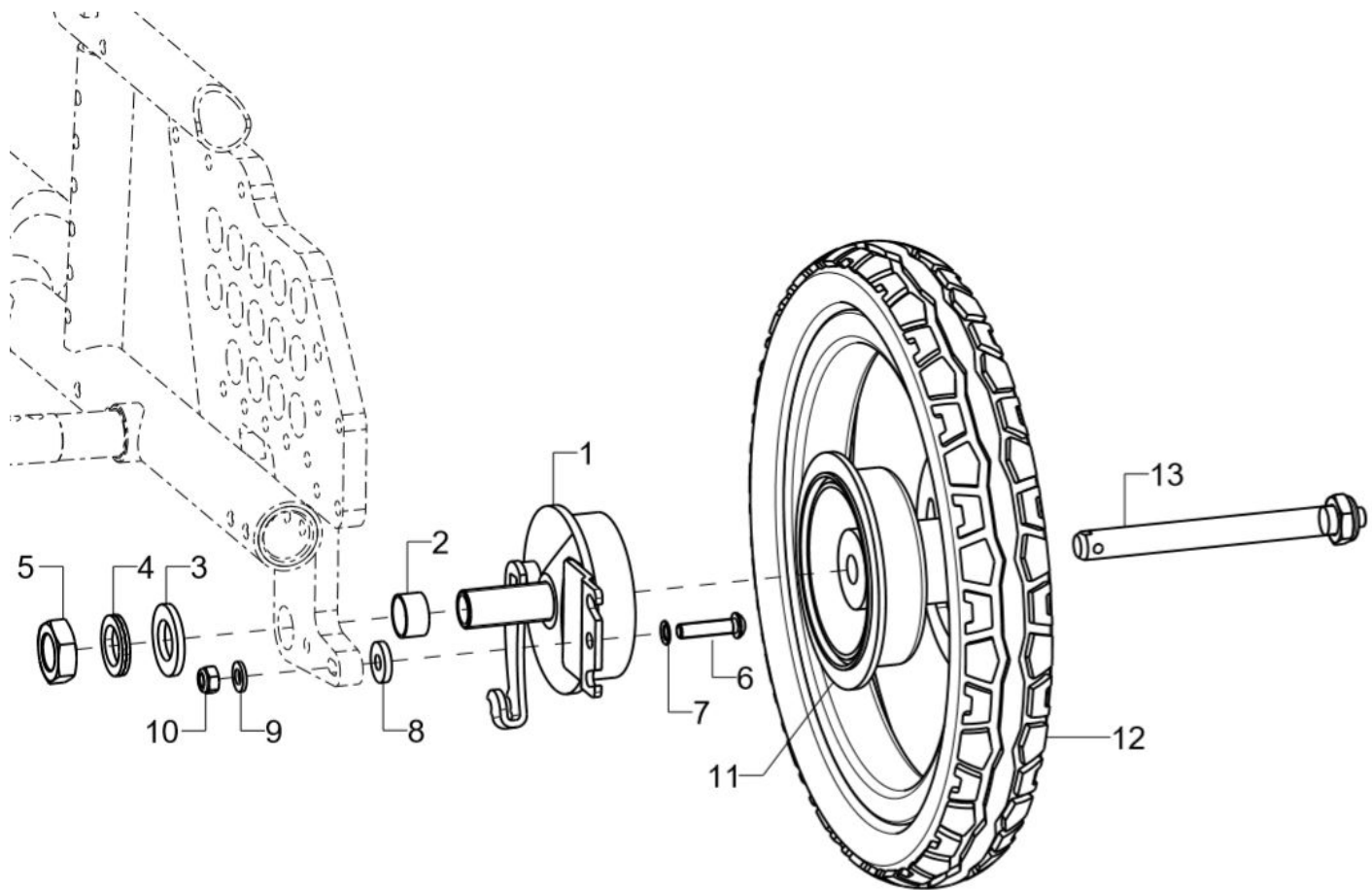


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-10	1	22002402			22.00.24.99.02.00
o.B.	1	22002401			22.00.24.99.01.00
2	1				22.00.12.03.00.69
3	1		washer DIN 125 - A 17		02.61.17.02.71.20
4	1		lock washer M16 (Nord-Lock)		02.99.21.70.01.10
5	1		nut M16x1 SW 24		02.19.01.60.11.21
6	1		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
7	1		lock washer S6		02.99.10.60.01.10
8	1		washer DIN 7349 - 6,4		02.77.06.53.00.20
9	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
10	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
11-12	2	22120003			22.12.00.99.01.00
o.B.	2	22124003			22.12.00.99.02.00
11	2	22120103			22.12.01.00.00.00
12	2	201241			04.61.11.20.70.20
o.B.	2	200512			XX.XX.XX.XX.XX.XX
o.B.	2	201240			04.61.21.20.70.20
13	2	210003	quick-release axle 12,7x103 mm for standard hub from 5/2013 and light weight wheel		04.51.51.21.00.31

16 inch wheels

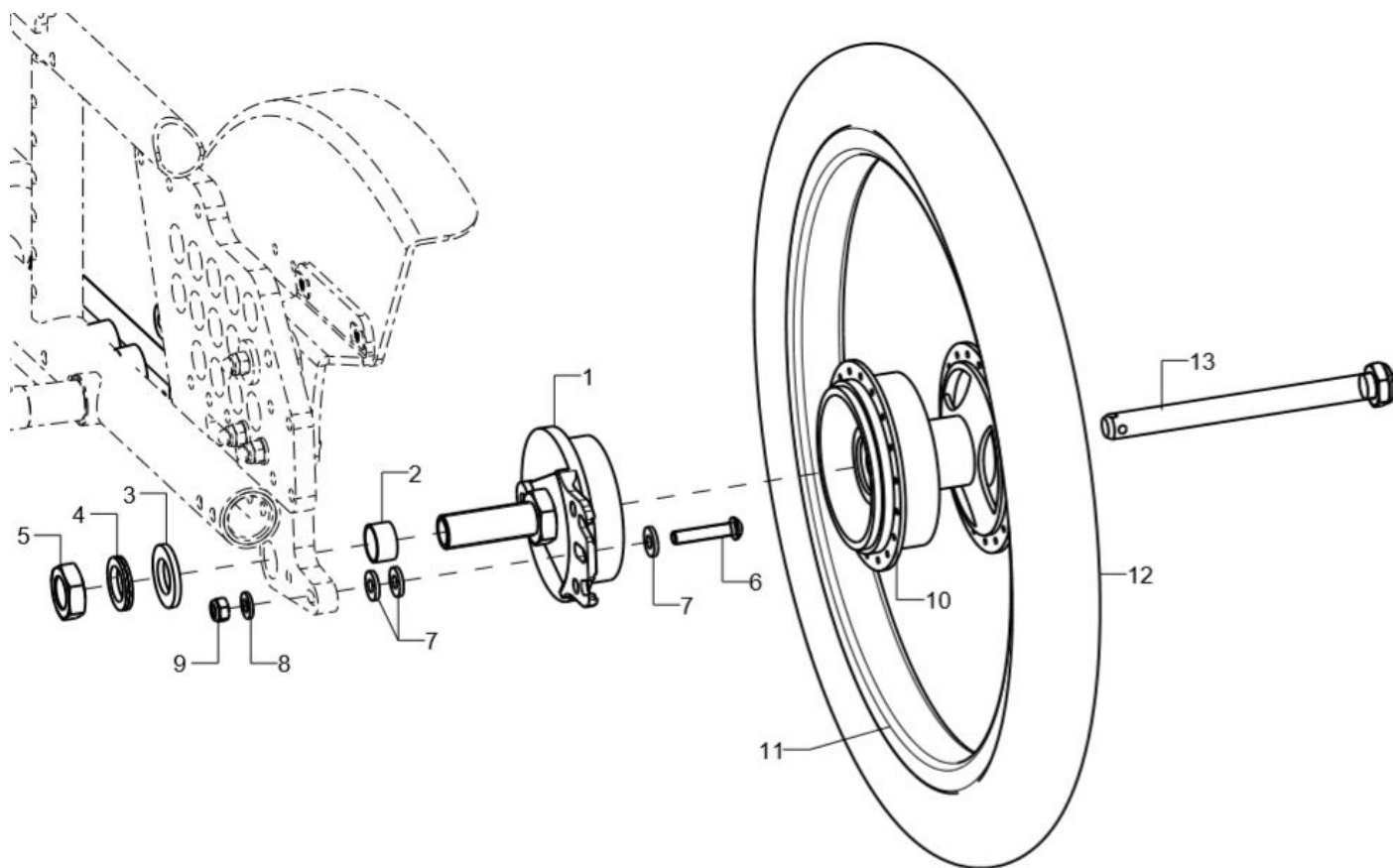


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-9	1	220016	drum brake anchor, black coated		04.51.67.00.01.10
2	1				22.00.12.03.00.69
3	1		washer DIN 125 - A 17		02.61.17.02.71.20
4	1		lock washer M16 (Nord-Lock)		02.99.21.70.01.10
5	1		nut M16x1 SW 24		02.19.01.60.11.21
6	1		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
7	3		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
8	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
9	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
10	1	22160003			
o.B.	1	22164003			
11		22160002			
12	1	201612			
o.B.	1	200516			
o.B.	1	231611			
13	1	220001	quick-release axle D12x123mm for drum brake		04.51.51.21.20.30

Wheel base extension

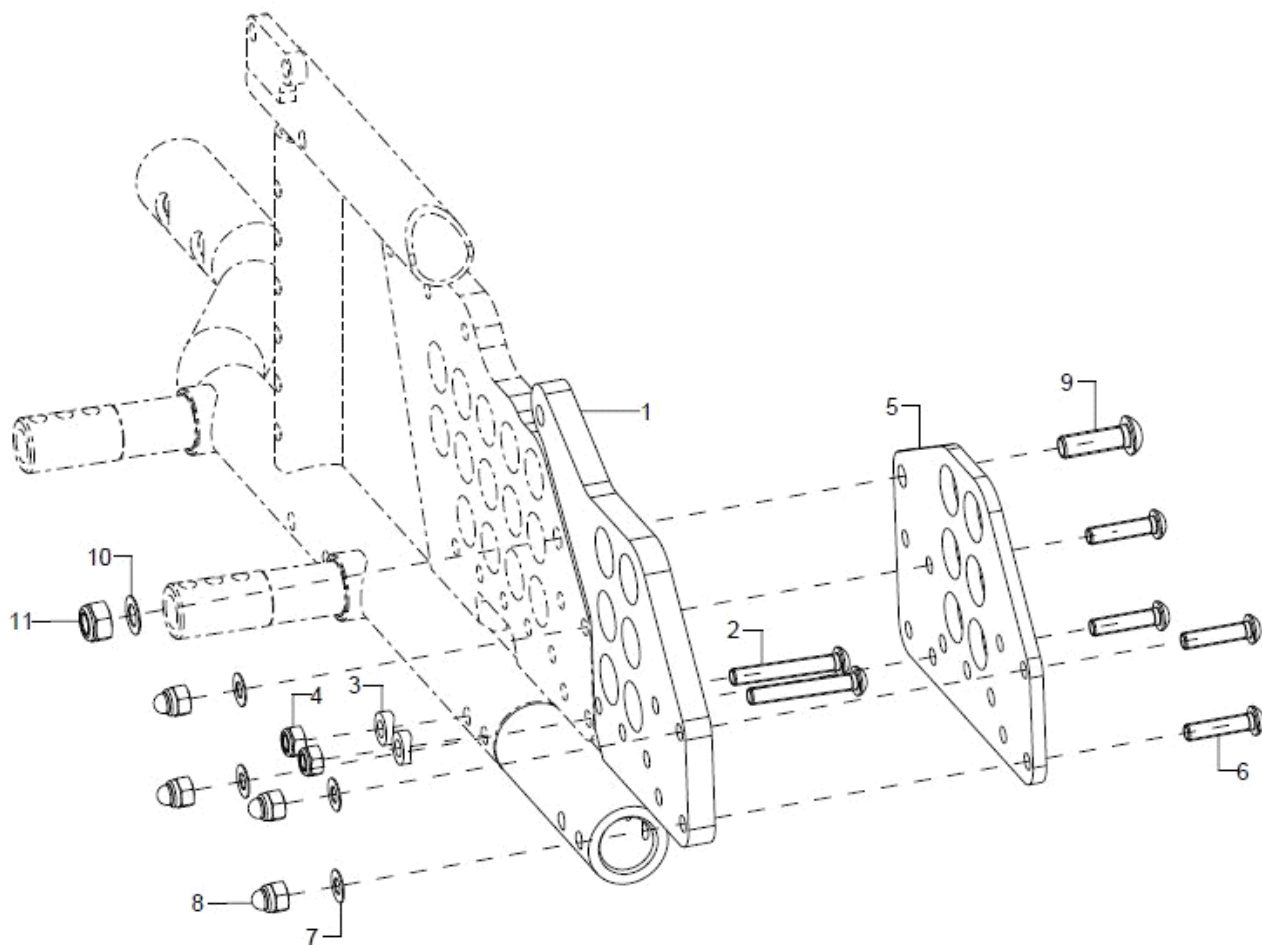


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-4	1	767602			76.76.02.01.02.76
o.B.	1	767601			76.76.02.01.01.76
2	2		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
3	2		saddle washer $\varnothing 12/6 \times 2,5$		35.04.16.01.00.10
4	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
5-11	2	767604			76.76.02.04.00.76
6	4		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
7	4		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
8	4		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
9	1				01.42.09.76.65.00
10	1				02.68.08.05.00.30
11	1		hexagon nut self-locking low form DIN 985 - M8		02.32.00.80.01.20

Screw adapter

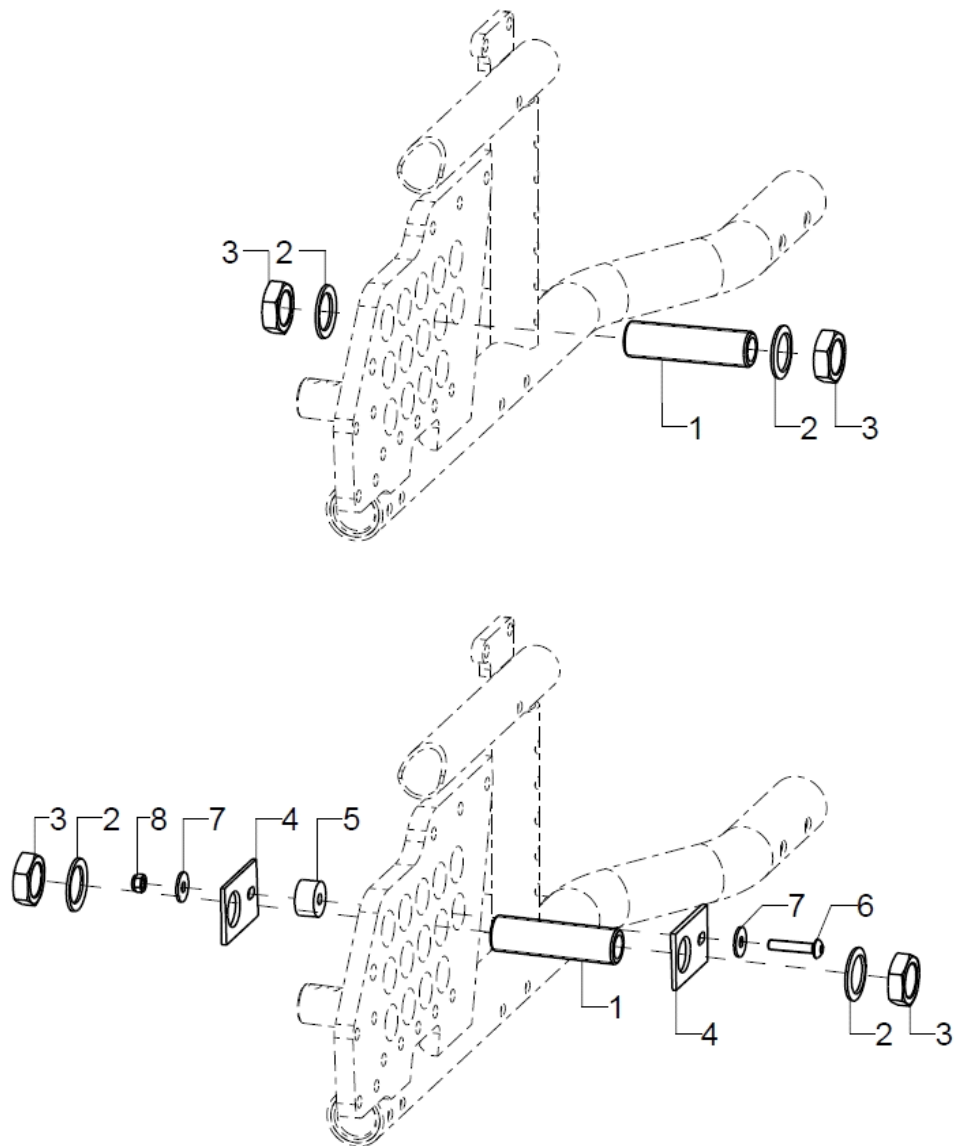


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-3	2	210026	axle adapter M18x1 d12,7 65mm incl. mounting set		04.51.61.26.52.02
2	2		lock washer S18		02.99.11.80.01.10
3	2		hexagon nut M18x1		02.19.01.30.11.21
1-3	2	210026	axle adapter M18x1 d12,7 65mm incl. mounting set		04.51.61.26.52.02
2	2		lock washer S18		02.99.11.80.01.10
3	2		hexagon nut M18x1		02.19.01.30.11.21
4-8	2	201515			20.15.76..01.00.76
5	1				20.15.76.02.00.76
6	1		lens head screw with hexagon socket ISO 7380 - M5x25		01.42.07.96.65.00
7	2		washer DIN 9021 - M5		02.79.05.31.20.20
8	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20

Drum brake anchor

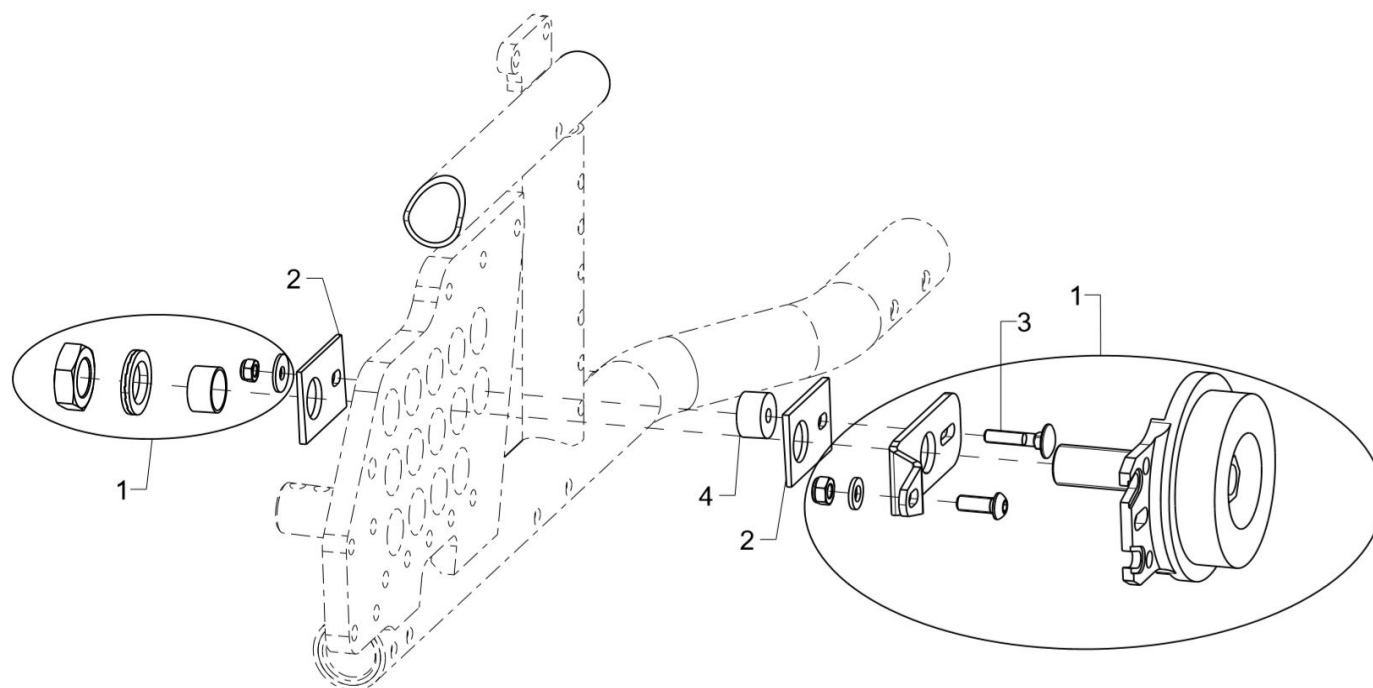

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	2	220005	drum brake body complete, incl. adapter and counter wrench		22.00.01.99.00.10
o.B.	2	220013	drum brake anchor pre-assembled		
2-4	2	201515			20.15.76..01.00.76
3	1				01.11.07.94.00.40
4	1				20.15.76.02.00.76

Rear wheels

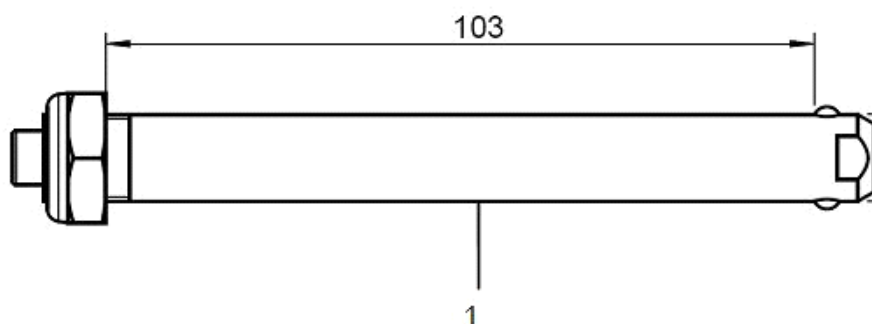

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	2	212000	driving wheel 20" with standard hub		
o.B.	2	212200	22" driving wheel with standard hub		
o.B.	2	212400	24" driving wheel standard hub		
o.B.	2	222000	20" driving wheel with drum brake		
o.B.	2	222200	22" driving wheel with drum brake		
o.B.	2	222400	24" driving wheel standard with drum brake		
o.B.	2	212040	20" profile wheel with built-in hand rim, standard hub		
o.B.	2	212240	22" profile wheel with built-in hand rim, standard hub		
o.B.	2	212440	24" profile wheel with built-in hand rim, standard hub		
o.B.	2	222040	20" profile wheel with built-in hand rim with drum brake		
o.B.	2	222240	22" profile wheel with built-in hand rim with drum brake		
o.B.	2	222440	24" profile wheel with built-in hand rim with drum brake		
o.B.	2	212077	20" lightweight wheel with hand rim, standard		
o.B.	2	212277	22" lightweight wheel with hand rim, standard		
o.B.	2	212477	24" lightweight wheel with hand rim, standard		
o.B.	2	212081	20" lightweight wheel with drum brake hub, black		
o.B.	2	212281	22" lightweight wheel with drum brake, black		
o.B.	2	212481	24" lightweight wheel with drum brake, black		
1	2	210003	quick-release axle 12,7x103 mm for standard hub from 5/2013 and light weight wheel		04.51.51.21.00.31
o.B.	2	220001	quick-release axle D12x123mm for drum brake		04.51.51.21.20.30
o.B.	2	200011	groove ball bearing ø28/12 6001z		04.32.12.08.00.10
o.B.	2	210007	groove ball bearing D28/12,7 6001RS		
o.B.	2	210008	groove ball bearing ø28,53/12,7		04.32.13.08.00.10



Tyres


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	2	202016			
o.B.	2	202216			
o.B.	2	202416			
o.B.	2	202212	tire for pneumatic tube 22x1" Marathon plus, black		
o.B.	2	202412	tire for pneumatic tube 24x1" Marathon plus, black		04.61.12.41.20.21
o.B.	2	202021	tire for pneumatic tube 20x1 3/8" grey		04.61.12.00.20.10
o.B.	2	202221	tire for pneumatic tube 22x1 3/8" grey		04.61.12.20.30.10
o.B.	2	202421	tire for pneumatic tube 24x1 3/8" grey		04.61.12.40.30.10
o.B.	2	200520	tube 20"		
o.B.	2	200522	tube 22"		
o.B.	2	200524	tube 24"		
o.B.	2	232013			
o.B.	2	232213			
o.B.	2	232413			
o.B.	2	232014			
o.B.	2	232214			
o.B.	2	232414			
o.B.	2	232012			04.61.22.00.10.11
o.B.	2	232212			04.61.22.20.10.11
o.B.	2	232412			04.61.22.40.10.11



Handrims

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	2	302101	handrim 20" high, stainless steel polished		
o.B.	2	302201	handrim 22" standard, stainless steel polished		
o.B.	2	302301	handrim 22" high, stainless steel polished		
o.B.	2	302401	handrim 24" standard, stainless steel polished		
o.B.	2	302501	handrim 24" high, stainless steel polished		
o.B.	2	3420	Maxgrepp-handrim 20", 4 loops complete		
o.B.	2	3422	Maxgrepp-handrim 22", 4 loops complete		
o.B.	2	3424	Maxgrepp-handrim 24", 4 loops complete		
o.B.	2	3120	hand rim, stainless steel 20", 8 knobs		
o.B.	2	3122	hand rim, stainless steel 22", 8 knobs		
o.B.	2	3124	hand rim, stainless steel 24", 8 knobs		
o.B.	2	3320	hand rim aluminium, 20", black, pvc coating		
o.B.	2	3322	hand rim aluminium, 22", black, pvc coating		
o.B.	2	3324	hand rim aluminium, 24", black, pvc coating		
o.B.	2	342424			
o.B.	2	302078			
o.B.	2	302278			
o.B.	2	302478			

Spoke guards

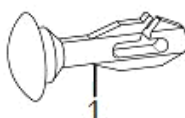

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	250001	clip for transparent spoke cover		25.00.99.01.00.10
o.B.	2	252000	spoke cover 20", Unicolor transparent		
o.B.	2	252200	spoke cover 22", Unicolor transparent		
o.B.	2	252400	spoke cover 24", Unicolor transparent		

Design-Dekor-Speichenschützer können Sie unter den folgenden Internetadressen wählen:


www.speichen-schutz.de
www.wheelchairgadgets.com



Handrim covers


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	2	322002	handrim cover silicone black ø442		04.67.06.20.00.40
o.B.	2	322202	handrim cover silicone black ø478		04.67.06.22.00.30
o.B.	2	322205	handrim cover silicone black ø453		04.67.06.22.00.20
o.B.	2	322208	handrim cover silicone black ø388		04.67.06.22.00.10
o.B.	2	322402	handrim cover silicone black ø532		04.67.06.24.00.30
o.B.	2	322404	handrim cover silicone black ø504		04.67.06.24.00.10
o.B.	2	322405	handrim cover silicone black ø514		04.67.06.24.00.20
o.B.	2	322603	handrim cover silicone black ø565		04.67.06.24.00.40
o.B.	2	322001	handrim cover profile wheel, Ergo 20“		
o.B.	2	322201	handrim cover profile wheel, Ergo 22“		
o.B.	2	322401	handrim cover profile wheel, Ergo 24“		

Double handrim Standard

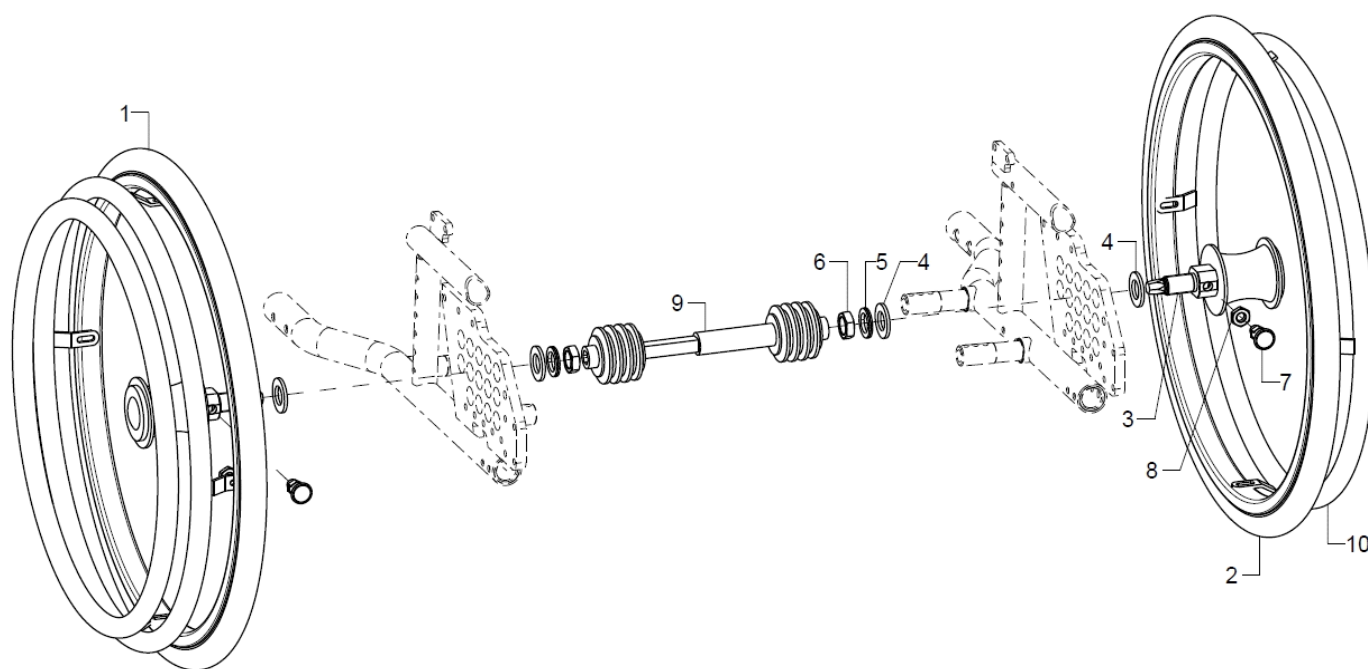

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-9	1	261401	double handrim 20" standard hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXXXX
o.B.	1	261403	double handrim 22" standard hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXXXX
o.B.	1	261405	double handrim 24" standard hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXXXX
1+3-9	1	261440	20" double handrim driving wheel standard hub, driving side		XXXXXXXXXXXX
o.B.	1	261442	22" double handrim driving wheel standard hub, driving side		XXXXXXXXXXXX
o.B.	1	261444	24" double handrim driving wheel standard hub, driving side		XXXXXXXXXXXX
2-8	1	261441	20" double handrim driving wheel standard hub, opposite side		XXXXXXXXXXXX
o.B.	1	261443	22" double handrim driving wheel standard hub, opposite side		XXXXXXXXXXXX
o.B.	1	261445	24" double handrim driving wheel standard hub, opposite side		XXXXXXXXXXXX
3-8	1	261429	mounting set DHR drum brake 3-8		XXXXXXXXXXXX
3	1				
4	1		washer DIN 125 - B 19		02.61.19.02.72.20
5	1		lock washer M18 (Nord-Lock)		02.99.21.95.01.10
6	1		hexagon nut M18x1		02.19.01.30.11.21
7	1				04.14.34.50.01.00
8	1		hexagon nut DIN 934 - M10		02.19.01.00.01.20
9	1	26140004			26.14.10.01.04.00
o.B.	1	26140005			26.14.10.01.05.00
o.B.	1	26140006			26.14.10.01.06.00
o.B.	1	26140007			26.14.10.01.07.00
o.B.	1	261431	hand rim for other side 22"		XXXXXXXXXXXX
o.B.	1	26140008			26.14.10.01.08.00
10	1	261430	hand rim for other side 20"		XXXXXXXXXXXX
o.B.	1	261432	hand rim for other side 24"		XXXXXXXXXXXX

Double handrim Drum brake

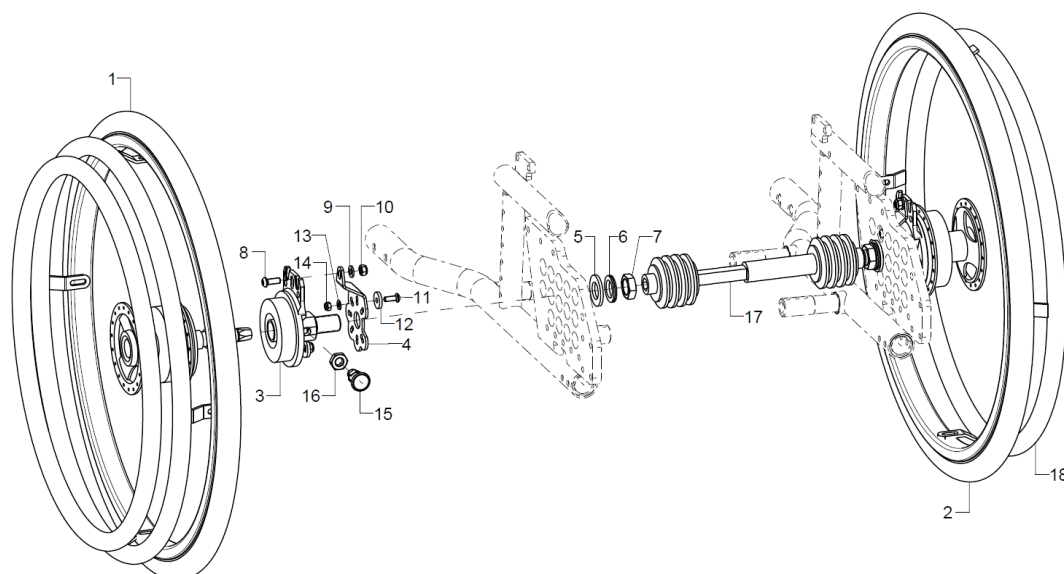

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-17	1	261402	double handrim 20" drum brake hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXX
o.B.	1	261404	double handrim 22" drum brake hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXX
o.B.	1	261406	double handrim 24" drum brake hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXX
1	1	261450	20" double handrim driving wheel drum brake hub, driving side		XXXXXXXXXX
o.B.	1	261452	22" double handrim driving wheel drum brake hub, driving side		XXXXXXXXXX
o.B.	1	261454	24" double handrim driving wheel drum brake hub, driving side		XXXXXXXXXX
2	1	261451	20" double handrim driving wheel drum brake hub, opposite side		XXXXXXXXXX
o.B.	1	261453	22" double handrim driving wheel drum brake hub, opposite side		XXXXXXXXXX
o.B.	1	261455	24" double handrim driving wheel drum brake hub, opposite side		XXXXXXXXXX
3-16	1	261428	mounting set DHR drum brake		XXXXXXXXXX
3	1				
4	1	220012	holder other side drum brake		
5	1		washer DIN 125 - B 19		02.61.19.02.72.20
6	1		lock washer M18 (Nord-Lock)		02.99.21.95.01.10
7	1		hexagon nut M18x1		02.19.01.30.11.21
8	1		lens head screw with hexagon socket ISO 7380 - M6x16		01.42.08.52.65.00
9	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
10	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
11	1				01.43.07.90.65.00
12	1				22.00.12.02.00.10
13	1		washer DIN 125 - A 5,3		02.61.05.31.01.20
14	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
15	1				04.14.34.50.01.00
16	1		hexagon nut DIN 934 - M10		02.19.01.00.01.20
17	1	26140004			26.14.10.01.04.00
o.B.	1	26140005			26.14.10.01.05.00
o.B.	1	26140006			26.14.10.01.06.00
o.B.	1	26140007			26.14.10.01.07.00
o.B.	1	26140008			26.14.10.01.08.00
18	1	261430	hand rim for other side 20"		XXXXXXXXXX
o.B.	1	261431	hand rim for other side 22"		XXXXXXXXXX




o.B.	1	261432	hand rim for other side 24"	XXXXXXXXXX


Knee lever brake

Please select the required mounting variant for each brake variant.


Bremsenvarianten

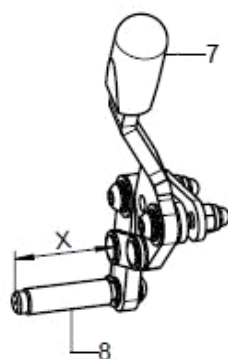


1
KHB Standard

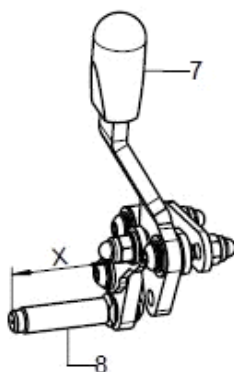


2
KHB Pull to Lock

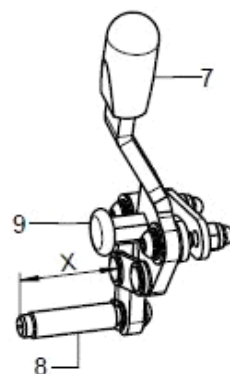

3
KHB verriegelbar

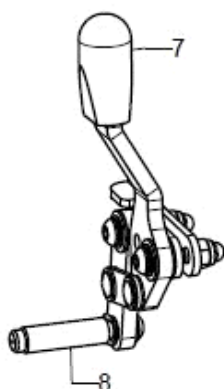


4
KHB mit reduziertem
Hebelweg

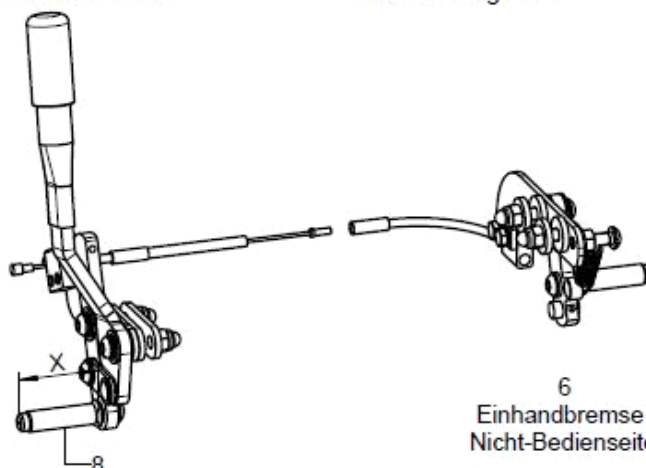

5
Einhandbremse Bedienseite

6
Einhandbremse
Nicht-Bedienseite


Anbauvarianten

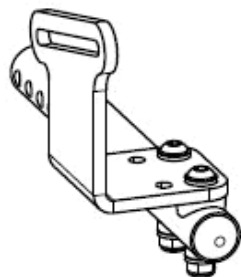


Bei offenem Rahmen
mit/ohne Radstands-
verlängerung




Bei geschlossenem
Rahmen ohne Rad-
standsverlängerung

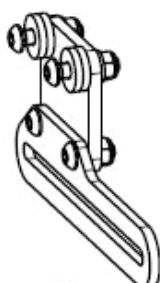

Bei geschlossenem
Rahmen + Doppelgreif-
ring + Einhandbremse
ohne Radstandsver-
längerung

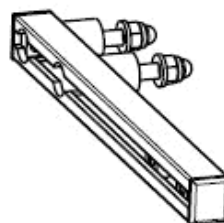

Bei geschlossenem
Rahmen mit Rad-
standsverlängerung

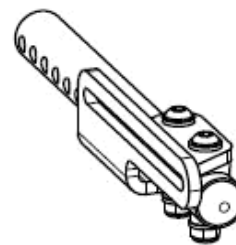


Bei kleinen ST, offenem
Rahmen, ohne Radstands-
verlängerung
20" bis ST32
22" bis ST34
24" bis St36




pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1		35251601			
1		35251602			
1					
1					
1		35251603			
1+7+8	1	352502	knee lever brake standard right complete		XXXXXXXXXXXXX
o.B.	1	352501	knee lever brake standard left complete		XXXXXXXXXXXXX
2+7+8	1	352902	knee lever brake pull-to-lock right complete		XXXXXXXXXXXXX
o.B.	1	352901	knee lever brake pull-to-lock left complete		XXXXXXXXXXXXX
3+7-9	1	351302	knee lever brake lockable complete right		XXXXXXXXXXXXX
o.B.	1	351301	knee lever brake lockable complete left		XXXXXXXXXXXXX
4+7+8	1	352532			
o.B.	1	352531			
5+6+8	1	350902	one-hand brake right, complete user side + non-operating side		XXXXXXXXXXXXX
o.B.	1	350901	one-hand brake left, complete user side + non-operating side		XXXXXXXXXXXXX
5+8	1	350904	one-hand brake, operating side right		XXXXXXXXXXXXX
o.B.	1	350903	one-hand brake, operating side left		XXXXXXXXXXXXX
6+8	1	350912	one-hand brake non-operating side complete		XXXXXXXXXXXXX
7	2	350017	cap black 15x6,8		35.00.17.01.00.10
8	1	352518	brake bolt 40 mm left-hand thread incl. mounting set		35.25.01.18.00.10
o.B.	1	352506	brake bolt milled 40 mm incl. mounting set		35.25.01.15.00.10
o.B.	1	352519	brake bolt 50 mm left-hand thread incl. mounting set		35.25.01.19.00.10
o.B.	1	352507	brake bolt milled 50 mm incl. mounting set		35.25.01.16.00.10
9	1	350503	latch 6/M10, red lockable		04.14.43.60.02.00

Knee lever brake for 12 inch wheels

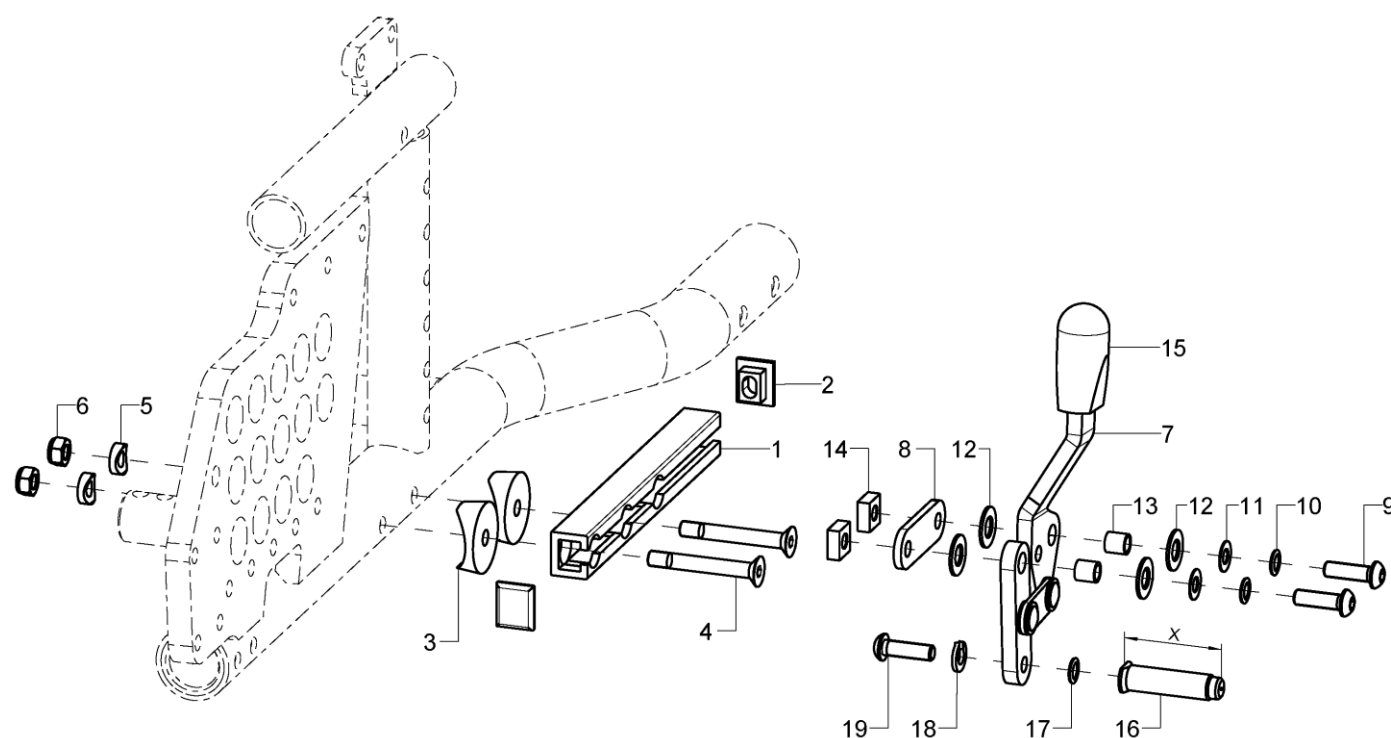


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-19	1	35251522			XX.XX.XX.XX.XX.XX
1-6	2	35251510			35.25.15.10.00.76
2	2				35.01.02.05.00.10
3	2				04.63.39.28.06.30
4	2		counter sunk screw with hexagon socket DIN 7991 M6X45		01.65.08.61.05.00
5	2		saddle washer Ø12/6x2,5		35.04.16.01.00.10
6	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
7	2	352504	brake pad standard pre-mounted right		35.25.01.02.02.10
o.B.	2	352503	brake pad standard pre-mounted left		35.25.01.02.01.10
8	2	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
9-14	2	352590			XX.XX.XX.XX.XX.XX
9	2		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
10	2		lock washer S6		02.99.10.60.01.10
11	2		fitted washer DIN 988 - 12x18x0,5		02.68.12.00.50.30
12	4		fender disc 8,4x16x1 PE		02.81.16.00.10.70
13	2				35.00.18.01.00.30
14	2		square nut SW 12x5-M6		02.51.00.60.01.20
15	2	350017	cap black 15x6,8		35.00.17.01.00.10
16-19	1	352518	brake bolt 40 mm left-hand thread incl. mounting set		35.25.01.18.00.10
o.B.	1	352506	brake bolt milled 40 mm incl. mounting set		35.25.01.15.00.10
altern.	1	352519	brake bolt 50 mm left-hand thread incl. mounting set		35.25.01.19.00.10
o.B.	1	352507	brake bolt milled 50 mm incl. mounting set		35.25.01.16.00.10
17	1		lock washer S6		02.99.10.60.01.10
18	1		spring ring DIN 127- A6		02.85.30.60.05.10
19	1		lens head screw with hexagon socket ISO 7380 , M6x20 left-hand thread		01.42.08.54.05.01
o.B.	1		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00

Brake lever extension

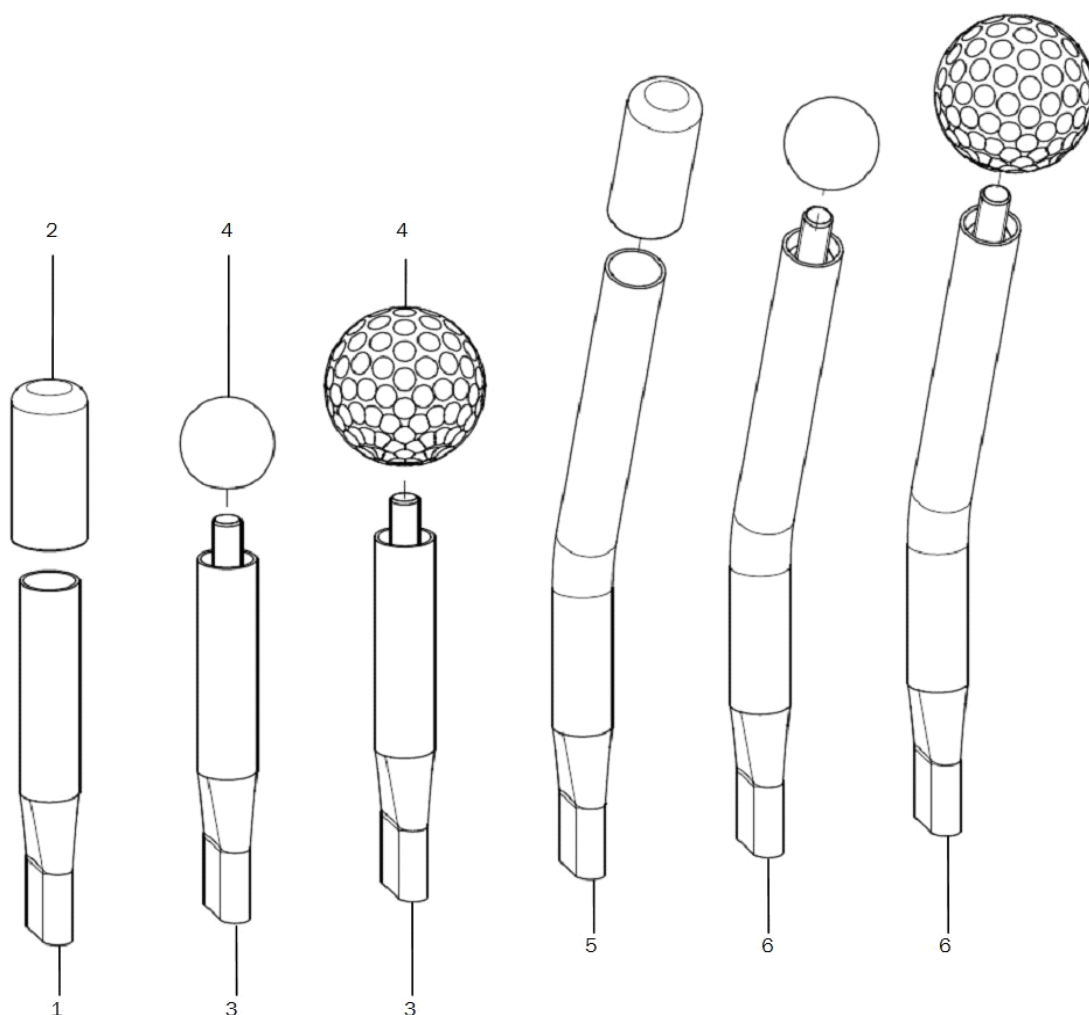


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-2	2	3510	brake lever extension with cap		35.01.03.02.00.10
2	2	351011	PVC cap one-hand brake		04.63.00.16.40.40
3-4	2	3512	brake lever extension 100 for ball knob		35.14.03.01.00.30
4	2	351016	ball knob		04.06.12.50.80.40
altern.	2	351211	golf ball		04.06.14.20.80.00
5+2	2	3515	brake lever extension 23° with cap		35.15.03.01.00.10
6+4	2	3516	brake lever extension 23° for ball knob		35.16.03.01.00.10

Braking handle drum brake

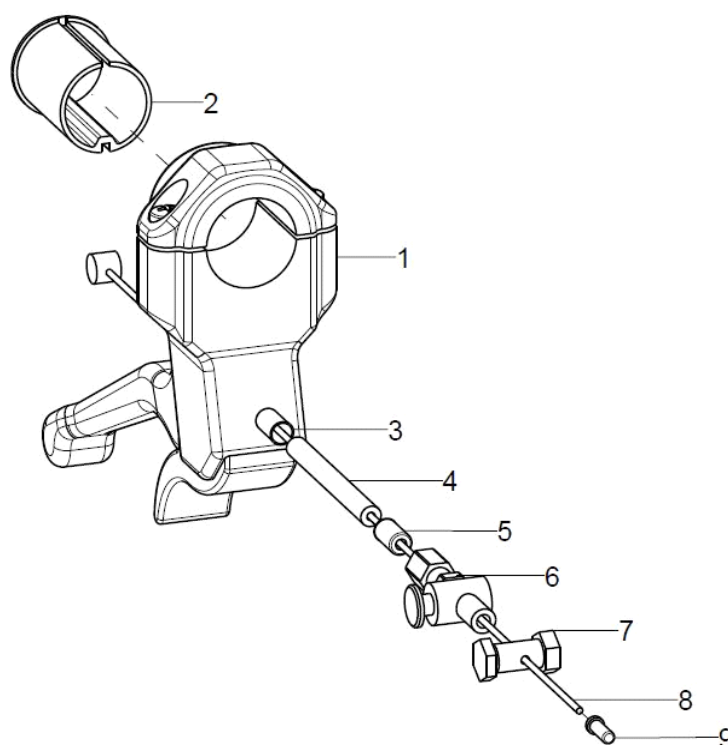

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-9	2	352016	hand brake lever complete		XXXXXXXXXXXX
1+2	1	352010	hand brake lever for attendant brake incl. mounting set		04.54.92.22.20.10
2	1	352011			35.20.11.01.00.28
3-9	2	352006			35.20.06.99.00.10
3	1				04.54.91.50.11.21
4	1		outer shell for Bowden cable black 150mm		04.54.91.21.53.00
5	1		end bush metal ø5		04.54.91.50.10.41
6	1		set screw for Bowden cable M6x40 mm		04.54.01.40.60.20
7	1		clamp screw for bowden cable		04.54.11.30.00.20
8	1		inner Bowden cable 1,5x2250		04.54.91.11.52.20
9	1		end cap for inner Bowden cable ø1-1,5 mm		04.54.91.50.10.40

2-point Hip belt with Velcro and synthetic lock

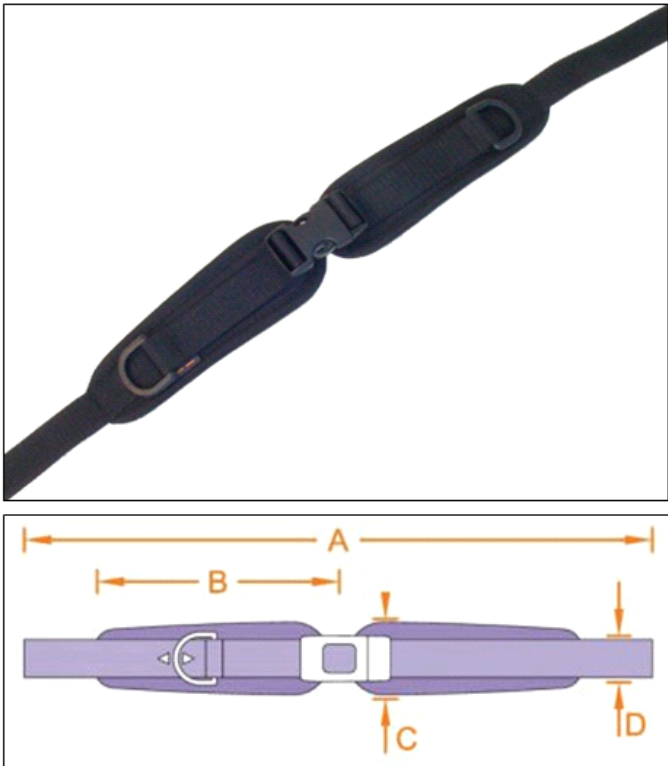

Größe	A	B	C	D
1	148	14	5	2,5
2	156	17	6	2,5
3	166	22	7	2,5

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	78050201			04.56.91.05.01.00
o.B.	1	78050202			04.56.91.05.02.00
o.B.	1	78050203			04.56.91.05.03.00
o.B.	1	780107			78.01.06.02.01.00


2-point Hip belt with metal lock



Größe	A	B	C	D
1	148	14	5	2,5
2	156	17	6	2,5
3	166	22	7	2,5

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	78050204			04.56.91.06.01.00
o.B.	1	78050205			04.56.91.06.02.00
o.B.	1	78050206			04.56.91.06.03.00
o.B.	1	78050210			04.56.91.01.01.11
o.B.	1	780107			78.01.06.02.01.00


4-point Hip belt with Velcro and synthetic lock

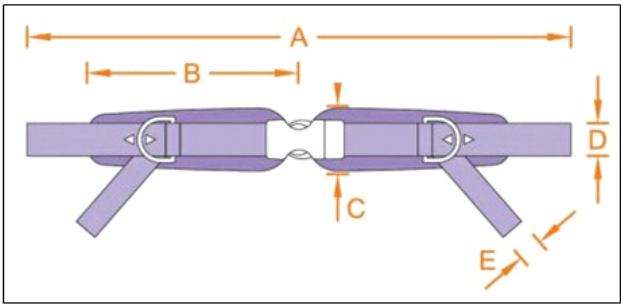

Größe	A	B	C	D	E
1	148	14	5	2,5	2,5
2	156	17	6	2,5	2,5
3	166	22	7	2,5	2,5

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	78050401			04.56.91.07.01.00
o.B.	1	78050402			04.56.91.07.02.00
o.B.	1	78050403			04.56.91.07.03.00
o.B.	1	780107			78.01.06.02.01.00


4-point Hip belt with metal lock

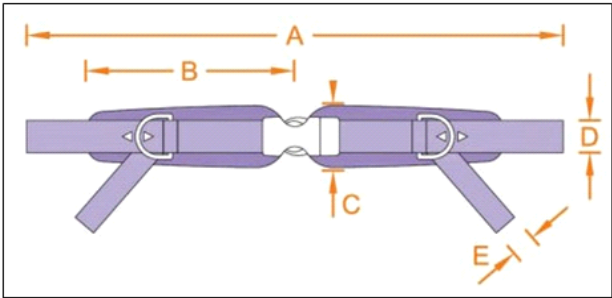

Größe	A	B	C	D	E
1	148	14	5	2,5	2,5
2	156	17	6	2,5	2,5
3	166	22	7	2,5	2,5

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	78050404			04.56.91.08.01.00
o.B.	1	78050405			04.56.91.08.02.00
o.B.	1	78050406			04.56.91.08.03.00
o.B.	1	78050210			04.56.91.01.01.11
o.B.	1	780107			78.01.06.02.01.00

Butterfly harness

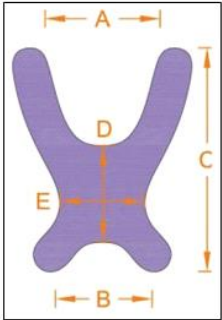

Größe	A	B	C	D	E
1	9	21	27	10	12
2	12	24	36	14	14
3	17	30	42	15	18
4	20	32	48	17	21

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	781001			04.56.92.05.01.00
o.B.	1	781002			04.56.92.05.02.00
o.B.	1	781003			04.56.92.05.03.00
o.B.	1	781004			04.56.92.05.04.00
o.B.	1	780107			78.01.06.02.01.00